M-LNCS®/LNCS®/ LNOP®

Adhesive Replacement Tapes

DIRECTIONS FOR USE

- Single Patient Use Only
- Not made with natural rubber latex
- Non-sterile

DESCRIPTION

The M-LNCS®/LNCS®/ LNOP® adhesive replacement tapes are used to replace the attachment tape portion of the M-LNCS/LNCS /LNOP adhesive sensors in the event that the tape becomes soiled or torn prior to the end of sensor life.

- INSTRUCTIONS**
1. Remove the existing tape and discard (Fig. 1).
 2. Remove the replacement tape from the release liner (Fig. 2).
 3. Position the replacement tape over the sensor, aligning the detector component with the sensor cable (Fig. 3).

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Follow instructions for use		Lot code		Caution: Federal law (USA) restricts this device to sale by or on the order of a physician
	Consult instructions for use		Catalogue number (model number)		Mark of conformity to European Medical Device Directive 93/42/EEC
	Manufacturer		Masimo reference number		Authorized representative in the European community
	Date of manufacture YYYY-MM-DD		Do not use if package is damaged		Keep dry
	Do not re-use/Single patient use only		Not made with natural rubber latex		Non-sterile

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