M-LNCS™/LNCS®/LNOP®
SofTouch™ Replacement Wraps

en

DIRECTIONS FOR USE

- Single patient use only
- Not made with natural rubber latex
- Non-sterile

DESCRIPTION
The M-LNCS™/LNCS®/LNOP® SofTouch™ replacement wraps are used to replace the attachment wrap portion of the M-LNCS/LNCS/LNOP adhesive sensors in the event that the wrap becomes soiled or torn prior to the end of sensor life.

- INSTRUCTIONS
- Remove the existing wrap and discard (Fig. 1).
 - Remove the backing from the wrap (Fig. 2).
 - Position the replacement wrap over the sensor and press in place (Fig. 3).
- The following symbols may appear on the product or product labeling:

Table with 4 columns: SYMBOL, DEFINITION, SYMBOL, DEFINITION. It lists various symbols used on the product, including instructions for use, CE mark, EC REP, manufacturer, lot code, use-by date, catalogue number, patient type, storage conditions, and atmospheric pressure limitation.

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Table with 4 columns: SIMBOLO, DEFINIZIONE, SIMBOLO, DEFINIZIONE. It lists various symbols used on the product, including instructions for use, CE mark, EC REP, manufacturer, lot code, expiration date, catalogue number, patient type, storage conditions, and atmospheric pressure limitation.

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Table with 4 columns: SYMBOLE, DÉFINITION, SYMBOLE, DÉFINITION. It lists various symbols used on the product, including instructions for use, CE mark, EC REP, manufacturer, lot code, date of expiry, catalogue number, patient type, storage conditions, and atmospheric pressure limitation.

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Table with 4 columns: SÍMBOLO, DEFINICIÓN, SÍMBOLO, DEFINICIÓN. It lists various symbols used on the product, including instructions for use, CE mark, EC REP, manufacturer, lot code, expiration date, catalogue number, patient type, storage conditions, and atmospheric pressure limitation.

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


Table with 4 columns: SYMBOL, DEFINITION, SYMBOL, DEFINITION. It lists various symbols used on the product, including instructions for use, CE mark, EC REP, manufacturer, lot code, use-by date, catalogue number, patient type, storage conditions, and atmospheric pressure limitation.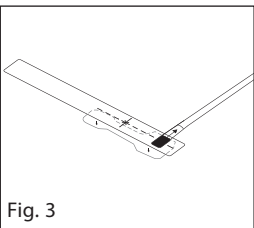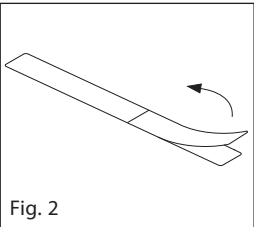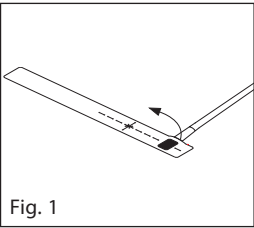

M-LNCS™/LNCS®/LNOP®
SofTouch™ Replacement Wraps

Authorized Representative
in the European community
EC REP

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© 2016 Masimo Corporation.