emergency

BASE STATION 1 NG

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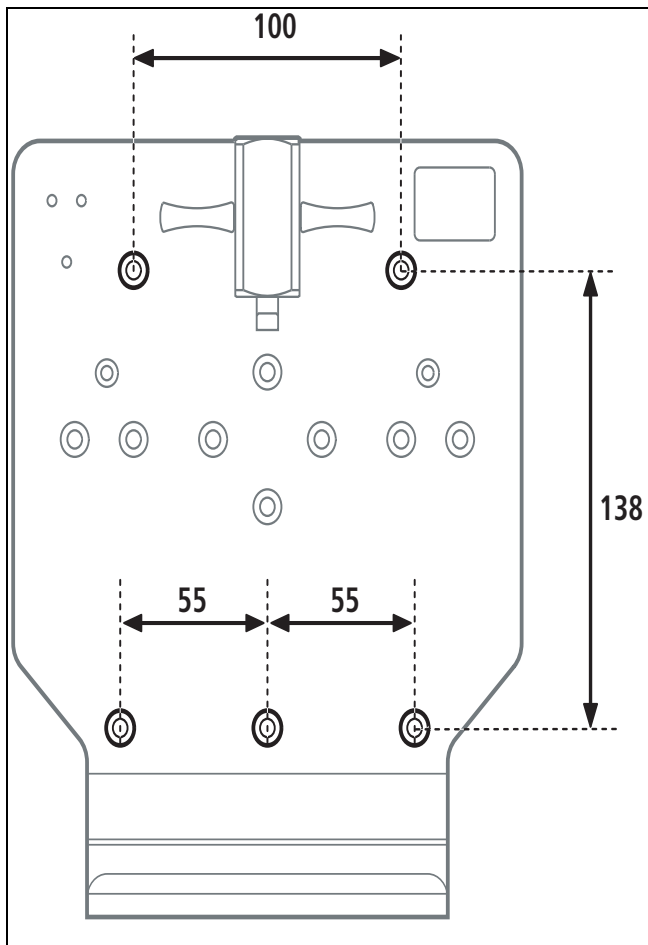
These assembly instructions describe how to fit the BASE-STATION 1 NG wall mounting. The wall mounting is an accessory for the following portable systems:

- LIFE-BASE 1 NG
- LIFE-BASE 1 NG XL
- LIFE-BASE 1 NG XS

Read the instructions for use for these portable systems (WM 67060), they constitute part of these assembly instructions.

The scope of supply includes the parts required to attach the wall mounting to the wall. Using additional attachment sets, you can also fit the wall mounting to a standard hospital rail (attachment set WM 15805) or to an ordinary rail (attachment set WM 15806). The relevant assembly instructions describe how to assemble the attachment sets.

Safety information

Warning!

- Only the LIFE-BASE 1 NG/LIFE-BASE 1 NG XL/LIFE-BASE 1 NG XS portable system may be suspended from the wall mounting. The weight and dimensions of this portable system are exactly matched to the wall mounting. Other portable systems may not be used.
- Suspending third-party products or other makes of adapter is not permitted as this does not comply with the use in accordance with purpose of this wall mounting. Refer to the instructions for use of the LIFE-BASE 1 NG/LIFE-BASE 1 NG XL/LIFE-BASE 1 NG XS portable system.

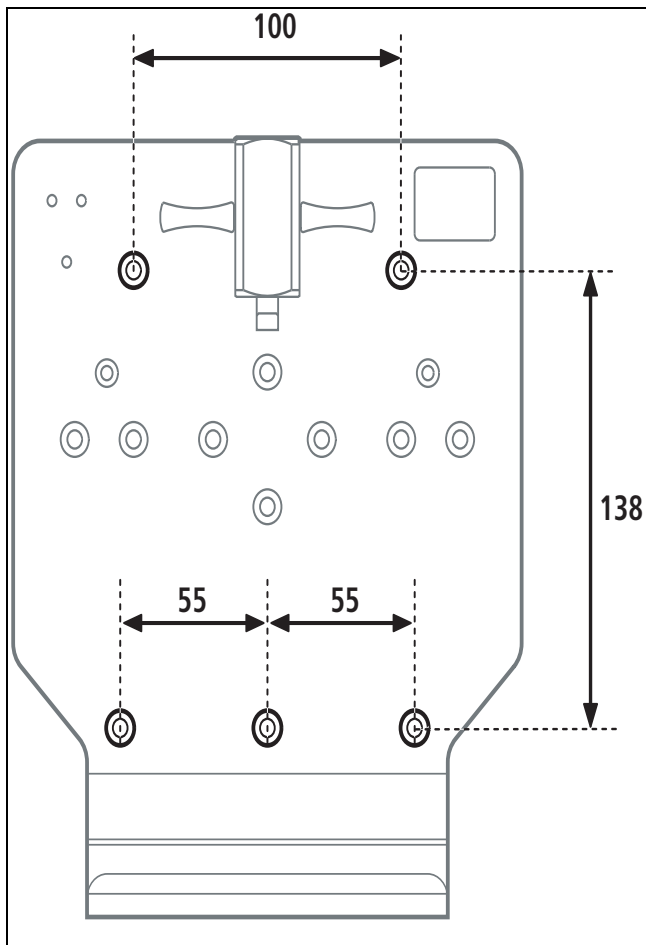
Note

- The surface for attachment must be sufficiently strong to hold the wall mounting and the portable system securely in accordance with DIN EN 1789 (ambulances) and/or RTCA DO 160E.
- The charger interface connection (optional) to the 12V on-board electricity supply must be fused with an 8A fuse.
- The wall mounting has been tested exclusively with M5 mounting screws to EN ISO 4762 (strength 8.8). If other screws are used, ensure that they are of equivalent or greater strength.
- The fitter is responsible for the fitting of the wall mounting.

Fitting to a wall

1. Look for a suitable place to mount the system, somewhere **flat and vertical**.
2. Mark 5 boreholes where you want to fix the system, as indicated in the drawing on the next page.
Note: You do not need all the bores in the wall mounting to attach it to a wall. The drawing on the next page shows you which bores you have to use and how they are spaced.
3. Drill the holes using a Ø 4.2 mm bit.
4. Insert 5 M5 threads in the substrate.
5. Screw on the wall mounting using the screws supplied. The portable system must be fitted horizontally.
6. If present, connect the charger interface to the 12V on-board electricity supply of the emergency vehicle using the connector for vehicles.

Note: you can also connect the charger interface directly to a 12 V on-board electricity supply or fit an alternative 12 V connector to the charger interface. To do this, please follow assembly instructions WM 67500.

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Medical Technology GmbH + Co. KG**