en

RD SET™

Adhesive Replacement Tapes

DIRECTIONS FOR USE			
 Single patient use only	 Not made with natural rubber latex	 Non-Sterile	

DESCRIPTION
The RD SET™ adhesive replacement tapes are used to replace the attachment tape portion of the RD SET disposable sensors in the event that the tape becomes soiled or torn prior to the end of the sensor life.

- INSTRUCTIONS**
- Remove the existing tape and discard (Fig. 1).
 - Remove the replacement tape from the release liner (Fig. 2).
 - Position the replacement tape over the sensor, aligning the emitter component with the sensor cable (Fig. 3).
- The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Consult instructions for use		Mark of Conformity to European Medical Device Directive 93/42/EEC
 (blue background)	Follow instructions for use		Authorized representative in the European community
	Manufacturer		Lot code
	Use-by date YYYY-MM-DD		Catalogue number (model number)
	Do not re-use/Single patient use only		Pediatric patient
	Non-Sterile		Storage humidity limitation
	Not made with natural rubber latex		Storage temperature range
	Do not use if package is damaged		Keep dry
	Masimo reference number		Atmospheric pressure limitation
Rx ONLY	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.		Instructions/Directions for Use/Manuels are available in electronic format @ http://www.masimo.com/TechDocs Note: eIFU is not available for CE mark countries.

RD SET is a trademark of Masimo Corporation.

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© 2017 Masimo Corporation.