RD rainbow SET™

Adhesive Replacement Tapes

en

DIRECTIONS FOR USE

- Single patient use only Not made with natural rubber latex Non-sterile

DESCRIPTION

The RD rainbow SET™ adhesive replacement tapes are used to replace the attachment tape portion of the RD rainbow SET infant (Inf) and neonatal (Neo) adhesive sensors in the event that the tape becomes soiled or torn prior to the end of sensor life.

INSTRUCTIONS

1. Remove the existing tape and discard (Fig. 1).
2. Remove the replacement tape from the release liner (Fig. 2).
3. Position the replacement tape over the sensor and press in place (Fig. 3).

The following symbols may appear on the product or product labeling:

SYMBOL	DEFINITION	SYMBOL	DEFINITION	SYMBOL	DEFINITION
	Consult instructions for use		Do not use if package is damaged		Pediatric patient
(blue background)	Follow instructions for use		Masimo reference number		Storage humidity limitation
	Manufacturer		Mark of Conformity to European Medical Device Directive 93/42/EEC		Storage temperature range
	Use-by date YYYY-MM-DD		Authorized representative in the European community		Keep dry
	Do not re-use/Single patient use only		Lot code		Atmospheric pressure limitation
	Non-Sterile		Catalogue number (model number)		Caution
	Not made with natural rubber latex	Rx ONLY	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.		
	Instructions/Directions for Use/Manuals are available in electronic format at http://www.Masimo.com/TechDocs Note: eIFU is not available for CE mark countries				

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© 2017 Masimo Corporation.