Mon-a-Therm™

Cable 400TM

en	Instructions For Use
fr	Mode d'emploi
de	Gebrauchsanweisung
nl	Gebruiksaanwijzing
it	Istruzioni per l'uso
es	Instrucciones de uso
sv	Bruksanvisning
da	Betjeningsvejledning
no	Bruksanvisning
fi	Käyttöohjeet
pt	Instruções de utilização
ru	Инструкция по применению
zh	使用说明
pl	Instrukcja użytkowania
cs	Návod k použití
sk	Návod na použitie
sl	Navodilo za uporabo
hu	Használati utasítás
el	Οδηγίες χρήσης
tr	Kullanma Talimatları
ar	تعليمات الاستخدام
bg	Инструкции за употреба
ro	Instrucțiuni de utilizare
et	Kasutusjuhend
lt	Naudojimo instrukcijos
lv	Lietošanas instrukcija
hr	Upute za uporabu
sr	Uputstvo za upotrebu
mk	Упатство за употреба

REF	502-0400A
REF	502-0401A
REF	502-0405A
REF	502-0410A
REF	502-0411A
REF	502-0415A

Mon-a-Therm™

Cable 400TM

Identification of a substance that is contained or present within the product or packaging.

Identification of a substance that is not contained or present within the product or packaging.

Follow instructions for use

This product contains DEHP. The intended use limits exposure to intact skin, minimizing the risk of DEHP absorption. In order to avoid undue risk of DEHP exposure in children and nursing or pregnant women, product should only be used as directed.

Description

The Mon-a-Therm™ cable 400TM is a reusable interface cable for use with Mon-a-Therm™ disposable probes and sensors using 400 series thermistors only.

Directions For Use

1. Plug the Mon-a-Therm™ cable 400TM into the instrument's mating receptacle.
2. Connect a compatible disposable Mon-a-Therm™ temperature probe to the cable. Refer to the probe's package insert for DIRECTIONS FOR USE.

Cleaning

The Mon-a-Therm™ cable may be surface-cleaned with a solution such as 70% isopropyl alcohol. If low-level disinfection is required, use a 1:10 bleach solution. Do not use undiluted bleach (5% - 5.25% sodium hypochlorite) or any cleaning solution other than those recommended here because permanent damage to the cable could occur.

CAUTIONS

- Do not sterilize. This cable is not designed for sterilization, including irradiation, or exposure to steam or ethylene oxide.
- Do not expose the connector pins to cleaning solution as this may damage the cable.
- Do not immerse the cable in liquids for cleaning.

To clean the cable:

1. Remove the cable from the Mon-a-Therm™ probe and patient monitor.
2. Saturate a clean gauze pad with 70% isopropyl alcohol. Wipe all surfaces of the cable with the pad, being careful not to apply excessive force to the cable, so as not to stretch and break internal wires. Allow the alcohol to remain on the surface of the cable for at least 1 minute.
3. Saturate another gauze pad with sterile or distilled water. Wipe all surfaces of the cable with the pad.
4. Dry the cable by wiping all surfaces with a clean, dry gauze pad.
5. Inspect the cable. If the cable is not visually clean, repeat the cleaning steps or dispose of the cable. If the cable or pins are damaged, dispose of the cable.

To disinfect the cable (low-level disinfection):

1. Saturate a clean gauze pad with the diluted bleach solution. Wipe all surfaces of the cable with the pad, being careful not to apply excessive forces to the cable which can stretch and break internal wires. Allow the bleach solution to remain on the surface of the cable for at least 1 minute.
2. Saturate another gauze pad with sterile or distilled water. Wipe all surfaces of the cable with the pad.
3. Dry the cable by wiping all surfaces with a clean, dry gauze pad.

WARNINGS

- As with any temperature monitoring system, use acceptable techniques during electrosurgical procedures to help reduce the possibility of the cable serving as an alternate path for radio frequency current and burning the surrounding tissues. Proper techniques at a minimum should include:
 - Properly locate and connect an adequate electrosurgical dispersive electrode close to the active surgical site.
 - Avoid draping the patient temperature monitor instrument cable over grounded surfaces or intertwining it with the cables of the electrosurgical generator.
- The operator is responsible for checking the compatibility of the monitoring equipment, the interface cable and the probe or sensor before use.
- Incompatible components can result in degraded performance.
- Do not use if package is opened or damaged.
- Do not use this device during MRI scanning. Conducted current may cause burns.

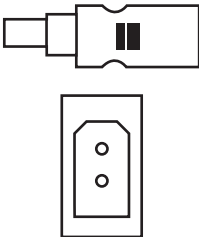
CAUTIONS

- Do not attempt to force connection of the sensor. The connectors are keyed by shape. Forcing the connection could result in damage to the sensor or cable connector and a resulting inability to function.
- Never pull directly on the cable wire to disconnect.
- Ensure that the cable connector does not lie in an area where solutions may pool during the surgical procedure.
- This interface cable is designed for use with specific monitoring equipment. Refer to the compatible Monitor and Sensor section for the cable desired.

Compatible Patient Monitors

Product Code	Cable Length	Monitor Compatibility	Monitor Connector
502-0400A	16 feet/4.87m	Most patient monitors using series 400 thermistor sensors, such as Mindray Passport2™** Monitor.	 1/4 inch, right angle phono plug
502-0410A	10 feet/3.05m		
502-0405A	16 feet/4.87m	Siemens Monitor series SC	 7 pin connector
502-0415A	10 feet/3.05m		
502-0401A	16 feet/4.87m	HP or Philips Monitors such as Philips Intellivue™** Monitor.	 2 pin connector
502-0411A	10 feet/3.05m		

Compatible Mon-a-Therm™ 400 series Probes and Sensors

Product Code	Size	Description	Connector
90049	9 Fr	Esophageal Stethoscope with Temperature sensor	
90041	12 Fr		
90042	18 Fr		
90043	24 Fr		
90045	N/A	Skin Sensor	
90050	9 Fr	General Purpose Probe	
90044	12 Fr		
90053	8 Fr	Foley Catheter with Temperature sensor	
90054	10 Fr		
90055	12 Fr		
90056	14 Fr		
90051	16 Fr		
90052	18 Fr		

Disposal

Dispose of the cable in accordance with applicable regulations for biologically hazardous waste.

Storage and Shipping Conditions

- Temperature: -40°C to 70°C
- Humidity: 15% to 95% (non-condensing)

Protect from moisture.

Additional Copies of Instructions

Additional copies of these instructions can be obtained through your authorized distributor. In addition, permission is hereby granted under Covidien copyrights to purchasers of products obtained from an authorized distributor to make copies of these instructions for use by such purchasers.

Warranty

To obtain information about a warranty, if any, for this product, contact Covidien's Technical Services Department or your local Covidien representative.

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COVIDIEN, COVIDIEN with logo and Covidien logo are U.S. and international trademarks of Covidien AG.
^{TM*} Third party brands are trademarks of their respective owners. All other brands are trademarks of a Covidien company.
EC REP Covidien Ireland Limited,