LNC™ MP Series Patient Cables

DIRECTIONS FOR USE	
Reusable	 Not made with natural rubber latex
	 Non-sterile

INDICATIONS
The LNC™ MP Series Patient Cables and Masimo Sensors are indicated for the continuous noninvasive monitoring of arterial oxygen saturation (SpO₂) and pulse rate for adult, pediatric, infant and neonatal patients. LNC MP Patient Cables are indicated for use with the following devices and sensors:

When used with Masimo SET™ Technology:

- Masimo SET IntellVue Module
- Masimo SET IntellVue Multi-Measurement Sensor (MMS)
- Masimo SET IntellVue Multi-Measurement Sensor X2 (MMS X2)

Sensors	Weight Range	Saturation Accuracy (70 - 100% SpO ₂)		Pulse Rate Accuracy (25 - 240 bpm)		Low Perfusion Accuracy (70 - 100% SpO ₂)	
		No Motion	Motion	No Motion	Motion	Saturation	Pulse Rate
LNCs Adtu/ Adtu-3	>30kg	± 2%	± 3%	± 3 bpm	± 5 bpm	± 2%	± 3 bpm
LNCs Pdtu/ Pdtu-3	10 - 50 kg	± 2%	± 3%	± 3 bpm	± 5 bpm	± 2%	± 3 bpm
LNCs Infnt-3	3 - 20 kg	± 2%	± 3%	± 3 bpm	± 5 bpm	± 2%	± 3 bpm
LNCs Neo/ Neo-3	<3 kg	± 3%	± 3%	± 3 bpm	± 5 bpm	± 3%	± 3 bpm
	>40 kg	± 2%	± 3%	± 3 bpm	± 5 bpm	± 2%	± 3 bpm
LNCs NeoP/ NeoP-3	<1 kg	± 3%	± 3%	± 3 bpm	± 5 bpm	± 3%	± 3 bpm
LNCs Trauma	>30 kg	± 2%	± 3%	± 3 bpm	± 5 bpm	± 2%	± 3 bpm
LNCs Neonatal	<1 kg	± 3%	± 3%	± 3 bpm	± 5 bpm	± 3%	± 3 bpm
LNCs Neonborn Infant/ Pediatrics	3-10 kg 10-30 kg	± 2%	± 3%	± 3 bpm	± 5 bpm	± 2%	± 3 bpm
LNCs DCI	>30 kg	± 2%	± 3%	± 3 bpm	± 5 bpm	± 2%	± 3 bpm
LNCs DCP	10 - 50 kg	± 2%	± 3%	± 3 bpm	± 5 bpm	± 2%	± 3 bpm
LNCs TC-i	>30 kg	± 3.5%	N/A	± 3 bpm	N/A	± 3.5%	± 3 bpm
LNCs TF-i	>30 kg	± 2%	N/A	± 3 bpm	N/A	± 2%	± 3 bpm
	1 - 3 kg	± 3%	± 3%	± 3 bpm	± 5 bpm	N/A	N/A
LNCs YI	>3 kg	± 2%	± 3%	± 3 bpm	± 5 bpm	N/A	N/A

The LNCs TF-i and TC-i sensors were not validated under motion conditions.

The LNCs YI was not validated under low perfusion conditions.

When used with Philips FAST SpO₂ Technology:

- FAST-SpO₂ IntellVue Module, M1020B A01
- FAST-SpO₂ M3/M4 Multi-Measurement Sensor (MMS), M3000A A01
- FAST-SpO₂ Multi-Measurement Sensor (MMS), M3010 A A01
- FAST-SpO₂ IntellVue Multi-Measurement Sensor X2 (MMS X2) M3002A A01
- FAST-SpO₂ IntellVue MP2, M8102A
- FAST-SpO₂ IntellVue MPM/MPST/MPSSC, M8105A/AT/AS

Sensors	Weight Range	Saturation Accuracy (70 - 100% SpO ₂)		Pulse Rate Accuracy (25 - 240 bpm)	
		No Motion	Motion	No Motion	Motion
LNCs Adtu/Adtu-3	> 30 kg	± 2%		± 3 bpm	
LNCs Pdtu/Pdtu-3	10 - 50 kg	± 2%		± 3 bpm	
LNCs Infnt-3	3 - 20 kg	± 2%		± 3 bpm	
LNCs Neo/Neo-3	< 3 kg	± 3%		± 3 bpm	
	> 40 kg	± 2%		± 3 bpm	
LNCs NeoP/NeoP-3	< 1 kg	± 3%		± 3 bpm	
LNCs DCI	> 30 kg	± 2%		± 3 bpm	
LNCs DCP	10 - 50 kg	± 2%		± 3 bpm	
LNCs TC-i	> 30 kg	± 3.5%		± 3 bpm	
LNCs TF-i	> 30 kg	± 2%		± 3 bpm	
	1 - 3 kg	± 3%		± 3 bpm	
LNCs YI	> 3 kg	± 2%		± 3 bpm	

LNC MP Series Patient Cables and the above Masimo Sensors have been validated with the Masimo SET and Philips FAST-SpO₂ devices listed above. The saturation accuracy of the neonatal sensors were validated on adult volunteers and 1%w added to account for the properties of fetal hemoglobin. The LNCs YI multiuse sensor was validated on adult adults.

WARNINGS

- Ensure the cable is physically intact, with no broken or frayed wires or damaged parts. Visually inspect the cable and discard if cracks or discoloration are found.
- Carefully route cable and patient cable to reduce the possibility of patient entanglement or strangulation.
- Do not attempt to reprocess, recondition or recycle Masimo patient cables as these processes may damage the electrical components, potentially leading to patient harm.
- Failure to properly connect the sensor or the extender module to the cable will result in intermittent readings, inaccurate results, or no reading.
- To avoid damage to the cable, always hold it by the connector rather than the cable when connecting or disconnecting either end.
- Always refer to the extender module operator's manual for complete instructions or additional instructions.
- Refer to the specific sensor's *Directions for Use* for detailed sensor instructions and warnings.
- Probes and cables are designed for use with specific monitors. The user and/or operator needs to verify the compatibility of the monitor, probe, and cable before use, otherwise patient injury can result.

INSTRUCTIONS

A. Attaching LNC MP Patient Cables to the Instrument

Refer to Figures 1a and 1b. Orient the connector of the cable to match with the patient cable connector on the SpO₂ Module (Fig. 1a) or the MMS (Fig. 1b).

CAUTION: ENSURE YOU ARE CONNECTING A LNC MP PATIENT CABLE TO EITHER A MASIMO SET OR PHILIPS FAST SpO₂ SOCKET. THERE ARE A NUMBER OF SIMILAR CONNECTORS WITH DIFFERENT COLORS AND DIFFERENT MECHANICAL KEYING. NEVER FORCE A PATIENT CABLE CONNECTOR INTO THE MODULE OR MMS. FAILURE TO USE A LNC MP PATIENT CABLE MAY RESULT IN DAMAGE TO THE MODULE OR MMS, INACCURATE READINGS, OR NO READINGS.

Note: There is a difference in color shade between the LNC MP Patient Cable and the Philips MMS/ SpO₂ Module receptacle. However, this is an acceptable configuration. Masimo SET performance is not available when connected to an instrument with Philips FAST SpO₂ Technology.

B. Attaching the LNC MP Patient Cable to a LNCs Sensor

- Refer to Figure 2. Insert the Sensor cable connector completely into the LNC MP Patient Cable.
- Refer to Figure 3. Completely close the protective cover.

Disconnecting the LNC MP Patient Cable from a LNCs Sensor

- Refer to Figure 4. Lift the protective cover to gain access to the sensor connector.
- Refer to Figure 5. Pull firmly on the sensor connector to remove from the patient cable.

CLEANING

Clean the LNC MP Patient Cable by wiping it with a 70% isopropyl alcohol pad and allow it to dry.

CAUTION

Do not soak or immerse the cable in any liquid solution. Do not attempt to sterilize.

ENVIRONMENTAL

Operating Temperature	41° F to 104° F (5° C to 40° C)
Storage Temperature	-40° F to 158° F (-40° C to 70° C)
Relative Humidity	5% to 95% noncondensing

WARRANTY

Masimo warrants to the initial buyer only that these products, when used in accordance with the directions provided with the Products by Masimo, will be free of defects in materials and workmanship for a period of six (6) months.

THE FOREGOING IS THE SOLE AND EXCLUSIVE WARRANTY APPLICABLE TO THE PRODUCTS SOLD BY MASIMO TO BUYER. MASIMO EXPRESSLY DISCLAIMS ALL OTHER ORAL, EXPRESS OR IMPLIED WARRANTIES, INCLUDING WITHOUT LIMITATION ANY WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. MASIMO'S SOLE OBLIGATION AND BUYER'S EXCLUSIVE REMEDY FOR BREACH OF ANY WARRANTY SHALL BE, AT MASIMO'S OPTION, TO REPAIR OR REPLACE THE PRODUCT.

WARRANTY EXCLUSIONS

This warranty does not extend to any product that has been used in violation of the operating instructions supplied with the product, or has been subject to misuse, neglect, accident or externally created damage. This warranty does not extend to any product that has been connected to any unintended instrument or system, has been modified, or has been disassembled or reassembled. This warranty does not extend to sensors or patient cables that are not reproduced, reconditioned or recycled.

IN NO EVENT SHALL MASIMO BE LIABLE TO BUYER OR ANY OTHER PERSON FOR ANY INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES INCLUDING WITHOUT LIMITATION LOST PROFITS, EVEN IF ADVISED OF THE POSSIBILITY THEREOF. IN NO EVENT SHALL MASIMO'S LIABILITY ARISING FROM ANY PRODUCTS SOLD TO BUYER (UNDER A CONTRACT, WARRANTY, TORT OR OTHER CLAIM EXCEED THE AMOUNT PAID BY BUYER FOR THE LOT OF PRODUCTS) INVOLVED IN SUCH CLAIM. IN NO EVENT SHALL MASIMO BE LIABLE FOR ANY DAMAGES ASSOCIATED WITH A PRODUCT THAT HAS BEEN REPRODUCED, RECONDITIONED OR RECYCLED. THE LIMITATIONS IN THIS SECTION SHALL NOT BE DEEMED TO PRECLUDE ANY LIABILITY THAT, UNDER APPLICABLE PRODUCTS LIABILITY LAW, CANNOT LEGALLY BE PRECLUDED BY CONTRACT.

NO IMPLIED LICENSE

Purchase or possession of this patient cable does not carry any express or implied license to use this cable, and no use of this cable is not an authorized device or separately authorized to use LNC MP Patient Cables.

CAUTION: FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN. For professional use. See instructions for full prescribing information, including indications, contraindications, warnings, precautions and adverse events.

The following symbols may appear on the product or product labeling:

SYMBOLS	DEFINITION
	Caution
	Follow Instructions for Use
	Manufacturer
	Date of Manufacture
	Non-sterile
	Not made with natural rubber latex
Rx ONLY	Federal law (USA) restricts this device to sale by or on the order of a physician
CE	Mark of Conformity to European Medical Device Directive 93/42/EEC
REP	Authorized representative in the European community
LOT	Lot code
REF	Catalogue number (model number)
>	Greater than
<	Less than
	Do Not Discard

Patents: http://www.masimo.com/patents.htm

Masimo, ® SET, TF-i and LNCs are federally registered trademarks of Masimo Corporation.

LNC is a trademark of Masimo Corporation.

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PHILIPS is a trademark of Koninklijke Philips Electronics N.V.