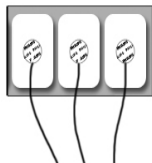
# Radiolucent Leadwires and Attached Electrodes

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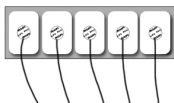
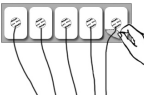
## **EN – Instructions for Use**

### **Intended Use/Indications for Use**

ECG electrodes are designed for use in general electrocardiographic procedures in healthcare environments where ECG monitoring is deemed necessary and is ordered by a physician. Such procedures include in particular patient ECG surveillance and ECG diagnosis recording. ECG electrodes are single-use, non-sterile and disposable and are to be used on intact (uninjured) skin.

#### **Intended Users**

Intended for use by licensed healthcare professionals.

#### **Contraindications**

There are no known contraindications.

#### **Patient Population**

All electrodes listed in this Instructions for Use are for adult patient use.

### **WARNINGS**

- Do not allow monitoring electrodes or lead wires to touch defibrillation pads. Such contact can cause electrical arcing, patient burns during defibrillation and may divert therapy current away from the heart.
- Do not reuse due to risk of cross infection, degradation of adhesive or electrical performance.
- Do not use on patients with a history of skin sensitization or irritation in direct electrode contact. Report any incompatibility reactions observed to Philips.
- Employ recommended procedures from electrosurgical or monitoring equipment manufacturers to minimize risk of burns at electrode site during electrosurgical procedures.
- Do not apply over open wounds, lesions, infected, irritated or inflamed areas to prevent further damage of skin.
- Do not use solvents to clean skin prior to ECG. Solvents trapped under electrodes may lead to abnormal skin reactions.
- Use caution when removing electrodes to prevent skin damage.
- Dispose of electrodes properly.
- These radiolucent leadwire electrodes may be used in conjunction with X-ray equipment but are not suitable for use in the MRI environment as there is the potential for patient burns.
- To avoid patient injury, ensure that the leadwires are carefully positioned to avoid entanglement, choking, strangulation or inhibit circulation in patient extremities.
- Do not use leadwire electrodes for more than 72 hours.
- Ensure that the leadwire connectors never come into contact with other conductive parts or with earth due to risk of electrical shock or burn.
- Ensure that the leadwire separator is removed from connectors and properly disposed.
- These leads are unshielded against electromagnetic interference - not to be used on defibrillators for monitoring for transcutaneous pacing or cardioversion.
- Properly dispose electrode backing plastic liner to prevent possible patient aspiration.

### **Cautions**

- Federal (US) law restricts this device to sale by or on the order of a medical practitioner.
- Do not use if gel is dry.
- Replace electrodes if they no longer firmly stick to patient skin.
- Use electrodes within seven days of opening pouch.

- Keep unused electrodes in original pouch which is closed by folding over.
- Do not use electrodes beyond the expiration date marked on the package.
- Ensure that application sites are properly prepared and are dry, clean and free of excessive hair. Inaccurate readings could result.
- Use care when applying electrode to avoid gel displacement. Loss of adhesion could occur.
- These Radiolucent leadwire electrodes are translucent in the X-ray environment but are not designed for use in the MR environment.
- Ensure that the leadwire connector is properly inserted into the trunk cable.
- These leadwire electrodes are not designed for telemetry use.

## Visual Inspection Prior to Use

Before applying any electrode to a patient, visually inspect the electrode. Check for any damage that could compromise accurate readings or cause patient/user injury. If visual inspection reveals any damage, follow appropriate disposal procedures. See “Disposal” on page 2.

## Compatibility

The operator is responsible for checking compatibility of their Philips monitoring equipment and these electrodes. Incompatible components may result in degraded performance.

## Patient Application

To apply the Radiolucent disposable ECG electrodes to a patient (see illustrations on page i):

1. Prepare the electrode site according to your healthcare facility’s protocol.
2. Gently peel the pad off the sheet.
3. Apply to prepared site on patient.
4. Remove Separator and discard.
5. Connect leadwire to trunk cable.

## Reordering Information

| Part <span style="border: 1px solid black; padding: 0 2px;">REF</span> | Description                            |
|------------------------------------------------------------------------|----------------------------------------|
| 989803156221                                                           | Radiolucent 3 Lead Electrode Set, AAMI |
| 989803156231                                                           | Radiolucent 3 Lead Electrode Set, IEC  |
| 989803156261                                                           | Radiolucent 5 Lead Electrode Set, AAMI |
| 989803156271                                                           | Radiolucent 5 Lead Electrode Set, IEC  |

## Disposal

Follow approved medical waste disposal methods as specified by your patient care facility or local regulations.

## Incident Reporting

Any serious incident that has occurred in relation to this device should be reported to Philips and the Competent Authority of the European Economic Area (EEA) countries, including Switzerland and Turkey, in which the user and/or patient is established.

## Operating, Storage, Transport Specifications

| Temperature                | Atmospheric Pressure | Relative Humidity (non-condensing) |
|----------------------------|----------------------|------------------------------------|
| 5°C to 30°C (41°F to 86°F) | 570hPa to 1000hPa    | 15% RH to 90% RH                   |

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Copyright © 2020
All rights reserved
Nederland B.V.