Microstream™ Advance

Adult Oral-Nasal CO₂ Filter Line

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EN Instructions for Use

Intended Use

Microstream™ Advance Adult and Pediatric Oral-Nasal and Nasal CO₂ Filter Line are intended to conduct a sample of the subject's breathing to a gas measurement device (capnograph) while simultaneously administering supplemental oxygen projected near the nose and mouth for inhalation. The device is to be used with monitors using Microstream™ technology. The product is intended for single patient use only.

Intended Users

Microstream™ Advance Adult and Pediatric Oral-Nasal and Nasal CO₂ Filter Line are intended to be used by licensed healthcare providers only.

Indications for Use

Microstream™ Advance Adult and Pediatric Oral-Nasal and Nasal CO₂ Filter Line are indicated for measurement of exhaled carbon dioxide (CO₂) for monitoring purposes. These devices are limited by the indications for use of the connected Philips monitoring equipment in healthcare facilities. These devices are intended to interact with patient intact skin and nasal mucosa.

Contraindications

There are no known contraindications.

WARNINGS

- Loose or damaged connections may compromise ventilation or cause an inaccurate measurement of respiratory gases. Securely connect all components, turning the sampling line connector clockwise into the monitor CO₂ port until it can no longer be turned. Check connections for leaks according to standard clinical procedures, and ensure CO₂ values appear.
- Do not attempt to clean, disinfect, sterilize or flush any part of the sampling line. Re-use of single-use accessories could pose a cross-contamination risk to the patient or damage the functioning of the monitor.
- Inaccurate CO₂ readings may occur if the patient has clogged nares.
- Sampling lines are recommended for use with oxygen provided at up to 5l/min. At higher levels of oxygen provision, dilution of CO₂ readings may occur, leading to lower CO₂ values.
- Ensure no kinks are present in CO₂ and O₂ tubing, as kinked tubing may cause inaccurate CO₂ sampling or affect O₂ delivery.
- For 989803204501, 989803204541, 989803204351, 989803204401, 989803204411, 989803204601, 989803204561, and 989803204681 only: This product can expose you to chemicals, including DINP, which are known to the State of California to cause cancer. For more information, go to www.P65Warnings.ca.gov.
- Carefully route patient cabling (CO₂ sampling line) to reduce the possibility of patient entanglement or strangulation.
- Use of tools that can generate sparks in highly oxygenated environment can lead to patient injury.

Caution

Federal (US) law restricts this device to sale by or on the order of a medical practitioner.

Required Compatible Components

These devices are to be used with Philips monitors using Microstream™ technology.

Applying the Oral-Nasal Filter Line

When performing the following procedure, reference the step-by-step patient application graphics on Pages i and ii.

Patient Setup

- ① CO₂ oral-nasal sampling with O₂ for adult patients.
- ② CO₂ oral-nasal sampling with O₂ for pediatric patients.
- ③ CO₂ nasal sampling with O₂.

Connecting the Oral-Nasal CO₂/Nasal CO₂ Filter Line to the Patient

- ④ Fit tubing to patient.
- ⑤ O₂ provision.
- ⑥ Attach O₂ tubing to O₂ source
- ⑦ Attach the CO₂ connector to monitor.

Operational Notes

- The response time of the 2m sampling line is ≤ 3.5 seconds, 4m sampling lines ≤ 5.9 seconds.
- Sampling lines with Extended Duration in their names include a moisture reduction component (Nafion®* or its equivalent) for use in higher humidity environments where long duration use of CO₂ sampling is required. During use, do not cover the moisture reduction component on these lines (situated at the patient's chest area) as covering this component may lead to faster occlusion of the sampling line.
- Ensure CO₂ values appear on the screen following connection of the sampling line.

TM* Trademark of its respective owner.

Reordering Information

Part Number	Description	Quantity
989803204381	Adult Oral-Nasal CO2 Filter Line with O2 Tubing, Extended Duration Note: Length: 2m	25
989803204551	Adult Oral-Nasal CO2 Filter Line with O2 Tubing, Short-term use: Procedural/Emergency Note: Length: 2m	25
989803204361	Adult Oral-Nasal CO2 Filter Line with O2 Tubing, Short-term use: Procedural/Emergency Note: Length: 4m	25
989803204571	Adult Oral-Nasal CO2 Filter Line with O2 Connector, Short-term use: Procedural/Emergency Note: Length: 2m	25
989803204371	Adult Oral-Nasal CO2 Filter Line with O2 Connector, Short-term use: Procedural/Emergency Note: Length: 4m	25
989803204391	Adult Oral-Nasal CO2 Filter Line with O2 Tubing, Extended Duration Note: Length: 4m	25
989803204481	Adult Nasal CO2 Filter Line with O2 Tubing, Short-term use: Procedural/Emergency Note: Length: 2m	25
989803204491	Adult Nasal CO2 Filter Line with O2 tubing, Short-term use: Procedural/Emergency Note: Length: 4m	25
989803204581	Adult Nasal CO2 Filter Line with O2 tubing, Extended Duration Note: Length: 2m	25
989803204611	Adult Nasal CO2 Filter Line, Short-term use: Procedural/Emergency Note: Length: 2m	25
989803204621	Adult Nasal CO2 Filter Line, Extended Duration Note: Length: 2m	25
989803204561	Pediatric Oral-Nasal CO2 Filter Line, Short-term use: Procedural/Emergency Note: Length: 2m	25
989803204601	Pediatric Nasal CO2 Filter Line, Short-term use: Procedural/Emergency Note: Length: 2m	25
989803204541	Pediatric Oral-Nasal CO2 Filter Line with O2 Tubing, Short-term use: Procedural/Emergency Note: Length: 2m	25
989803204351	Pediatric Oral-Nasal CO2 Filter Line with O2 tubing, Short-term use: Procedural/Emergency Note: Length: 4m	25
989803204401	Pediatric Oral-Nasal CO2 Filter Line with O2 Tubing, Extended Duration Note: Length: 2m	25
989803204411	Pediatric Oral-Nasal CO2 Filter Line with O2 Tubing, Extended Duration Note: Length: 4m	25
989803204591	Pediatric Nasal CO2 Filter Line with O2 tubing, Extended Duration Note: Length: 2m	25
989803204501	Pediatric Nasal CO2 Filter Line with O2 tubing, Short-term use: Procedural/Emergency Note: Length: 2m	25
989803204691	Adult Oral-Nasal CO2 Filter Line with O2 Tubing, Short-term use: Procedural/Emergency Note: Length: 2m	25

989803204711	Adult Oral-Nasal CO2 Filter Line with O2 Tubing, Short-term use: Procedural/Emergency Note: Length: 4m	25
989803204701	Adult Oral-Nasal CO2 Filter Line with O2 Tubing, Extended Duration Note: Length: 2m	25
989803204741	Adult Nasal CO2 Filter Line with O2 tubing, Extended Duration Note: Length: 2m	25
989803204681	Pediatric Oral-Nasal CO2 Filter Line with O2 Tubing, Short-term use: Procedural/Emergency Note: Length: 2m	25

Disposal

Discard any product that becomes damaged or deteriorated. Follow approved medical waste disposal methods specified by your facility or local regulations.

Incident Reporting

Any serious incident that has occurred in relation to this device should be reported to Philips and the competent authority of the European Economic Area (EEA) countries, including Switzerland and Turkey, in which the user and/or patient is established.

Environmental Specifications

The following table provides the temperature and humidity ranges that must be maintained for these products during storage and transport:

Specification	Storage	Transport
Temperature	-20°C to 55°C (-4°F to 131°F)	-20°C to 55°C (-4°F to 131°F)
Humidity	10% RH to 90% RH	10% RH to 90% RH

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Copyright © 2020
Microstream™ is a trademark of Oridion
Böblingen GmbH
Medical 1987 Ltd. Oridion Medical 1987
Ltd. is a Medtronic company.