
Nellcor™ Sensor Extension Cable

REF DEC4

REF DEC8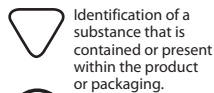

IP22

Protected against access to hazardous parts with a finger and protected against vertically falling water drops when sensor tilted up to 15°.

This device is not made with natural rubber latex or DEHP.

Description/Intended Use


The Nellcor™ DEC-4 and DEC-8 are reusable sensor extension cables of 4-foot (1.22m) and 8-foot (2.44m) lengths, respectively. The extension cable is designed for use to increase the cable length between a Nellcor sensor and a Nellcor oximeter, an instrument containing Nellcor oximetry, or a Nellcor-compatible instrument licensed to use Nellcor sensors.

Consult the instrument operator's manual or contact the individual manufacturers for the compatibility of particular instruments with a DEC-4/8 cable.

Carefully read the **Warning** and **Cautions** below before using a DEC-4 or DEC-8 cable.

Instructions for Use

When connecting a Nellcor sensor to the DEC-4/8 sensor extension cable, always orient the sensor plug so that NELLCOR is visible, as illustrated below.

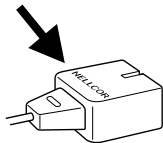- 1) Plug the Nellcor sensor into the connector of the DEC-4/8 cable that has the clear plastic sensor latch. Ensure that NELLCOR is visible on the sensor plug when connecting it to the DEC-4/8 cable.
A Plug the sensor straight into the connector; do not twist.

en	Instructions For Use
fr	Mode d'emploi
de	Gebrauchsanweisung
nl	Gebruiksaanwijzing
es	Instrucciones de uso
it	Istruzioni per l'uso
sv	Bruksanvisning
da	Brugsanvisning
pt	Instruções de uso
ru	Инструкция по применению
zh	使用说明

- 2) Snap the clear plastic sensor latch of the DEC-4/8 cable in place, over the sensor plug. **B**
- 3) Plug the connector at the other end of the DEC-4/8 cable into the oximeter, oximeter module, or patient module **C**, according to the instrument's operator manual.

To disconnect the sensor from the DEC-4/8 cable, open the clear latch. Then, while grasping the sensor plug, pull the plug straight out from the connector. Do not twist. Do not disconnect by pulling on the sensor cable or the DEC-4/8 cable.

Cleaning

Clean the extension cable by wiping it with a cleaning agent such as isopropyl alcohol.

WARNING: This cable is **not** designed for sterilization, including exposure to irradiation, steam, and ethylene oxide. Do **not** immerse the cable in cleaning or disinfecting solutions, or water. Damage to the cable can result.

Warnings

- 1) Do not use a damaged extension cable or one with exposed electrical contacts.
- 2) Do **not** connect two or more extension cables together. Inaccurate readings may result.
- 3) Failure to properly connect the sensor plug and the cable connector will result in intermittent or no readings.

Cautions

- 1) Electrosurgical interference may compromise performance and use of the extension cable may increase the potential for such interference.
- 2) To avoid damage to the extension cable or the sensor, always disconnect the sensor from the cable by grasping the connectors, not the cables.
- 3) Do **not** use with Nellcor N-30 monitors. Inaccurate readings may result.
- 4) Do **not** use with four-button Nellcor N-100 monitors. Inaccurate readings may result.

How Supplied

One Nellcor DEC-4 or DEC-8 sensor extension cable is supplied in a nonsterile package.

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© 2011 Covidien.
Covidien Inc, 15 Hampshire Street, Mansfield, MA 02048 USA.
EC REP Covidien Ireland Limited, IDA Business & Technology Park, Tullamore.