
Nellcor™

Pulse Oximetry Interface Cable


REF DOC4

REF DOC10


en	Instructions For Use
fr	Mode d'emploi
de	Gebrauchsanweisung
nl	Gebruiksaanwijzing
it	Istruzioni per l'uso
es	Instrucciones de uso
sv	Bruksanvisning
da	Brugsanvisning
no	Bruksanvisning
fi	Käyttöohjeet
pt	Instruções de uso
ru	Инструкция по применению
zh	使用说明
pl	Instrukcja użytkowania
cs	Návod k použití
sl	Navodila za uporabo
hu	Használati útmutató
el	Οδηγίες χρήσης
tr	Kullanma Talimatı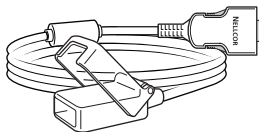


1

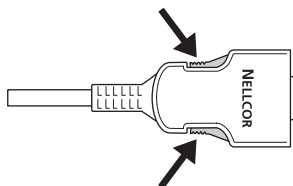2

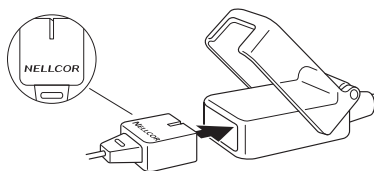3

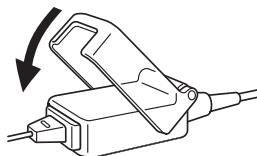DOC4, DOC10

Pulse Oximetry Interface Cable

This device is not made with natural rubber latex or DEHP.

Identification of a substance that is contained or present within the product or packaging.

Identification of a substance that is not contained or present within the product or packaging.

IP22

Protected against access to hazardous parts with a finger and protected against vertically falling water drops when sensor tilted up to 15°.

DOC CABLE	CABLE LENGTH
DOC4	4 FOOT (1.2 METERS)
DOC10	10 FOOT (3 METERS)

Directions for Use

Description/Intended Use

The Nellcor™ Pulse Oximetry Interface Cable links a Nellcor oxygen sensor to a Nellcor or Nellcor-compatible monitor. The DOC cable is for use only with Nellcor oxygen sensors that have a slot on both sides of the plug.

Note: The DOC cable is **ONLY** compatible with pulse oximeters that contain OxiMax technology. Consult the information provided by the equipment manufacturer to determine DOC cable compatibility.

Instructions for Use (Illustrations on page 2)

Use **ONLY** Nellcor oxygen sensors that have both sides of the connecting plug slotted. When connecting the sensor to the cable, always orient the sensor plug so that NELLCOR is visible.

1. To connect the DOC cable to the monitor, depress the buttons on both sides of the cable connector as it is inserted into the monitor socket. Release the buttons after the cable connector is firmly seated in the socket. **(1)**
2. Plug the sensor into the other end of the DOC cable with the clear plastic sensor latch. Ensure that NELLCOR is visible on the sensor plug as it is connected to the cable. Plug the

sensor straight in; do not twist. **(2)**

3. Snap the clear plastic sensor latch in place over the sensor plug. **(3)**
4. To disconnect the sensor from the DOC cable, open the clear plastic sensor latch. Grasp the sensor plug, and pull it straight out from the cable connector; do not twist. Do not disconnect the sensor by pulling it out by the sensor cable.
5. To disconnect the DOC from the monitor, depress the buttons on both sides of the cable connector. Pull the connector straight out from the monitor socket and release the buttons; do not twist. **(1)**

Cleaning

Clean the cable by wiping it with isopropyl alcohol.

WARNINGS

- **Examine cable before use. Do not use a damaged cable or one with exposed wires.**
- **Do not connect the DOC to any monitor for which the DOC is not intended for use, as this could result in electromagnetic interference or susceptibility to or from other devices within the patient environment.**
- **As with all medical equipment, carefully route cables to reduce the possibility of patient entanglement or strangulation.**

CAUTIONS

- Do not sterilize. This cable is not designed for sterilization, including irradiation, or exposure to steam or ethylene oxide. Do not immerse the cable in cleaning solutions, disinfecting solutions, or water. Damage to the cable can result.
- When disconnecting the cable from the monitor, always depress the buttons on both sides of the connector as it is removed from the monitor socket. Failure to do so may result in damage to the cable connector or to the monitor socket.
- To avoid damaging the DOC, grasp the connector rather than the cable when disconnecting it from the sensor or from the monitor.
- Check cable connections. Failure to properly connect the plugs will result in intermittent readings or no readings.

Warranty

To obtain information about a warranty for this product, contact Covidien Technical Services or your local Covidien representative.

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© 2011 Covidien.
registered trademarks of Covidien AG.
Other brands are trademarks of a
Covidien Ireland Limited,