Nellcor™

Pediatric SpO<sub>2</sub> Sensor Clip

Reusable

|    |                          |
|----|--------------------------|
| en | Instructions For Use     |
| fr | Mode d'emploi            |
| de | Gebrauchsanleitung       |
| nl | Gebruiksaanwijzing       |
| es | Instrucciones de uso     |
| it | Istruzioni per l'uso     |
| sv | Bruksanvisning           |
| da | Brugervejledning         |
| pt | Instruções de uso        |
| ru | Инструкция по применению |
| zh | 使用说明                     |

Identification of a substance that is contained or present within the product or packaging.

Identification of a substance that is not contained or present within the product or packaging.

Do not use if package is opened or damaged

IP22

Protected against access to hazardous parts with a finger and protected against vertically falling water drops when sensor tilted up to 15°.

OxiMax™

Technology

This device is not made with natural rubber latex or DEHP.

Indications/Contraindications

The Nellcor™ Pediatric SpO<sub>2</sub> Sensor Clip Reusable, model D-YS PD, is intended for use only with the Nellcor oxygen sensor, model D-YS, when noninvasive, arterial oxygen saturation and pulse rate monitoring are required for patients weighing between 3 kg and 40 kg.

**The Pediatric SpO<sub>2</sub> Sensor Clip is intended for spot-check, attended monitoring for a period of 20 minutes or less.** It may be used in any environment, including hospital, mobile, and home use, where spot-check monitoring is desired and compatible pulse oximetry monitors are indicated for use. (See the D-YS model sensor directions for use for compatibility information.)

Note: The Pediatric SpO<sub>2</sub> Sensor Clip is free from naturally-occurring latex.

Instructions for Use

The Pediatric SpO<sub>2</sub> Sensor Clip consists of a plastic clip with two extended metal loops for holding the pads of the D-YS model sensor, and two removable elastomeric sleeves, available separately as model PDSLV, that cover the metal loops and plastic clip. Figure **A** shows (a) the metal loops and (b) the plastic clip. Figure **B** shows the sleeve with (a) placement target (soccer ball symbol), (b) placement indicator (straight line), and (c) the dimpled finger grip. The sleeves must be removed from the plastic clip (as described in Step 1) to attach the D-YS model sensor.

To assemble the Pediatric SpO<sub>2</sub> Sensor Clip with the D-YS model sensor:

- Remove the elastic sleeves (one at a time) from the plastic clip by gently lifting the wide dimpled end of each sleeve off the wide end of the clip **C**. Then grasp the sides of the narrow end of the sleeve and gently slide the sleeve off the clip.
- With the optical surfaces of the D-YS model sensor touching each other, insert the pads through the center opening of the wide end of the plastic clip **D**. Note the top and bottom orientation of the white and black sensor pads relative to the Nellcor logo on the side of the plastic clip.
- Engage the “button” of the sensor pad with the extended metal loop by grasping the pad and gently pushing it into the loop. Repeat for the other sensor pad. When both sensor pads are properly secured in the metal loops, the optical surfaces of the emitter and detector will face each other **E**.
- Replace each elastic sleeve by gently sliding the narrow end of the sleeve over the metal loop and sensor pad **F**. Then gently stretch the pocket of the wide end of the sleeve over the wide end of the plastic clip. When the clip is properly assembled, the emitter and detector will be visible through the clear window inside each sleeve. The soccer ball on the sleeve is a visual indicator of the emitter or detector underneath.
- The Pediatric SpO<sub>2</sub> Sensor Clip is now ready to be applied to the patient. The preferred sensor site is the index finger. A smaller finger may be used as an alternate site if it offers a better fit with the shape of the Pediatric SpO<sub>2</sub> Sensor Clip

Figure **G** shows an end view of the Pediatric SpO<sub>2</sub> Sensor Clip on a finger that (a) is too small, leaving gaps on either side of the finger and sleeves; (b) is too large, leaving gaps between the top and bottom of the finger and the sleeves such that the emitter and detector are not in direct contact with the tissue; (c) fits correctly; and (d) also fits correctly. Choose a site for the Pediatric SpO<sub>2</sub> Sensor Clip that fits like that shown in (c) or (d).

Note: When placing the Pediatric SpO<sub>2</sub> Sensor Clip on a finger, the white sensor pad (containing the emitter) must be placed on the nail side.

Note: When selecting a sensor site, priority should be given to an extremity free of an arterial catheter, blood pressure cuff, or intravascular infusion line.

- Position the clip on the finger such that (a) the straight line on the sleeve (the placement indicator) is at the tip of the finger **H**. **Do not cover the entire finger with the clip. Improper placement may lead to inaccurate measurements. The Pediatric SpO<sub>2</sub> Sensor Clip is intended for spot-check, attended monitoring for a period of 20 minutes or less. Clip pressure could compromise skin integrity and/or possibly cause neural damage.**

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© 2012 Covidien.
COVIDIEN, COVIDIEN with logo and Covidien logo are U.S. and internationally registered trademarks of Covidien AG. Other brands are trademarks of a Covidien company.
Covidien LLC, 15 Hampshire Street, Mansfield, MA 02048 USA.
Covidien Ireland Limited, IDA Business & Technology Park, Tullamore.