
Air Hoses for Noninvasive Blood Pressure Cuffs

Instructions for Use


Neonatal NBP Air Hose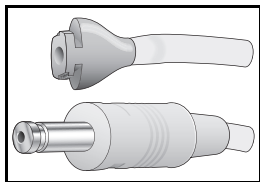

REF M1596C 1.5m Hose

REF M1597C 3.0m Hose

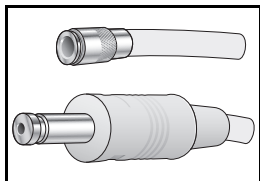
Adult NBP Air Hose

REF M1598B 1.5m Hose

REF M1599B 3.0m Hose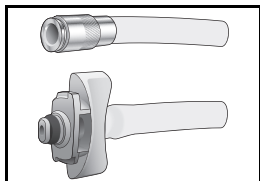

Mobile CL Adult Bayonet Air Hose

REF 989803187431 1.0m Hose

PHILIPS


(EN) Instructions for Use	1
(BG) Указания за употреба.....	1
(CS) Návod k obsluze.....	2
(DA) Brugervejledning	2
(DE) Gebrauchsanweisung.....	3
(EL) Οδηγίες χρήσης.....	3
(ES) Instrucciones de uso.....	4
(ET) Kasutusjuhend.....	4
(FI) Käyttöohjeet	5
(FR) Manuel d'utilisation	5
(HR) Upute za uporabu.....	6
(HU) Használati útmutató	6
(ID) Petunjuk Penggunaan	7
(IT) Istruzioni d'uso	7
(JA) ユーザーズガイド	8
(KK) Пайдалану нұсқаулығы	8
(KO) 사용 설명서	9
(LT) Naudojimo nurodymai	9

(LV) Lietošanas instrukcija.....	10
(MK) Упатства за користење	10
(NL) Gebruiksaanwijzing.....	11
(NO) Brukerhåndbok.....	11
(PL) Instrukcja obsługi	12
(PT-BR) Instruções de uso	12
(PT-PT) Instruções de utilização	13
(RO) Instrucţiuni de utilizare	13
(RU) Инструкция по эксплуатации	14
(SK) Návod na použitie	14
(SL) Navodila za uporabo	15
(SR) Uputstvo za upotrebu	15
(SV) Bruksanvisning	16
(TR) Kullanım Talimatları	16
(UK) Інструкції з використання	17
(VI) Hướng Dẫn Sử Dụng	17
(ZH-CN) 使用说明.....	18
(ZH-TW) 使用說明.....	18


	EN: Manufacturer; BG: Производител; CS: Výrobce; DA: Producent; DE: Hersteller; EL: Κατασκευαστής; ES: Fabricante; ET: Tootja; FI: Valmistaja; FR: Fabricant; FR: Produced; HU: Gyártó; ID: Produsen; IT: Fabriccane; JA: 製造業者; KK: Өндүрүш; KO: 제조자; LT: gamintojas; LV: Ražotājs; MK: Производител; NL: Fabrikant; NO: Produsent; PL: Producent; PT: Fabricante; PT-BR: Fabricante; RO: Producător; RU: Производитель; SK: Výrobca; SL: Proizvajalec; SR: Proizvođač; SV: Tillverkare; TR: Üretici; UK: Виробник; VI: Nhà sản xuất; ZH-CN: 制造商; ZH-TW: 製造商
EC REP	EN: Authorized Representative in the European Community; BG: Упълномощен представител в Европейската общност; CS: Autorizovaný zástupce v Evropském společenství; DA: Autoriseret repræsentant i EU; DE: Autorisierte EU-Vertretung; EL: Εξουσιοδοτημένος αντιπρόσωπος για την Ευρωπαϊκή Ένωση; ES: Representante autorizado en la Unión Europea; ET: Volitatud esindaja Euroopa Ühenduses; FI: Valtuutettu edustaja Euroopan yhteisössä; FR: Représentant autorisé pour la Communauté européenne; HR: Ovlašteni zastupnik u Europskoj zajednici; HU: Hivatalos képviselő az Európai Közösségekben; ID: Perwakilan UE Resmi; IT: Rappresentante autorizzato nella Comunità europea; JA: Authorized EU Representative; KK: Еуропалык қауымдастықтың уәкілетті өкілі; KO: 유럽 공인 대리점; LT: įgaliojatis atstovas Europos Bendrijoje; LV: Pilnvarotais pārstāvis Eiropas Kopienā; MK: Овластен претставник во Европската заедница; NL: Officiële vertegenwoordiger in de Europese Unie; NO: Autorisert EU-representant; PL: Autoryzowany przedstawiciel na terenie Wspólnoty Europejskiej; PT: Representante autorizado na Comunidade Europeia; PT-BR: Representante autorizado na União Europeia; RO: Reprezentant autorizat în Comunitatea Europeană; RU: Уполномоченный представитель в странах Европейского сообщества; SK: Autorizovaný zástupca pre Európske spoločenstvo; SL: Pooblaščen zastopnik za Evropsko skupnost; SR: Ovlašćeni predstavnik Evropske zajednice; SV: Auktoriserad representant i Europeiska unionen; TR: Yetkilil Avrupa Topluluğu Temsilcisi; UK: Уповноважений представник на території ЄС; VI: Đại diện được ủy quyền ở Cộng đồng châu Âu; ZH-CN: 欧盟授权代表; ZH-TW: 授權的歐洲共同體代表
REF	EN: Catalogue Number; BG: Каталоген номер; CS: Katalogové číslo; DA: Katalognummer; DE: Katalognummer; EL: Αριθμός καταλόγου; ES: Número de catálogo; ET: Kataloginumber; FI: Luettelonumero; FR: Numéro de référence; HR: Kataloški broj; HU: Katalógusszám; ID: Nomor Katalog; IT: Numero di catalogo; JA: カタログ番号; KK: Каталог бойынша нөмөрі; KO: 카탈로그 번호; LT: katalogo numeris; LV: Kataloga numurs; MK: Каталогски број; NL: Catalogusnummer; NO: Katalognummer; PL: Numer katalogowy; PT: Número de catálogo; PT-BR: Número de catálogo; RO: Număr de catalog; RU: Номер по каталогу; SK: Kataloží číslo; SL: Kataloška številka; SR: Kataloški broj; SV: Katalognummer; TR: Katalog Numarası; UK: Номер за каталогом; VI: Số catalog; ZH-CN: 目录号; ZH-TW: 目錄編號
LOT	EN: Batch Code; BG: Партиден код; CS: Kód série; DA: Kode for parti; DE: Chargenbezeichnung; EL: Κωδικός παρτίδας; ES: Código de lote; ET: Partikood; FI: Eräkkoodi; FR: Code de lot; HR: Šifra serije; HU: Tételszám; ID: Kode Batch; IT: Codice lotto; JA: 製造番号又は製造記号; KK: Партиция коды; KO: 배치 코드; LT: partijos kodas; LV: Sērijas numurs; MK: Код на серија; NL: Batchcode; NO: Partikode; PL: Kod partii; PT: Código de lote; PT-BR: Código de lote; RO: Codul lotului; RU: Код партии; SK: Kód šarže; SL: Številka serije; SR: Broj serije; SV: Batchkod; TR: Parti Kodu; UK: Код партії; VI: Mã lô; ZH-CN: 批号; ZH-TW: 批次碼
	EN: Quantity per box; BG: Количество в кутия; CS: Množství v krabici; DA: Antal pr. æske; DE: Menge pro Karton; EL: Ποσότητα ανά συσκευασία; ES: Cantidad por caja; ET: Arv karbis; FI: Määrä laatikossa; FR: Quantité par boîte; HR: Količina po kutiji; HU: Mennyiség dobozoként; ID: Isi per kotak; IT: Quantità per scatola; JA: 1箱あたりの数量; KK: Бир чорпактың саны; KO: 상자당 수량; LT: kiekis dėžutėje; LV: Skaitis kastītē; MK: Количина по кутија; NL: Hoeveelheid per doos; NO: Antall per pakke; PL: Liczba sztuk w kartonie; PT: Quantidade por caixa; PT-BR: Quantidade por caixa; RO: Cantitate per cutie; RU: Количество в коробке; SK: Množstvo v škatuli; SL: Količina v škatli; SR: Količina u kutiji; SV: Antal per låda; TR: Kutu başına miktar; UK: Кількість в одній коробці; VI: Số lượng mỗi hộp; ZH-CN: 每盒数量; ZH-TW: 每盒數量
	EN: Storage temperature range; BG: Диапазон на температурата при съхранение; CS: Rozsah skladovací teploty; DA: Opbevaringstemperatur; DE: Temperaturbereich bei Lagerung; EL: Εύρος πριόν θερμοκρασίας φύλαξης; ES: Rango de temperatura de almacenamiento; ET: Hoiustamistemperatuur vahemik; FI: Säilytyslämpötila; FR: Limites de température de stockage; HR: Raspon temperature za pohranu; HU: Tárolási hőmérséklet-tartomány; ID: Rentang suhu penyimpanan; IT: Temperatura di conservazione a magazzino; JA: 保管温度範囲; KK: Сактау температурасынын аукумы; KO: 보관 온도 범위; LT: laikymo temperatūra; LV: Uzglabāšanas temperatūras diapazons; MK: Опер на температура за складирање; NL: Bereik opslagtemperatuur; NO: Temperaturområde for oppbevaring; PL: Zakres temperatury przechowywania; PT: Intervalo da temperatura de armazenamento; PT-BR: Intervalo de temperatura de armazenamento; RO: Interval de temperatură pentru depozitare; RU: Диапазон температуры при хранении; SK: Rozsah teplôt pri skladovaní; SL: Temperaturni razpon za shranjevanje; SR: Raspon temperature pri skladištenju; SV: Förvaringstemperaturområde; TR: Saklama sıcaklığı aralığı; UK: Обмеження температури; VI: Phạm vi nhiệt độ bảo quản; ZH-CN: 存储温度范围; ZH-TW: 儲存溫度範圍
	EN: Consult instructions for use; BG: Консултирайте се с указанията за употреба; CS: Prostudujte si návod k obsluze; DA: Se brugsanvisningen; DE: Gebrauchsanweisung beachten; EL: Συμβουλευτείτε τις οδηγίες χρήσης; ES: Consultar las instrucciones de uso; ET: Lugege kasutusjuhendit; FI: Lue käyttöohjeet; FR: Consulter le manuel d'utilisation; HR: Pogledajte upute za uporabu; HU: Lásd a használati útmutatót; ID: Baca petunjuk penggunaan; IT: Consultare le Istruzioni d'uso; JA: ユーザーズガイド参照; KK: Пайдалану нұсқаулығын қараңыз; KO: 사용 설명서 참조; LT: žr. naudojimo instrukcijas; LV: Skatiet lietošanas instrukcijas; MK: Помогнете во упатствата за користење; NL: Raadpleeg de gebruiksaanwijzing; NO: Se brukenhåndboken; PL: Zapoznaj się z instrukcją obsługi; PT: Consultar as instruções de utilização; PT-BR: Consulte as instruções de uso; RO: Consultati instrucțiunile de utilizare; RU: См. инструкции по эксплуатации; SK: Pozrite si návod na použitie; SL: Glejte navodila za uporabo; SR: Pogledajte uputstvo za upotrebu; SV: Läs bruksanvisningen; TR: Kullanım talimatlarını başvurun; UK: Читайте інструкції з використання; VI: Tham khảo hướng dẫn sử dụng; ZH-CN: 参见使用说明; ZH-TW: 請見使用說明

 YYY-MM-DD	EN: Date and country of manufacture; BG: Дата и държава на производство; CS: Datum a země výroby; DA: Fremstillingsdato og -land; DE: Herstellungsdatum und -land; EL: Ημερομηνία και χώρα κατασκευής; ES: Fecha y país de fabricación; ET: Tootmiskuu päev ja riik; FI: Valmistuspäivä-määrä ja -maa; FR: Date et pays de fabrication; HR: Datum i zemlja proizvodnje; HU: Gyártás helye és dátuma; ID: Tanggal dan negara produksi; IT: Data e paese di fabbricazione; JA: 製造年月日 / 製造国; KK: Өндүрілген күні және елі; KO: 제조일 및 제조국; LT: pagaminimo data ir šalis; LV: Ražošanas datums un valsts; MK: Датум и земја на производство; NL: Datum en land van productie; NO: Dato og produksjonsland; PL: Data i kraj produkcji; PT: Data e país de fabrico; PT-BR: Data e país do fabricante; RO: Data și țara de producție; RU: Дата и страна производства; SK: Dátum a krajina výroby; SL: Datum in država proizvodnje; SR: Datum i zemlja proizvodnje; SV: Tillverkningsdatum och -land; TR: Üretim tarihi ve ülkesi; UK: Дата та країна-виробник; VI: Ngày tháng và quốc gia của nhà sản xuất; ZH-CN: 生产日期和地区; ZH-TW: 製造日期和國家
	EN: Caution; BG: Вниманиe; CS: Varování; DA: Forsigtig; DE: Vorsicht; EL: Προσοχή; ES: Precaución; ET: Ettevaatust!; FI: Varoituu; FR: Mise en garde; HR: Oprez; HU: Figyelem; ID: Perhatian; IT: Attenzione; JA: 注意; KK: Ескертүү; KO: 주의; LT: perspėjimai; LV: Uzmanību; MK: Вниманиe; NL: Let op; NO: Obs!; PL: Przestroga; PT: Cuidado; PT-BR: Cuidado; RO: Atenție; RU: Вниманиe; SK: Varovanie; SL: Pozor; SR: Oprez; SV: Viktigt; TR: Dikkat; UK: УВАГА!; VI: Thận trọng; ZH-CN: 小心; ZH-TW: 小心事項
R_x only	EN: Prescription Only; BG: Само по предписание; CS: Pouze na lékařský předpis; DA: Kun på recept; DE: Anwendung nur auf ärztliche Verordnung; EL: Χρήση μόνο κατόπιν ιατρικής συνταγής; ES: Sujeto a prescripción médica; ET: Kasutamine ainult retsepti alusel; FI: Vain lääkärin määräyksestä; FR: Sur prescription uniquement; HR: Samo na recept; HU: Csak orvosi rendelvényre; ID: Hanya dengan Resep Dokter; IT: Solo su prescrizione medica; JA: 医家向け医療機器; KK: Тек дөрілгердің тағайындауымен қолданылады; KO: 처방 필요; LT: tik su receptu; LV: Nepieciešama recepte; MK: Само со рецепта; NL: Alleen op voorschrift; NO: Kun på resept; PL: Wyłącznie na zlecenie lekarza; PT: Apenas com prescrição médica; PT-BR: Somente sob prescrição; RO: Doar la recomandarea medicului; RU: Только по назначению врача; SK: Iba na predpis; SL: Izdaja se samo na recept; SR: Samo po nalogu lekara; SV: Får endast användas enligt läkares ordination; TR: Sadece Reçeteyele Kullanılır; UK: Використовувати лише за приписом лікаря; VI: Chỉ sử dụng theo chỉ định của bác sĩ; ZH-CN: 仅凭处方; ZH-TW: 僅限處方使用
 %RH	EN: Storage humidity range; BG: Диапазон на влажността при съхранение; CS: Rozsah skladovací vlhkosti; DA: Fugtighedsområde ved opbevaring; DE: Luftfeuchtigkeitsbereich bei Lagerung; EL: Εύρος τιμών υγρασίας φύλαξης; ES: Rango de humedad de almacenamiento; ET: Hoiustamise niiskussvahemik; FI: Säilytyskosteus; FR: Limites d'humidité de stockage; HR: Raspon vlažnosti za pohranu; HU: Tárolási páratartalom-tartomány; ID: Rentang kelembapan penyimpanan; IT: Umidità per la conservazione a magazzino; JA: 保管湿度範囲; KK: Сактау кезіндегі ылғалдылық ауқымы; KO: 보관 습도 범위; LT: laikymo drėgmė; LV: Uzglabāšanas mitruma diapazons; MK: Омер на влажност за складирање; NL: Bereik opslagvochtigheid; NO: Fuktighetsområde for oppbevaring; PL: Zakres wilgotności przechowywania; PT: Intervalo da humidade de armazenamento; PT-BR: Faixa de umidade de armazenamento; RO: Interval de umiditate pentru depozitare; RU: Диапазон влажности при хранении; SK: Rozsah vlhkosti pri skladovaní; SL: Razpon vlažnosti za shranjevanje; SR: Raspon vlažnosti pri skladištenju; SV: Luftfuktighetsområde vid förvaring; TR: Saklama nem aralığı; UK: Діапазон вологості для зберігання; VI: Phạm vi độ ẩm bảo quản; ZH-CN: 存储时湿度范围; ZH-TW: 儲存濕度範圍
MD	EN: Medical Device; BG: Медицинско изделие; CS: Zdravotnický přístroj; DA: Medicinsk udstyr; DE: Medizinprodukt; EL: Ιατροτεχνολογικό προϊόν; ES: Dispositivo médico; ET: Meditsiniseade; FI: Lääkinnällinen laite; FR: Dispositif médical; HR: Medicinski proizvod; HU: Orvostechnikai eszköz; ID: Perangkat Medis; IT: Dispositivo medico; JA: 医療機器; KK: Медициналык құрылғы; KO: 의료기기; LT: medicinos prietaisai; LV: Medicīnas ierīce; MK: Медицински уред; NL: Medisch apparaat; NO: Medisinsk utstyr; PL: Wyrób medyczny; PT: Dispositivo médico; PT-BR: Equipamento médico; RO: Dispozitiv medical; RU: Медицинское устройство; SK: Zdravotnícka pomôcka; SL: Medicinski pripomoček; SR: Medicinski uređaj; SV: Medicinteknisk produkt; TR: Tıbbi Cihaz; UK: Медичний пристрій; VI: Thiết bị y tế; ZH-CN: 医疗设备; ZH-TW: 醫療設備
UDI	EN: Unique Device Identifier; BG: Уникален идентификатор на устройството; CS: Jediněný indikátor zařízení; DA: Unik enhedsindikator; DE: Eindeutige Geräteerkennung; EL: Μονοδικό αναγνωριστικό συσκευής; ES: Identificación exclusiva del dispositivo; ET: Seadme kordumatu identifitseerimistunnus; FI: Laitteen yksilöivä tunnus; FR: Identifiant unique de l'appareil; HR: Jedinstveni identifikator proizvoda; HU: Egyedi eszközazonosító; ID: Indikator Perangkat Unik; IT: Identificatore univoco del dispositivo; JA: 機器固有識別子; KK: Құрылғының бірегей идентификаторы; KO: 의료기기 고유식별코드; LT: unikalus prietaiso identifikatorius; LV: ierīces unikālais indikators; MK: Уникатна идентификација на уредот; NL: Unieke apparaatindicator; NO: Unik enhetsindikator; PL: Unikalny identyfikator wyrobu; PT: Indicador único de dispositivo; PT-BR: Indicador único de dispositivo; RO: Identifier unic al dispozitivului; RU: Уникальный идентификатор устройства; SK: Jediný identifikačný zariadenia; SL: Edinstven identifikator pripomočka; SR: Jedinstvena identifikacija uređaja; SV: Unik produktindikator; TR: Benzersiz Cihaz Göstergesi; UK: Унікальний ідентифікатор пристрою; VI: Chỉ báo thiết bị duy nhất; ZH-CN: 唯一设备指示符; ZH-TW: 唯一裝置指示


M1596C, M1597C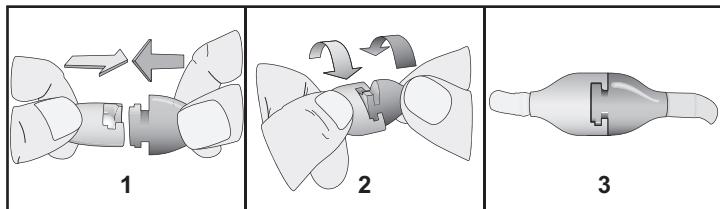

M1598B, M1599B, Mobile CL

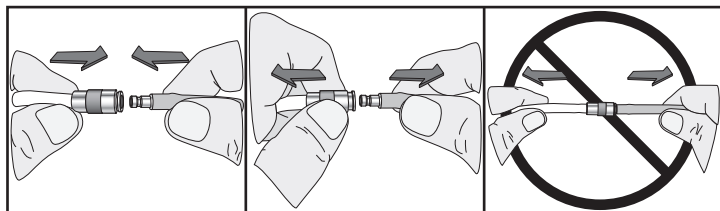Mobile CL

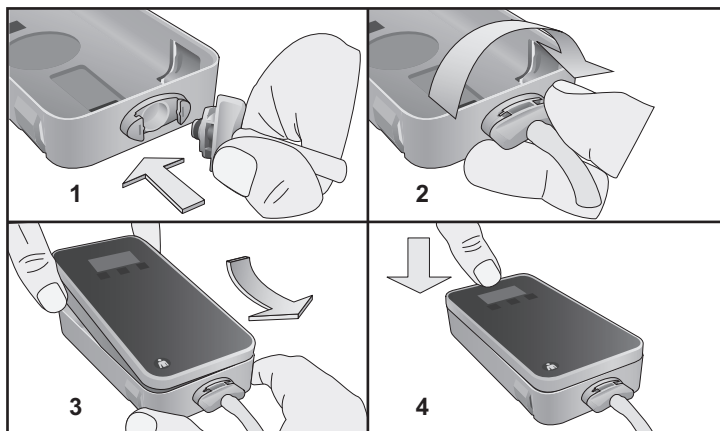EN - Instructions for Use

Associated Documents - *NBP Cuffs Instructions for Use; NBP Cuffs and Air Hoses - Care, Cleaning and Disinfection*. **Intended Use** - Philips air hoses are to be used with compatible devices intended for use by, or under the supervision of, a licensed physician or other healthcare provider for noninvasive measurement of neonatal, pediatric and adult human blood pressure. **Indications** - These devices are limited by the indications for use of the connected monitor in healthcare facilities. These devices are intended to interact with patient intact skin only. **Contraindications** - There are no known contraindications.

Warnings - Use neonatal and infant #5 cuffs only with hoses designated for Neonatal use (see cover and *Selecting the Appropriate Hose* below).

• Use pediatric and adult cuffs only with hoses designated for Adult use (see cover and *Selecting the Appropriate Hose* below.). • Hoses are not designed to be sterilized. Do not autoclave. • Lubricate the cuff-end connector of M1598B, M1599B, and Mobile CL hoses with a non-silicone spray release agent when it becomes difficult to connect to cuffs. Do not lubricate the instrument-end connectors. Do not lubricate the connectors for the M1596C and M1597C hoses. • Refer to the *NBP Cuffs and Air Hoses - Care, Cleaning and Disinfection* instructions for a list of validated cleaners and disinfectants and the appropriate procedures for prevention of cross-contamination between patient use. • Philips NBP air hoses use connectors which have the potential to mismatch with small-bore connectors used in non-NBP healthcare applications. When connecting Philips NBP air hose, ensure connectors are properly connected to Philips NBP devices only.

Caution - Federal (US) law restricts this device to sale by or on the order of a medical practitioner.

Visual Inspection Prior to Use - Before using the hose on a patient, perform a visual inspection to determine if the hose has reached its end of life. Visually inspect the hose for cracks, damaged connectors, or similar wear or damage. If damage or severe soiling occurs, see *Disposal* below.

Selecting the Appropriate Hose - Neonatal, Infant #5 use M1596C or M1597C; Infant, Pediatric or Small Adult, Adult, Large Adult, Thigh use M1598B, M1599B or Mobile CL. **Compatibility** - Only use hoses on systems that list them as accessories. **Reordering Information** - The following part numbers are CE marked and available in the European Union: M1596C — Neonatal NBP Air Hose 1.5m; M1597C — Neonatal NBP Air Hose 3.0m; M1598B — Adult NBP Air Hose 1.5m; M1599B — Adult NBP Air Hose 3.0m. **Cleaning and Disinfection** - Refer to the *NBP Cuffs and Air Hoses - Care, Cleaning and Disinfection* instructions for a list of validated cleaners and disinfectants and the appropriate procedures. **Disposal** - To determine if a product has reached its end of life, visually check for cracks, damaged connectors, or similar wear or damage. Whenever the inspection reveals the product is no longer suitable for use, then follow approved medical waste disposal methods as specified by your patient care facility or local regulations. **Incident Reporting** - Any serious incident that has occurred in relation to this device should be reported to Philips and the Competent Authority of the European Economic Area (EEA) countries, including Switzerland and Turkey in which the user and/or patient is established. **Environmental Specifications - Storage**: Temperature: -20° - 60° C (-4° - 140° F); Humidity: 0 - 90% relative humidity at 60° C (140° F); **Operating**: Temperature: 0° - 50° C (32° - 122° F) Humidity: 0 - 90% relative humidity.

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Copyright © 2021
Nederland B.V.