PHILIPS

Cardiac Output Adapter Cables

M1642A

M1643A

To contact your local sales office, go to:

www.healthcare.philips.com

Manufactured for:
Philips Medizin Systeme Böblingen
GmbH
Hewlett-Packard-Strasse 2
71034 Böblingen
Germany

Subject to modification
© 2020 Philips Medizin Systeme
Böblingen GmbH
Published in Germany 06/2020
4535 649 32891

English

Instructions for Use

The cardiac output adapter cables are intended to connect catheters and injectate temperature probes to the cardiac output input of Philips patient monitors. These products are for use only in combination with Philips patient monitors when listed as accessories in the monitor's user documentation.

Recommended for	Length of monitor/ catheter cable	Product Number
Right heart catheter measurement only	2.7 m/0.2 m	M1642A
Transpulmonary measurement or Right heart catheter measurement	2.4 m/2.4 m	M1643A

For warnings, cautions, intended use, operating instructions and specifications, see your patient monitor's Instructions for Use.

Cleaning and Disinfection

Clean and disinfect Cardiac Output adapter cables before first use and before use on another patient.

Validated Cleaner and Disinfectant

Philips has validated the following cleaner and disinfectant which were tested for efficacy and use on the cables. These agents were tested by Philips for both efficacy and prevention of damage to the Philips products being cleaned and disinfected:

Cleaner:

- Enzymatic detergent - composition:

Name	Percent %
Alcohols, C9-11, ethoxylated surfactant	1 - 5
Glycerin	1 - 5
Citric Acid	1 - 5
N,N-Dimethyloctadecylamine oxide	1 - 2
Subtilisins (proteolytic enzymes)	0.1 - 1

- Please refer to the detergent manufacturer's label for proper dilution and water temperature instructions.

Disinfectant:

- 70% Isopropyl Alcohol (Chemical Abstract Service #67-63-0)

Getting Started

Before cleaning and disinfecting the product:

- Visually inspect the cables for damage or deterioration. Check for any type of damage, such as tears, cracks, or bent connector pins. Discard the product if damaged since there is a potential for incorrect readings.
- Always replace the product when there is extensive visible damage to the product (such as exposed wires or breaks).
- Visually inspect the product for moderate to severe soiling. Products with moderate to severe soiling are not intended to be cleaned or disinfected. If this situation occurs, dispose of the product.
- Secure the following supplies:
 - non-linting wipes
 - cotton swabs with tapered mini tips
 - the validated cleaner and disinfectant using the concentration recommended by the manufacturer.

Cleaning Cables

Lay the cable on a clean, flat surface. Position the connector ends of the cable to ensure that no liquid can enter the inside of the connector during cleaning.

If at any time during the procedure the non-linting wipes become visibly soiled, discard soiled wipes and use more clean wipes.

Use cotton swabs during the procedure, as needed, to clean and rinse seams, grooves, and narrow spaces.

To Clean

1. Saturate a non-linting wipe with the validated cleaner.
2. Saturate all surfaces of the product with the cleaner.
3. Allow surfaces to remain visibly wet for 3-5 minutes.
4. Dampen another non-linting wipe with cleaner.
5. Wipe all surfaces of the product until they are visibly clean. Be sure to clean all sides of the product.

To Rinse

6. Saturate a non-linting wipe with tap water.
7. Start at the monitor-end connector and wipe all surfaces of the product at least one time.
8. Visually inspect the product and repeat steps 6 and 7, if needed. If any part is not visibly clean, use a cotton swab to clean between the grooves and narrow spaces.
9. Using clean wipes, dry all surfaces of the product.

Disinfecting the Cable

Lay the cable on a clean, flat surface. Position the connector ends of the cable to ensure that no liquid can enter the connector during disinfection.

If at any time during the procedure non-linting wipes become visibly soiled, discard soiled wipes and use more clean wipes.

Use cotton swabs during the procedure, as needed, to disinfect and rinse seams, grooves, open areas, and narrow spaces.

To Disinfect

1. Saturate a non-linting wipe with the validated disinfectant.
2. Thoroughly saturate all surfaces of the product.
3. Allow surfaces to stay visibly wet for 10 minutes. Ensure that all seams, grooves and narrow spaces are saturated.

To Rinse and Complete Final Inspection

4. Saturate a non-linting wipe with tap water.
5. Start at the monitor-end connector and wipe all surfaces of the product.
6. Repeat step 5 two more times, using more wipes as needed.
7. Using clean wipes, dry the entire product.
8. Visually inspect the product to ensure all surfaces are clean and dry.

Sterilizing

Never sterilize cables.

Explanation of Product Symbols

These symbols can appear on the product and its associated equipment and packaging.

Symbols					
R_x only	Prescription use only		Refer to instructions for use		Legal manufacturer and address
	Storage temperature range		Storage pressure range		Storage humidity range
	Indicates the country and date of manufacture	 www.philips.com/IFU	Indicates that IfU is available in electronic form at www.philips.com/IFU		number of pieces in packaging
MD	Medical Device		It is unsafe to use the device in MR environments (may be printed black on device)	CE	The Cardiac Output Adapter cables comply with the requirements of Regulation (EU) 2017/745 of 5 April 2017 (Medical Device Regulation).
UDI	Indicates the location of the UDI (Unique Device Identification). Identifier symbol with the GTIN (Global Trade Item Number).	REF	Indicates the manufacturer's catalog number	LOT	Indicates the manufacturer's batch code

Incident Reporting

Any serious incident that has occurred in relation to this device should be reported to Philips and the competent authority of the member state in which the user and/or patient is established.

In this context “member state” refers to European Economic Area (EEA) countries including Switzerland and Turkey.

End of Life and Disposal

Visually inspect the cables for exposed wires, cracks, damaged insulation, worn connectors and bent pins. Do not use the cables and dispose of them according to local laws for hospital waste, if they show any of these signs.

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