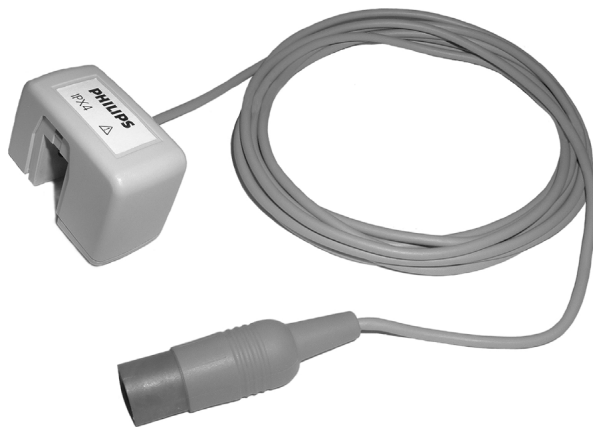
# Mainstream CO<sub>2</sub> Sensor and Airway Adapters

*Instructions for Use*

**Mainstream  
CO<sub>2</sub> Sensor  
M2501A**

**Reusable  
Airway Adapters  
M2513A, M2516A**

**Single-Patient-Use  
Airway Adapters  
M2533A, M2536A**

# PHILIPS

# Contents

|             |                                |    |              |                                      |     |
|-------------|--------------------------------|----|--------------|--------------------------------------|-----|
| •English    | Instructions for Use . . . . . | 1  | •Česky       | Návod k obsluze . . . . .            | 92  |
| •Français   | Manuel d'utilisation . . . . . | 8  | •Slovensky   | Návod na obsluhu . . . . .           | 99  |
| •Deutsch    | Gebrauchsanweisung . . . . .   | 15 | •Slovensko   | Navodila za uporabo . . . . .        | 106 |
| •Nederlands | Gebruiksaanwijzing . . . . .   | 22 | •Română      | Instrucțiuni de utilizare . . . . .  | 113 |
| •Español    | Instrucciones de uso . . . . . | 29 | •Български   | Инструкции за употреба . . . . .     | 120 |
| •Português  | Instruções de uso . . . . .    | 36 | •Eesti       | Kasutusjuhend . . . . .              | 127 |
| •Italiano   | Istruzioni d'uso . . . . .     | 43 | •Lietuviškai | Naudojimo instrukcija . . . . .      | 134 |
| •Norsk      | Bruksanvisning . . . . .       | 50 | •Latviešu    | Lietošanas instrukcijas . . . . .    | 141 |
| •Svenska    | Bruksanvisning . . . . .       | 57 | •Русский     | Инструкция по эксплуатации . . . . . | 148 |
| •Suomi      | Käyttöopas . . . . .           | 64 | •Ελληνικά    | Οδηγίες Χρήσης . . . . .             | 155 |
| •Dansk      | Brugsanvisning . . . . .       | 71 | •Türkçe      | Kullanım Talimatları . . . . .       | 162 |
| •Magyar     | Használati útmutató . . . . .  | 78 | •日本語         | ユーザーズ・ガイド . . . . .                  | 169 |
| •Polski     | Instrukcja obsługi . . . . .   | 85 | •繁體中文        | 使用說明 . . . . .                       | 176 |
|             |                                |    | •簡體中文        | 使用说明书 . . . . .                      | 183 |

# Contents

## Product Overview

### Legend for Photograph

- 1 Typical Philips CO<sub>2</sub> Measuring Device
- 2 Philips M2501A Mainstream CO<sub>2</sub> Sensor
- 3 Reusable or Disposable Philips Airway Adapter (these are not included with the sensor and must be purchased separately)

### Indications

The cable from the M2501A Mainstream CO<sub>2</sub> Sensor may be plugged into any Philips CO<sub>2</sub> measuring device that has the M2501A listed as an accessory in its Instructions for Use.

Use the M2501A Sensor in combination with any compatible Airway Adapter to acquire ETCO<sub>2</sub>, IMCO<sub>2</sub> and Respiratory Rates from an intubated adult, pediatric, or infant patient. Indications for use are in all care areas where ETCO<sub>2</sub> monitoring is prescribed by a licensed medical practitioner.

| Philips Reusable Airway Adapters                      |        |                    |                                                                                             |
|-------------------------------------------------------|--------|--------------------|---------------------------------------------------------------------------------------------|
| Model                                                 | Color  | Patient            | Application                                                                                 |
| M2513A                                                | Black  | Adult or Pediatric | For intubated patients with endotracheal tubes > 4mm. Adds approximately 5cc of dead space. |
| M2516A                                                | Red    | Infant             | For intubated patients with endotracheal tubes ≤ 4mm. Adds < 1cc of dead space.             |
| Philips Single-Patient-Use Disposable Airway Adapters |        |                    |                                                                                             |
| Model                                                 | Color  | Patient            | Application                                                                                 |
| M2533A                                                | Clear  | Adult              | For intubated patients with endotracheal tubes > 4mm. Adds approximately 5cc of dead space. |
| M2536A                                                | Violet | Infant             | For intubated patients with endotracheal tubes ≤ 4mm. Adds < 1cc of dead space.             |

## Warnings

Observe the following Warnings to avoid injury to the patient and clinician:

- Support the M2501A Sensor and Airway Adapter to prevent stress on the Endotracheal Tube.
- Do not apply excessive tension to any cable. Do not position cables or tubing in any manner that may cause entanglement or strangulation.
- Explosion Hazard: Do not use the M2501A sensor in the presence of flammable anesthetics.
- Electrical Shock Hazard: Always turn off and remove power before cleaning the M2501A Sensor.
- Failure of Operation: If the M2501A Sensor fails to respond as described, discontinue use until the situation has been corrected. Refer servicing only to qualified service personnel.
- Do not use the M2501A Sensor if it shows visible damage, has exposed electrical contacts, or has become wet from spills or condensation
- Inspect each Airway Adapter prior to use to assure that its color coding matches the patient application (refer to *Compatible Accessories*).
- Do not clean, disinfect, sterilize, or reuse Airway Adapters designed for *single-patient-use*. To do so may compromise M2501A Sensor functionality and system performance, and cause a potential hazard. Performance for single-patient-use items are not guaranteed if reused.
- Avoid infection by using only sealed single-patient-use Airway Adapters, or disinfected and sterilized reusable Airway Adapters.
- Replace any Airway Adapter in which you observe excessive moisture or secretions.
- Observe the CO<sub>2</sub> waveform. If you see changes or abnormal appearance such as an elevated baseline, check your patient and then the ventilatory circuit. Waveforms can be affected by changes in patient condition, a faulty Sensor, or faulty Airway Adapter. See *Troubleshooting Tips* on the following page for more information.

## Cautions

Observe the following Cautions to avoid product damage that could otherwise lead to safety concerns:

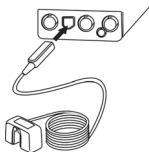
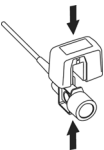
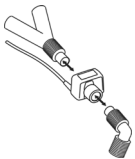
- Use only compatible accessories described in this Instructions for Use (IFU).
- Do not immerse or sterilize the M2501A Sensor, or any Airway Adapter, except as directed in this IFU.
- Before using a reusable Airway Adapter, check that the Adapter windows are dry and residue free, and have not been damaged from handling, cleaning, or sterilizing.
- Do not apply any tension to the M2501A Sensor cable.
- Do not exceed storage or operating temperature limits.
- Observe precautions for electrostatic discharge (ESD) and electromagnetic interference (EMI) to and from other equipment.

## Troubleshooting Tips

Check the following whenever a change in patient  $\text{ETCO}_2$  waveform is observed:

- Check patient status.
- Check positioning of Endotracheal Tube.
- Ensure Sensor is correctly connected to Airway Adapter.
- Check that Airway Adapter is not damaged and is securely connected to ventilator circuit.
- If problem is not identified, change the Airway Adapter.
- If problem is still not identified, change the M2501A Sensor.

## Instructions for Use

| Step | Assembly                                                                                                                                                                                                          |
|------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| 1    | <p>Plug the Sensor cable into the monitor's CO<sub>2</sub> input connector.</p>                                                  |
| 2    | <p>Choose the appropriate Airway Adapter and connect it to the Sensor head. It will click into place when seated correctly.</p>  |
| 3    | <p>Install at the proximal end of the circuit between the elbow and the ventilator Y-section.</p>                                |

## Cleaning and Disinfection

### Cautions

- The M2501A Sensor can be cleaned and disinfected, *but never sterilized by any means.*
- The M2513A and M2516A reusable Airway Adapters can be cleaned, disinfected, and sterilized.
- The M2533A and M2536A single-patient-use Airway Adapters must be disposed after use. Follow your institution's standard protocol for medical product disposables.

### Cleaning and Disinfection Schedule

Clean and disinfect the M2501A Sensor and reusable Airway Adapters (M2513A and M2516A) after every use. Be sure to follow procedures described in the following pages.

## Approved Cleaning Agents and Disinfectants

Use only recommended cleaners and disinfectants listed below. Others may damage product lifetime or cause safety hazards.

| Approved Cleaning Agent |
|-------------------------|
|-------------------------|

|                                                                                                   |
|---------------------------------------------------------------------------------------------------|
| Use only mild soap to clean the M2501A Sensor, or the M2513A and M2516A reusable Airway Adapters. |
|---------------------------------------------------------------------------------------------------|

| Approved Disinfectants |  |
|------------------------|--|
|------------------------|--|

| M2501A Mainstream CO <sub>2</sub> Sensor | M2513A, M2516A Reusable Airway Adapters |
|------------------------------------------|-----------------------------------------|
|------------------------------------------|-----------------------------------------|

- 70% Isopropyl Alcohol
- 10% Aqueous solution of sodium hypochlorite (bleach)
- Steris Coverage<sup>®</sup> Spray HB

- 70% Isopropyl Alcohol
- 10% Aqueous solution of sodium hypochlorite (bleach)
- Gluteraldehyde 2.4% solution
- Cidex<sup>®</sup>

## M2501A Sensor Cleaning and Disinfection Procedure

Perform the following procedure to clean and disinfect the M2501A Sensor.

| Step | M2501A Sensor Cleaning and Disinfection                                                                                                                                 |
|------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| 1    | Using a soft, clean cloth dampened with a mild soap solution, wipe all M2501A Mainstream CO <sub>2</sub> Sensor surfaces clean, from one end of the cable to the other. |
| 2    | Change to a water-dampened cloth and wipe the Sensor and cable completely, removing all residue.                                                                        |
| 3    | Repeat step 1 substituting one of the approved disinfectants for the cleaning agent.                                                                                    |
| 4    | Repeat step 2 using a cloth slightly-dampened with sterile water.                                                                                                       |
| 5    | Check that the Sensor windows are clean and dry before reuse.                                                                                                           |

## Reusable Airway Adapters Cleaning and Disinfection Procedure

Perform the following procedure to clean and disinfect the reusable M2513A and M2516A Airway Adapters.

| Step | M2513A, M2516A Cleaning and Disinfection                                                                                      |
|------|-------------------------------------------------------------------------------------------------------------------------------|
| 1    | Wash the Airway Adapter with a mild soap solution, being sure to completely clean the inside and outside surfaces.            |
| 2    | Rinse the Airway Adapter in clean water, then soak it in one of the approved disinfectants described earlier in this section. |
| 3    | Rinse the Airway Adapter in sterile water, then allow it to dry completely.                                                   |

**NOTE:** After cleaning and disinfection, the reusable Airway Adapters can be *sterilized* using approved sterilization methods described in the following section.

## Sterilization

Perform the following procedure to sterilize the reusable M2513A and M2516A Airway Adapters.

| Step | M2513A, M2516A Sterilization                                                                                                                                                                                                                                                                                                                   |
|------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| 1    | Clean and disinfect the Airway Adapter.                                                                                                                                                                                                                                                                                                        |
| 2    | After disinfecting, immerse and soak the Airway Adapter in one of the following sterilants <sup>a</sup> :<br>Cidex <sup>®</sup> or equivalent Gluteraldehyde 2.4% solution: Follow the manufacturer's instructions.<br><sup>b</sup> PeraSafe <sup>®</sup> or equivalent peracetic acid 0.26% solution: Follow the manufacturer's instructions. |
| 3    | Allow appropriate aeration time before reuse.                                                                                                                                                                                                                                                                                                  |

- a. Alternatively, you may autoclave the M2513A at 121°C (250°F) for 20 minutes, rather than soaking it in sterilant. Never autoclave the M2516A.  
b. PeraSafe should not be used in the U.S.

## CO<sub>2</sub> Sensor Adapter Zero

An “Adapter Zero” is a quick procedure that adjusts the M2501A Sensor to the optical characteristics of each compatible Airway Adapter. The procedure is necessary only:

- When switching the M2501A Sensor from one Airway Adapter type to another, such as when switching from a disposable to a reusable Airway Adapter.
- When requested by the CO<sub>2</sub> monitor.

For additional information about the Adapter Zero procedure, reference the CO<sub>2</sub> monitor’s Instructions for Use.

## Maintenance and Testing

Refer to the CO<sub>2</sub> monitor’s Installation and Service Guide for maintenance and testing of the M2501A Sensor.

**NOTE:** Test procedures in the CO<sub>2</sub> monitor’s Service Guide specify use of the gas and gas cylinder regulator listed below.

| Gas Cylinder Regulator and Verification Gas |                                                                                   |
|---------------------------------------------|-----------------------------------------------------------------------------------|
| M2506A Gas                                  | Gas used to check M2501A Sensor operation.                                        |
| M2505A Gas Cylinder Regulator               | Provides gas flow control when using M2506A Gas to check M2501A Sensor operation. |

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This product complies with the Medical Device Directive  
93/42/EEC.

All devices described herein are latex-free.

**R<sub>x</sub> only**

4512 610 24901

For more information, please call your local Philips sales office listed in your telephone directory or a Philips regional office listed below for the location of your nearest sales office.

**United States:**

Philips Medical Systems  
North America Corporation  
3000 Minuteman Road  
Andover, MA 01810-1099 USA  
(800) 225-0230

**Canada:**

Philips Medical Systems Canada  
281 Hillmount Road  
Markham, ON Canada  
L6C 2S3  
(800) 291-6743

**Latin America Headquarters:**

Philips Medical Systems  
1550 Sawgrass Corporate Parkway  
#300  
Sunrise, FL 33323 USA  
Tel: (954) 835-2600  
Fax: (954) 835-2626

**Europe, Middle East and Africa:**

Philips Medizin Systeme Böblingen GmbH  
Cardiac and Monitoring Systems  
Hewlett-Packard-Str. 2  
71034 Böblingen, Germany  
Fax: (+49) 7031 463 1552

**Asia Pacific Headquarters:**

Philips Medical Systems  
30/F Hopewell Centre  
17 Kennedy Road  
Wanchai  
Hong Kong  
Tel: (852) 2821 5888  
Fax: (852) 2527 6727

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© Copyright 2005, 2007 Koninklijke Philips Electronics N.V.
All rights reserved.