PHILIPS

M2785A, M2786A

Flow Sensors

M2781A, M2782A, M2783A


Combined CO₂/Flow Sensors

Contents

Instructions for Use (English)	1
Manuel d'utilisation (Français)	3
Gebruiksaanwijzing (Deutsch)	5
Gebruiksaanwijzing (Nederlands)	7
Instrucciones de Uso (Español)	9
Instruções de Uso (Português)	11
Istruzioni d'uso (italiano)	13
Bruksanvisning (norsk)	15
Bruksanvisning (svensk)	17
Käyttöohje (suomi)	19
Brugsanvisning (dansk)	21
Használati útmutató (Magyar)	23
Instrukcja obsługi (Polski)	25
Návod na použití (česky)	27
Návod na použitie (slovensky)	29
Navodila za uporabo (slovensko)	31
Instrucțiuni de utilizare (Română)	33
Инструкции за използване (Български)	35
Kasutusjuhend (eestikeelne)	37
Naudojimo instrukcija (lietuviškai)	39
Lietošanas pamācība (latviski)	41
Инструкция по эксплуатации (на русском языке)	43
Οδηγίες Χρήσης (Ελληνικά)	45
Kullanım Talimatları (Türkçe)	47
ユーザーズ・ガイド (日本語)	49
使用說明 (繁體中文)	51
使用说明书 (简体中文)	53

Applying the Sensor

Use these sensors with the Philips M1014A Spirometry Module and an IntelliVue patient monitor on ventilated patients to measure airway flow, volume and pressure.

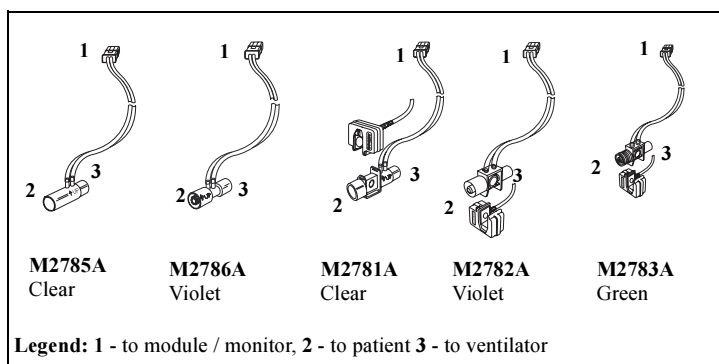Definition of Symbols Used on Product or Labelling			
	Caution. See documentation		For use with endotracheal tubes of indicated size
	For adult use		For pediatric use
	For infant use		For neonatal use

WARNINGS

- Periodically check the sensor and tubing for excessive moisture or secretion build up.
- A system leak, (for example, caused by un-cuffed endotracheal tubes or a damaged sensor) may significantly effect readings of flow, volume, pressure and other respiratory mechanics parameters.
- Reuse poses a patient hazard. Do not disassemble, clean, immerse, disinfect or sterilize.
- Do not apply tension to the flow sensor tubing.
- Do not allow the sensor in the patient circuit when not connected to the monitor.

Instructions for Use (adhere to this sequence)

1. Remove the sensor assembly from the package and verify that no component is damaged.
2. Click the connector into place in the Flow Sensor receptacle on the monitor before connecting to the breathing circuit.
3. If using a combined CO₂/Flow Sensor, press the CO₂ sensor onto the adapter until it clicks into place. Perform a zero procedure with room air only if prompted by the monitor.
4. Install the sensor between the endotracheal tube and the ventilator circuit wye. If using a Heat Moisture Exchanger (HME), place the sensor between HME and wye, positioning it so that the tubing and arrow on the sensor point upwards.
5. Ensure that the patient ventilator circuit's integrity is not compromised after inserting the sensor.

As with all flow measuring devices, adverse conditions may affect flow measurement accuracy. Do not

place the combined CO₂/Flow sensor between the endotracheal tube and the elbow (pediatric/adult circuit) as patient secretions may block the adapter windows. Do not touch the adapter windows. Ensure the striped tubing of the flow sensor is always proximal to the patient. Keeping the tubing upright prevents moisture from draining into the tubing of the flow sensor or combined CO₂/Flow sensor. Periodically check the flow sensor and tubing. Perform a purge cycle if droplets are visible within the sensor or tubing. Repeat if necessary. If purging does not remove the droplets, replace the sensor.

CAUTIONS

- Do not apply excessive tension to any sensor cable or pneumatic tubing.
- Excessive moisture in the sensor may affect the accuracy of the flow measurement. Inserting the sensor in the ventilator circuit with the tubes upright helps avoid effects of excessive moisture.
- Remove the sensor from the circuit when delivering an aerosolized medication as increased medication viscosity may contaminate the sensor windows, causing the sensor to fail prematurely.

Sensor Disposal

Follow your country's laws and local regulations regarding hospital waste disposal.

The components of this product with patient contact are latex free.

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Copyright © 2015
Böblingen GmbH