Noninvasive Blood Pressure Cuffs

Single Care

REF

989803182281 989803182291
989803182301 989803182311
989803182321

Gentle Care

REF

M4572B, M4573B M4574B,
M4575B, M4576B M4577B,
M4578B

Multi Care

REF

989803183311 989803183321
989803183331 989803183341
989803183351 989803183361
989803183371

Easy Care

REF

M4552B, M4553B M4554B,
M4555B M4556B, M4557B
M4558B, M4559B


Comfort Care

REF

M1571A, M1572A, M1573A,
M1574A, M1574XL, M1575A,
M1576A

Instructions for Use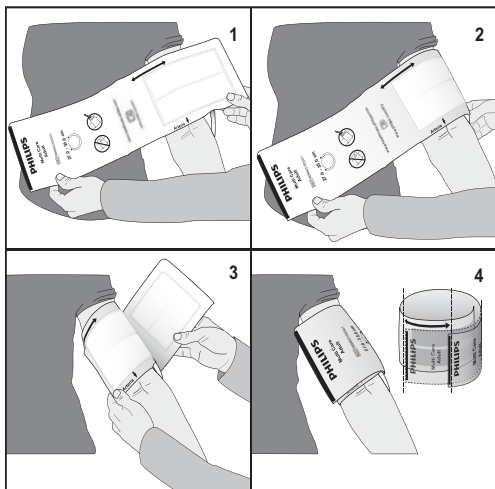

PHILIPS

(EN) Noninvasive Blood Pressure Cuffs Instructions for Use	1	(NL) Gebruiksaanwijzing voor NIBD-manchetten	20
(BG) Указания за употреба на маншети за неинвазивно кръвно налягане	1	(NO) Brukerhåndbok for noninvasive blodtrykksmansjetter	21
(CS) Návod k obsluze manžet pro neinvazivní měření krvniho tlaku	2	(PL) Instrukcja obsługi mankietów do nieinwazyjnego pomiaru ciśnienia krwi	22
(DA) Brugervejledning til noninvasive blodtrykksmansjetter	3	(PT-BR) Instruções de utilização das braçadeira para medição da pressão arterial	24
(DE) Blutdruckmanschetten – Gebrauchsanweisung	4	(PT) Instruções de uso para manguitos de pressão arterial não invasiva	23
(EL) Οδηγίες χρήσης για περιχειρίδες αναίμακτης πίεσης	5	(RO) Instrucțiuni de utilizare pentru manșetele de tensiune arterială neinvazivă	25
(ES) Instrucciones de uso de los manguitos de presión no invasiva	6	(RU) Инструкция по эксплуатации манжет для измерения АД	26
(ET) Vererõhumansettide kasutusjuhend	7	(SK) Návod na použitie manžiet na neinvazívne meranie krvného tlaku	27
(FI) Noninvasiivisten verenpainemansettien käyttöohjeet	8	(SL) Navodila za uporabo manšet za neinvazivno merjenje krvnega tlaka	28
(FR) Manuel d'utilisation des brassards de PNI	9	(SR) Uputstvo za upotrebu manžetni za neinvazivno merenje krvnog pritiska	29
(HR) Manžete za neinvazivno mjerenje tlaka Upute za uporabu	10	(SV) Bruksanvisning till noninvasiva blodtrykksmansjetter	30
(HU) Nem invazív vérnyomásmérő mandzsetták – Használati útmutató	11	(TR) Noninvazif Kan Basıncı Manşetleri Kullanım Talimatları	31
(ID) Petunjuk Penggunaan Manset Tekanan Darah Non-Invasif	12	(UK) Інструкції з використання манжет для неінвазивного вимірювання тиску	32
(IT) Istruzioni d'uso dei bracciali per la misurazione della pressione sanguigna	13	(VI) Hướng dẫn sử dụng Băng quấn đo huyết áp không xâm lấn	33
(JA) 非観血圧測定用カフ ユーザーズガイド	14	(ZH-CN) 无创血压袖带使用说明书	34
(KK) Инвазивті емес қан қысымын өлшеу манжеттерінің пайдалану нұсқаулары	15	(ZH-TW) 非侵入式血壓壓脈帶使用說明	35
(KO) 비침습 혈압 커프 사용 설명서	16		
(LT) Neinvazinių kraujų spaudimo matavimo veržiamųjų raišių naudojimo instrukcijos	17		
(LV) Neinvazīvu asinsspiediena aprūko lietošanas instrukcija	18		
(MK) Упатства за користење на манжетите за неинвазивно мерење на крвен притисок	19		

	EN: Manufacturer; BG: Производител; CS: Výrobce; DA: Producent; DE: Hersteller; EL: Κατασκευαστής; ES: Fabricante; ET: Tootja; FI: Valmistaja; FR: Fabricant; FR: Producent; HU: Gyártó; IT: Produttore; IT: Fabbicante; JA: 製造業者; KK: Өндүрүшү; KO: 제조자; LT: gamintojas; LV: Ražotājs; MK: Производител; NL: Fabrikant; NO: Produsent; PL: Producent; PT: Fabricante; PT-BR: Fabricante; RO: Producător; RU: Производитель; SK: Výrobca; SL: Proizvajalec; SR: Proizvođač; SV: Tillverkare; TR: Üretici; UK: Виробник; VI: Nhà sản xuất; ZH-CN: 制造商; ZH-TW: 製造商
EC REP	EN: Authorized Representative in the European Community; BG: Упълномощен представител в Европейската общност; CS: Autorizovaný zástupce v Evropském společenství; DA: Autoriseret repræsentant i EU; DE: Autorisierte EU-Vertretung; EL: Εξουσιοδοτημένος αντιπρόσωπος για την Ευρωπαϊκή Ένωση; ES: Representante autorizado en la Unión Europea; ET: Volitatud esindaja Euroopa Ühenduses; FI: Valtuutettu edustaja Euroopan yhteisössä; FR: Représentant autorisé pour la Communauté européenne; HR: Ovlašteni zastupnik u Europskoj zajednici; HU: Hivatalos képviselő az Európai Közösségben; ID: Perwakilan UE Resmi; IT: Rappresentante autorizzato nella Comunità europea; JA: Authorized EU Representative; KK: Еуропалык қауымдастықтағы уәкілетті өкіл; KO: 유럽 공인 대리점; LT: įgaliojasis atstovas Europos Bendrijoje; LV: Pilnvarotais pārstāvis Eiropas Kopienā; MK: Овластен претставник во Европската заедница; NL: Officiële vertegenwoordiger in de Europese Unie; NO: Autorisert EU-representant; PL: Autoryzowany przedstawiciel na terenie Wspólnoty Europejskiej; PT: Representante autorizado na Comunidade Europeia; PT-BR: Representante autorizado na União Europeia; RO: Reprezentant autorizat în Comunitatea Europeană; RU: Уполномоченный представитель в странах Европейского сообщества; SK: Autorizovaný zástupca pre Európske spoločenstvo; SL: Pooblaščen zastopnik za Evropsko skupnost; SR: Ovlašćeni predstavnik Evropske zajednice; SV: Auktoriserad representant i Europeiska unionen; TR: Yetkili Avrupa Topluluğu Temsilcisi; UK: Уповноважений представник на територият ЕС; VI: Đại diện được ủy quyền ở Cộng đồng châu Âu; ZH-CN: 欧盟授权代表; ZH-TW: 授權的歐洲共同體代表
REF	EN: Catalogue Number; BG: Каталоген номер; CS: Katalogové číslo; DA: Katalognummer; DE: Katalognummer; EL: Αριθμός καταλόγου; ES: Número de catálogo; ET: Katalooginumber; FR: Numéro de référence; HR: Kataloški broj; HU: Katalógusszám; ID: Nomor Katalog; IT: Numero di catalogo; JA: カタログ番号; KK: Каталог бойнша нөмүрү; KO: 카탈로그 번호; LT: katalogo numeris; LV: Kataloga numurs; MK: Каталогски број; NL: Catalogusnummer; NO: Katalognummer; PL: Numer katalogowy; PT: Número de catálogo; PT-BR: Número de catálogo; RO: Număr de catalog; RU: Номер по каталогу; SK: Katalógové číslo; SL: Kataloška številka; SR: Kataloški broj; SV: Katalognummer; TR: Katalog Numarası; UK: Номер за каталогом; VI: Số catalog; ZH-CN: 目录号; ZH-TW: 目錄編號
LOT	EN: Batch Code; BG: Партиден код; CS: Kód série; DA: Kode for parti; DE: Chargenbezeichnung; EL: Κωδικός παρτίδας; ES: Código de lote; ET: Partikood; FI: Eräkkoodi; FR: Code de lot; HR: Sifra serije; HU: Tételszám; ID: Kode Batch; IT: Codice lotto; JA: 製造番号又は製造記号; KK: Партия коды; KO: 배치 코드; LT: partijos kodas; LV: Sērijas numurs; MK: Код на серија; NL: Batchcode; NO: Partikode; PL: Kod partii; PT: Código de lote; PT-BR: Código de lote; RO: Codul lotului; RU: Код партии; SK: Kód šarže; SL: Številka serije; SR: Broj serije; SV: Batchcod; TR: Parti Kodu; UK: Код партії; VI: Mã lô; ZH-CN: 批号; ZH-TW: 批次碼
	EN: Quantity per box; BG: Количество в кутия; CS: Množství v krabici; DA: Antal pr. æske; DE: Menge pro Karton; EL: Ποσότητα ανά συσκευασία; ES: Cantidad por caja; ET: Arv karbis; FI: Määrä laatikossa; FR: Quantité par boîte; HR: Količina po kutiji; HU: Mennyiség dobozoként; ID: Isi per kotak; IT: Quantità per scatola; JA: 1箱あたりの数量; KK: Бир корпактагы саны; KO: 상자당 수량; LT: kiekis dėžutėje; LV: Skaitis kastītē; MK: Количина по кутија; NL: Hoeveelheid per doos; NO: Antall per pakke; PL: Liczba sztuk w kartonie; PT: Quantidade por caixa; PT-BR: Quantidade por caixa; RO: Cantitate per cutie; RU: Количество в коробке; SK: Množstvo v škatuli; SL: Količina v škatli; SR: Količina u kutiji; SV: Antal per låda; TR: Kutu başına miktar; UK: Кількість в одній коробці; VI: Số lượng mỗi hộp; ZH-CN: 每盒数量; ZH-TW: 每盒數量
	EN: Storage Temperature limits; BG: Температурни граници на съхранение; CS: Limity teploty skladování; DA: Temperaturgrænser for opbevaring; DE: Temperaturbereich bei Lagerung; EL: Όρια θερμοκρασίας φύλαξης; ES: Límites de temperatura de almacenamiento; ET: Säilitamise piirtemperatuurid; FI: Säilytyslämpötila; FR: Limites de température de stockage; HR: Temperaturna ograničenja za pohranu; HU: Tárolási hőmérséklet-tartomány; ID: Batas Suhu Penyimpanan; IT: Limiti di temperatura a magazzino; JA: 保管温度制限; KK: Сақтау температурасының шектері; KO: 보관 온도 한계; LT: laikymo temperatūros ribos; LV: Uzglabāšanas temperatūras ierobežojumi; MK: Температурни ограничувања за складирање; NL: Opslagtemperatuurlimiten; NO: Temperaturgrense; PL: Zakres temperatury przechowywania; PT: Temperatura de armazenamento; PT-BR: Limites de temperatura de armazenamento; RO: Limite temperatură de depozitare; RU: Температурные ограничения при хранении; SK: Hranicné skladovacie teploty; SL: Temperaturne omejitve za shranjevanje; SR: Ograničenja temperature za skladištenje; SV: Temperaturgränser vid förvaring; TR: Saklama sıcaklığı limitleri; UK: Обмеження температури; VI: Giới hạn nhiệt độ bảo quản; ZH-CN: 存放温度范围; ZH-TW: 儲存溫度限制
	EN: Consult instructions for use; BG: Консултирайте се с указанията за употреба; CS: Prostudujte si návod k použití; DA: Se brugsanvisningen; DE: Gebrauchsanweisung beachten; EL: Συμβουλευτείτε τις οδηγίες χρήσης; ES: Consultar las instrucciones de uso; ET: Lugege kasutusjuhendit; FI: Lue käyttöohjeet; FR: Consulter le manuel d'utilisation; HR: Pogledajte upute za uporabu; HU: Lásd a használati útmutatót; ID: Baca petunjuk penggunaan; IT: Consultare le Istruzioni d'uso; JA: ユーザーガイド参照; KK: Пайдалану нұсқаулығын қараңыз; KO: 사용 설명서 참조; LT: žr. naudojimo instrukcijas; LV: Skatiet lietošanas instrukcijas; MK: Погледнете во упутствата за користење; NL: Raadpleeg de gebruiksaanwijzing; NO: Se brukerhåndboken; PL: Zapoznać się z instrukcją obsługi; PT: Consultar as instruções de utilização; PT-BR: Consulte as instruções de uso; RO: Consultati instrucțiunile de utilizare; RU: См. инструкции по эксплуатации; SK: Pozrite si návod na použitie; SL: Glejte navodila za uporabo; SR: Pogledajte uputstvo za upotrebu; SV: Läs bruksanvisningen; TR: Kullanım talimatlarına bakınız; UK: Читайте інструкції з використання; VI: Tham khảo hướng dẫn sử dụng; ZH-CN: 参见使用说明; ZH-TW: 請見說明

 YYYY-MM-DD	EN: Date and Country of Manufacture; BG: Дата и държава на производство; CS: Datum a země výroby; DA: Fremstillingsdato og -land; DE: Herstellungsdatum und -land; EL: Ημερομηνία και χώρα κατασκευής; ES: Fecha y país de fabricación; ET: Valmistamiskuupäev ja -riik; FI: Valmistuspäivämäärä ja -maa; FR: Date et pays de fabrication; HR: Datum i zemlja proizvodnje; HU: Gyártás helye és dátuma; ID: Tanggal dan Negara Produksi; IT: Data e paese di fabbricazione; JA: 製造年月 / 製造国; KK: Өндүрілгөн күні жана өлкө; KO: 제조일 및 제조국; LT: pagaminimo data ir šalis; LV: Ražošanas datums un valsts; MK: Датум и земја на производство; NL: Datum en land van productie; NO: Produktionsdato og -land; PL: Data i kraj produkcji; PT: Data e país de fabrico; PT-BR: Data e país do fabricante; RO: Data și țara de producție; RU: Дата и страна производства; SK: Dátum a krajina výroby; SL: Datum in država proizvodnje; SR: Datum i zemlja proizvodnje; SV: Tillverkningsdatum och -land; TR: Üretim Tarih ve Ülkesi; UK: Дата та країна виробник; VI: Ngày tháng và quốc gia của nhà sản xuất; ZH-CN: 生产日期和地区; ZH-TW: 製造的日期和國家/地區
	EN: Caution; BG: Внимание; CS: Varování; DE: Vorsicht; EL: Προσοχή; ES: Precaución; ET: Ettevaatus; FI: Varoitutus; FR: Mise en garde; HR: Oprez; HU: Figyelem; ID: Perhatian; IT: Attenzione; JA: 注意; KK: Ескертүү; KO: 주의; LT: perspėjimai; LV: Uzmanību; MK: Внимание; NL: Let op; NO: OBS!; PL: Przestroga; PT: Cuidado; PT-BR: Cuidado; RO: Atentie; RU: Внимание; SK: Varovanie; SL: Pozor; SR: Oprez; SV: Viktigt; TR: Dikkat; UK: УВАГА!; VI: Thận trọng; ZH-CN: 小心; ZH-TW: 小心事項
R_x only	EN: Prescription Use Only; BG: Само по лекарско предписание; CS: Pouze na předpis; DA: Receptpligtigt; DE: Anwendung nur auf ärztliche Verordnung; EL: Χρήση μόνο κατόπιν ιατρικής συνταγής; ES: Solo bajo prescripción médica; ET: Kasutamiseks üksnes retsepti alusel; FI: Käyttö vain lääkärin määräyksestä; FR: Sur prescription uniquement; HR: Samo na recept; HU: Csak orvosi rendelvényre; ID: Hanya dengan Resep Dokter; IT: Solo su prescrizione medica; JA: 医家向け医療機器; KK: Тек дөрігөрдүн тағайындауы бойынша пайдаланылады; KO: 처방 사용에 한함; LT: naudoti tik skyrys gydytojų; LV: Lietošana tikai pēc noteikuma; MK: Само за припишана употреба; NL: Uitsluitend op recept; NO: Receptbelagt; PL: Wyłącznie na zlecenie lekarza; PT: Utilização apenas com prescrição; PT-BR: Apenas sob prescrição médica; RO: Doar la recomandarea medicului; RU: Только по назначению врача; SK: Iba na lekársky predpis; SL: Izdaja se samo na recept; SR: Samo na lekarski recept; SV: Får endast användas enligt läkares ordination; TR: Sadece Reçeteye Kullanılır; UK: Застосування лише за призначенням; VI: Chỉ sử dụng theo chỉ định của bác sĩ; ZH-CN: 仅凭处方; ZH-TW: 僅限處方使用
 %RH	EN: Storage Humidity Range; BG: Диапазон на влажността при съхранение; CS: Rozsah skladovací vlhkosti; DA: Fugtighedsområde ved opbevaring; DE: Luftfeuchtigkeitsbereich bei Lagerung; EL: Εύρος τιμών υγρασίας φύλαξης; ES: Rango de humedad de almacenamiento; ET: Hoiustamise niiskussvahemik; FI: Säilytyskosteus; FR: Limites d'humidité de stockage; HR: Raspon vlažnosti za pohranu; HU: Tárolási páratartalom-tartomány; ID: Rentang Kelembapan Penyimpanan; IT: Umidità per la conservazione a magazzino; JA: 保管湿度範囲; KK: Сақтау ылгалдылығынын ауқымы; KO: 보관 습도 범위; LT: laikymo drėgmės intervalas; LV: Uzglabāšanas mitruma diapazons; MK: Оперативна влажност за складирање; NL: Bereik opslagvochtigheid; NO: Fuktighetsområde for oppbevaring; PL: Zakres wilgotności przechowywania; PT: Intervalo da humidade de armazenamento; PT-BR: Faixa de umidade de armazenamento; RO: Interval de umiditate pentru depozitare; RU: Диапазон влажности при хранении; SK: Rozsah vlhkosti pri skladovaní; SL: Razpon vlažnosti za shranjevanje; SR: Raspon vlažnosti pri skladištenju; SV: Luftfuktighetsområde vid förvaring; TR: Saklama Nem Aralığı; UK: Діапазон вологості зберігання; VI: Phạm vi độ ẩm bảo quản; ZH-CN: 存放湿度范围; ZH-TW: 儲存濕度範圍
MD	EN: Medical Device; BG: Медицинско изделие; CS: Zdravotnický přístroj; DA: Medicinsk udstyr; DE: Medizinprodukt; EL: Ιατροτεχνολογικό προϊόν; ES: Dispositivo médico; ET: Meditsiiniseade; FI: Lääkinnällinen laite; FR: Dispositif médical; HR: Medicinski proizvod; HU: Orvostechnikai eszköz; ID: Perangkat Medis; IT: Dispositivo medico; JA: 医療機器; KK: Медициналык құрылғы; KO: 의료가기; LT: medicinos prietaisai; LV: Medicīnas ierīce; MK: Медицински уред; NL: Medisch apparaat; NO: Medisinsk utstyr; PL: Wyrób medyczny; PT: Dispositivo médico; PT-BR: Equipamento médico; RO: Dispozitiv medical; RU: Медицинское устройство; SK: Zdravotnícka pomôcka; SL: Medicinski pripomoček; SR: Medicinski uređaj; SV: Medicinteknisk produkt; TR: Tıbbi Cihaz; UK: Медичиний пристрій; VI: Thiết bị y tế; ZH-CN: 医疗设备; ZH-TW: 醫療設備
UDI	EN: Unique Device Identifier; BG: Уникателен идентификатор на устройството; CS: Jediněný indikátor zařizení; DA: Unik enhedsindikator; DE: Eindeutige Gerätekennung; EL: Μονοδικό αναγνωριστικό συσκευής; ES: Identificación exclusiva del dispositivo; ET: Seadme kordumatu identifitseerimistunnus; FI: Laitteen yksilöivä tunnus; FR: Identifiant unique de l'appareil; HR: Jedinstveni identifikator proizvoda; HU: Egyedi eszközazonosító; ID: Indikator Perangkat Unik; IT: Identificatore univoco del dispositivo; JA: 機器固有識別子; KK: Күрүлгөнүн бирегей идентификаторы; KO: 의료기기 고유식별 코드; LT: unikalus prietaiso identifikatorius; LV: Ierīces unikālais indikators; MK: Уникатна идентификација на уредот; NL: Unieke apparaatindicator; NO: Unik enhetsindikator; PL: Unikalny identyfikator wyrobu; PT: Indicador único de dispositivo; PT-BR: Indicador único de dispositivo; RO: Indicator unic al dispozitivului; RU: Уникальный идентификатор устройства; SK: Jediný identifikátor zariadenia; SL: Edinstven identifikator pripomočka; SR: Jedinstveni identifikacija uređaja; SV: Unik produktindikator; TR: Benzersiz Cihaz Göstergesi; UK: Унікальний ідентифікатор пристрою; VI: Chỉ báo thiết bị duy nhất; ZH-CN: 唯一设备标识; ZH-TW: 唯一裝置指示
 Single patient - multiple use	EN: Single patient - multiple use; BG: За употреба при един пациент – множество употреби; CS: Použití na jednom pacientovi - opakované použití; DA: Flergangsbrug — én patient; DE: Ein-Patienten-Produkt, mehrfach anwendbar; EL: Πολυαπλή χρήση — σε έναν μόνο ασθενή; ES: Uso para un único paciente (varios usos); ET: Kasutamiseks ühel patsiendil mitu korda; FI: Yhden potilaan käyttöön — useita käyttökertoja; FR: Monopatient — à usage multiple; HR: Višestruka uporaba na jednom pacijentu; HU: Egy beteg – többszöri alkalmazás; ID: Satu pasien - digunakan beberapa kali; IT: Per uso su un solo paziente - più utilizzi; JA: 単一患者に限り、繰り返し使用可能; KK: Бир емделүүлөргө бирнеше рет пайдалануу арналган; KO: 환자 1인용 - 다회용; LT: daugkartinis, vienas pacientui; LV: Lietošanai vienam pacientam vairākas reizes; MK: Еден пациент – повеќекратна употреба; NL: Voor gebruik bij één patiënt - hergebruik mogelijk; NO: Brukes til én pasient – brukes flere ganger; PL: Do wielokrotnego użytku u jednego pacjenta; PT: Um único paciente – utilização múltipla; PT-BR: Único paciente - vários usos; RO: Utilizare pentru un singur pacient – utilizare multiplă; RU: Для многократного индивидуального использования; SK: Použitie iba na jednom pacientovi – viacsobné použitie; SL: Za enega pacienta – več uporab; SR: Višestruka upotreba samo na jednom pacijentu; SV: Enpatientsbruk – flera användningar; TR: Tek hastada çoklu kullanım; UK: Бараторазове використання можливо тільки для одного пацієнта; VI: Dùng cho một bệnh nhân - dùng nhiều lần; ZH-CN: 单一病人 - 多次使用; ZH-TW: 單一病患 - 多次使用

	EN: Patient limb circumference range (cm); BG: Диапазон за обиколка на крайника на пациент (cm); CS: Rozsah obvodu pacientovy končiny (cm); DA: Interval for patientekstremitetomkreds (cm); DE: Extremitätenumfang (cm); EL: Εύρος περιφέρειας μέλους ασθενούς (cm); ES: Contorno de la extremidad del paciente (cm); ET: Patsiendi jäseme ümbermõõdu vahemik (cm); FI: Potilaan raajan ympärysmitta (cm); FR: Circonférence du membre du patient (cm); HR: Raspon opsega uda pacijenta (cm); HU: A beteg végtagjának kerülete (cm); ID: Rentang lingkaran tangan/kaki pasien (cm); IT: Circonferenza dell'arto del paziente (cm); JA: 患者の肢周長範囲 (cm); KK: Емделуші аяқ-қолының жуандығы (cm); KO: 환자의 사지 둘레 범위 (cm); LT: paciento galūnės apimties intervalas (cm); LV: Pacienta ekstremitātes apkārtmēra diapazons (cm); MK: Оңсег на обем на екстремитет на пациентот (cm); NL: Omtrek arm/been patiënt (cm); NO: Pasientens ekstremitetsomkrets (cm); PL: Zakres obwodu kończyny pacjenta (cm); PT: Intervalo da circunferência do membro do paciente (cm); PT-BR: Circunferência do membro superior ou inferior do paciente (cm); RO: Intervalul circumferinței membrului pacientului (cm); RU: Размер окружности конечности пациента (cm); SK: Rozsah obvodu končatiny pacienta (cm); SL: Razpon obsega okončine pacienta (cm); SR: Raspon obima ruke pacijenta (cm); SV: Omkretsintervall för patientens extremitet (cm); TR: Hasta ekstremitesi çevre aralığı (cm); UK: Діапазон обхвату кінцівок пацієнтів (cm); VI: Phạm vi chu vi chi bệnh nhân (cm); ZH-CN: 病人肢体围度范围 (cm); ZH-TW: 病患肢體圍周長範圍 (cm)
	EN: Proper application of cuff; BG: Правилно прилагане на маншета; CS: Správná aplikace manžety; DA: Korrekt påsætning af manchetter; DE: Richtige Anwendung der Manschette; EL: Σωστή εφαρμογή περιχειρίδας; ES: Aplicación correcta del manguito; ET: Manseti õige paigaldus; FI: Mansetin oikea asettaminen; FR: Application correcte du brassard; HR: Pravilna primjena manžete; HU: A mandzsetta helyes alkalmazása; ID: Pemasangan manset yang benar; IT: Applicazione corretta del bracciale; JA: 適切な装着; KK: Манжет дұрыс тағылған; KO: 올바른 커프 부착; LT: tinkama veržiamojo raiščio naudojimo sritis; LV: Atbilstošs aproces lietojums; MK: Правилно поставување на манжетната; NL: Correct aangebrachte manchet; NO: Riktig bruk av mansjetten; PL: Mankiet założony prawidłowo; PT: Aplicação correta da braçadeira; PT-BR: Aplicação adequada do manguito; RO: Aplicarea corespunzătoare a manșetei; RU: Правильное наложение манжеты; SK: Správna aplikácia manžety; SL: Pravilna namestitve manšete; SR: Pravilna upotreba manžetne; SV: Rätt användning av manchett; TR: Doğru manşet uygulaması; UK: Правильне розміщення манжети; VI: Cách gắn băng quấn đúng; ZH-CN: 正确使用袖带; ZH-TW: 壓脈帶的正確施用方式
	EN: Improper application of cuff; BG: Неправилно прилагане на маншета; CS: Nesprávná aplikace manžety; DA: Forkert påsætning af manchetter; DE: Falsche Anwendung der Manschette; EL: Λανθασμένη εφαρμογή περιχειρίδας; ES: Aplicación incorrecta del manguito; ET: Manseti vale paigaldus; FI: Mansetin väärä asettaminen; FR: Application incorrecte du brassard; HR: Nepravilna primjena manžete; HU: A mandzsetta helytelen alkalmazása; ID: Pemasangan manset yang salah; IT: Applicazione errata del bracciale; JA: 不適切な装着; KK: Манжет дұрыс тағылмаған; KO: 잘못된 커프 부착; LT: netinkama veržiamojo raiščio naudojimo sritis; LV: Neatbilstošs aproces lietojums; MK: Неправилно поставување на манжетната; NL: Incorrect aangebrachte manchet; NO: Feil bruk av mansjetten; PL: Mankiet założony nieprawidłowo; PT: Aplicação incorreta da braçadeira; PT-BR: Aplicação inadequada do manguito; RO: Aplicarea necorespunzătoare a manșetei; RU: Неправильное наложение манжеты; SK: Nesprávna aplikácia manžety; SL: Nepravilna namestitve manšete; SR: Nepravilna upotreba manžetne; SV: Felaktig användning av manchett; TR: Hatalı manşet uygulaması; UK: Неправильне розміщення манжети; VI: Cách gắn băng quấn sai; ZH-CN: 错误使用袖带; ZH-TW: 壓脈帶的不當施用方式

EN - Instructions for Use

Associated Documents - NBP Cuffs and Air Hoses Addendum - Care, Cleaning, and Disinfection; Air Hoses for NBP Cuffs Instructions for Use.

Intended Use - Philips multi-patient and single-patient cuffs are to be used with compatible devices intended for use by, or under the supervision of, a licensed physician or other healthcare provider for the noninvasive measurement of infant, pediatric, and adult human blood pressure. **Indications** - Philips multi-patient and single-patient cuffs are indicated for the measurement of noninvasive blood pressure and pulse rate for monitoring purposes. These devices are limited by the indications for use of the connected monitor in healthcare facilities. These devices are intended to interact with patient intact skin only. **Contraindications** - There are no known contraindications. **Compatibility** - NBP Cuffs and Air Hoses can be used with any monitor or defibrillator that lists the cuffs and air hoses as accessories in that product's Instructions for Use.

Warnings - Turn off auto function or remove cuff from monitor when not on patient. This may lead to cuff leaks. • When cleaning cuffs, use care to prevent fluid from entering tubing. Use end cap, if provided with cuff. Do not immerse tubing in any fluids. This may lead to cuff or monitor failure. • Avoid placing cuff on patient in a manner that can lead to a cuff tube becoming kinked (a kinked tube can cause inaccurate readings). • Single-patient (disposable) cuffs are not intended for use on more than one patient.

Caution - Federal (US) law restricts this device to sale by or on the order of a medical practitioner.

Visual Inspection Prior to Use - Before using a cuff on a patient, perform a visual inspection to determine if the cuff has reached end-of-life. To determine if the product has reached its end of life, visually inspect the cuff for the following damage or deterioration conditions: hook and loop not fastening, cuff leaks, and damaged or frayed material. Whenever visual inspection reveals a cuff is no longer suitable for continued use or does not perform as intended, dispose of the cuff. Refer to *Disposal* below. **Applying the Noninvasive Blood Pressure Cuff** - Refer to images on the front cover. Always inspect the device to ensure it is suitable for use. 1- Select the proper cuff size for the patient's limb circumference. Improper cuff size selection may lead to inaccurate readings. See *Select Appropriate Cuff Size* below. 2- Apply cuff to limb such that the side displaying the **Caution** symbol  is toward the patient and the arrow labeled "Arteria" is over the limb artery. 3- Wrap the cuff snugly around the patient's limb. Ensure that the index edge falls within the range indicated by the range arrow and arteria arrow remains correctly located over the limb artery. 4- Secure the connector on the cuff to the air hose connector to prevent leaking. To select the proper hose for use with the noninvasive blood pressure cuff, refer to *Air Hoses for NBP Cuffs Instructions for Use*.

Select Appropriate Cuff Size - Philips noninvasive blood pressure (NBP) cuffs are available to suit a variety of patient applications and are color-coded for size selection.

Cuff Size (Color)	Limb Circumference (cm)				
	Single Care	Gentle Care	Multi Care	Easy Care	Comfort Care
Infant (Orange)		10.0-15.0	10.0-15.0	10.0-15.0	10.0-15.0
Pediatric (Green)	14.0-21.5	14.0-21.5	14.0-21.5	14.0-21.5	14.0-21.5
Small Adult (Blue)	20.5-28.5	20.5-28.5	20.5-28.0	20.5-28.5	20.5-28.0
Adult (Navy)	27.5-36.0	27.5-36.0	27.0-35.0	27.5-36.0	27.0-35.0
Adult Long ¹ (Navy)	27.5-36.0	27.5-36.0	27.0-35.0	27.5-36.0	27.0-35.0
Large Adult (Burgundy)	35.0-45.0	35.0-45.0	34.0-43.0	35.0-45.0	34.0-43.0
Large Adult Long ¹ (Burgundy)		35.0-45.0		35.0-45.0	
Thigh (Grey)		44.0-56.0	42.0-54.0	44.0-56.0	42.0-54.0

¹ Edge striping denotes a Long cuff. Extra-Long (XL or Long) cuffs are for use only when the standard cuff size cannot be used on a patient due to fit.

Caution: With the use of Extra-Long cuffs outside the recommended range, there may be limitations to the accuracy of the measurement, as closure of any cuff on a circumference larger than the cuff's recommended range can result in incorrectly high readings.

Cleaning and Disinfection - Refer to *NBP Cuffs Care and Air Hoses Addendum - Care, Cleaning, and Disinfection* for a list of validated cleaners and disinfectants and the appropriate procedures. **Disposal** - Dispose of any cuff that becomes damaged or deteriorated. Follow approved disposal methods for medical waste as specified by your facility (hospital, clinic) or local regulations. **Incident Reporting** - Any serious incident that has occurred in relation to this device should be reported to Philips and the competent authority of the European Economic Area (EEA) countries, including Switzerland and Turkey in which the user and/or patient is established. **Environmental Specifications** - **Storage:** Temperature -20 °C to +60 °C (-4 °F to +140 °F) at Relative Humidity of 0 to 90 % at 60 °C (140 °F); **Operating - Comfort Care, Single Care, Multi Care, Easy Care:** 0 °C to +50 °C (32 °F to 122 °F) at 90 % Relative Humidity; **Operating - Gentle Care:** 0 °C to +40 °C (32 °F and 104 °F) at 90 % Relative Humidity. **Reordering Information** - The following products are CE marked and available in the European Union. **Easy Care:** M4552B — Easy Care Cuff, Infant; M4553B — Easy Care Cuff, Pediatric; M4554B — Easy Care Cuff, Small Adult; M4555B — Easy Care Cuff, Adult; M4556B — Easy Care Cuff, Adult XL; M4557B — Easy Care Cuff, Large Adult; M4558B — Easy Care Cuff, Large Adult XL; M4559B — Easy Care Cuff, Thigh; **Gentle Care:** M4572B — Gentle Care Cuff, Infant; M4573B — Gentle Care Cuff, Pediatric; M4574B — Gentle Care Cuff, Small Adult; M4575B — Gentle Care Cuff, Adult; M4576B — Gentle Care Cuff, Adult XL; M4577B — Gentle Care Cuff, Large Adult; M4578B — Gentle Care Cuff, Large Adult XL; **Comfort Care:** M1571A — Comfort Care Cuff, Infant; M1572A — Comfort Care Cuff, Pediatric; M1573A — Comfort Care Cuff, Small Adult; M1574A — Comfort Care Cuff, Adult; M1575A — Comfort Care Cuff, Large Adult; M1574XL — Comfort Care Cuff, Adult XL; M1576A — Comfort Care Cuff, Thigh.

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Taille de brassard (couleur)	Circonférence du membre (cm)				
	Single Care	Gentle Care	Multi Care	Easy Care	Comfort Care
Nourrisson (orange)		10,0–15,0	10,0–15,0	10,0–15,0	10,0–15,0
Enfant (vert)	14,0–21,5	14,0–21,5	14,0–21,5	14,0–21,5	14,0–21,5
Adulte petite taille (bleu)	20,5–28,5	20,5–28,5	20,5–28,0	20,5–28,5	20,5–28,0
Adulte (bleu marine)	27,5–36,0	27,5–36,0	27,0–35,0	27,5–36,0	27,0–35,0
Adulte, long ¹ (bleu marine)	27,5–36,0	27,5–36,0	27,0–35,0	27,5–36,0	27,0–35,0
Adulte grande taille (bordeaux)	35,0–45,0	35,0–45,0	34,0–43,0	35,0–45,0	34,0–43,0
Adulte grande taille, long ¹ (bordeaux)		35,0–45,0		35,0–45,0	
Cuisse (gris)		44,0–56,0	42,0–54,0	44,0–56,0	42,0–54,0

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Veličina manžete (boja)	Opseg uda (cm)				
	Single Care	Gentle Care	Multi Care	Easy Care	Comfort Care
Dojenčad (narančasta)		10,0-15,0	10,0-15,0	10,0-15,0	10,0-15,0
Djeca (zeleni)	14,0-21,5	14,0-21,5	14,0-21,5	14,0-21,5	14,0-21,5
Mala za odrasle (plavo)	20,5-28,5	20,5-28,5	20,5-28,0	20,5-28,5	20,5-28,0
Odrasli (tamno plava)	27,5-36,0	27,5-36,0	27,0-35,0	27,5-36,0	27,0-35,0
Dugačka za odrasle ¹ (tamno plava)	27,5-36,0	27,5-36,0	27,0-35,0	27,5-36,0	27,0-35,0
Velika za odrasle (tamno crvena)	35,0-45,0	35,0-45,0	34,0-43,0	35,0-45,0	34,0-43,0
Velika dugačka za odrasle ¹ (tamno crvena)		35,0-45,0		35,0-45,0	
Bedro (siva)		44,0 – 56,0	42,0 – 54,0	44,0 – 56,0	42,0 – 54,0

¹ Crta na rubu označuje dugačku manžetu. XL ili dugačke manžete namijenjene su za uporabu samo kad se standardna veličina manžeta ne može upotrijebiti na pacijentu.

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Velikost manšete (barva)	Obseg okončine (cm)				
	Single Care	Gentle Care	Multi Care	Easy Care	Comfort Care
Dojenčki (oranžna)		10,0–15,0	10,0–15,0	10,0–15,0	10,0–15,0
Otroci (zeleni)	14,0–21,5	14,0–21,5	14,0–21,5	14,0–21,5	14,0–21,5
Majhni odrasli (modra)	20,5–28,5	20,5–28,5	20,5–28,0	20,5–28,5	20,5–28,0
Odrasli (temno modra)	27,5–36,0	27,5–36,0	27,0–35,0	27,5–36,0	27,0–35,0
Odrasli, dolga ¹ (temno modra)	27,5–36,0	27,5–36,0	27,0–35,0	27,5–36,0	27,0–35,0
Veliki odrasli (temno rdeča)	35,0–45,0	35,0–45,0	34,0–43,0	35,0–45,0	34,0–43,0
Veliki odrasli, dolga ¹ (temno rdeča)		35,0–45,0		35,0–45,0	
Stegno (siva)		44,0–56,0	42,0–54,0	44,0–56,0	42,0–54,0

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Veličina manžetne (boja)	Obim ruke (cm)				
	Single Care	Gentle Care	Multi Care	Easy Care	Comfort Care
Novorođenčad (narandžasta)		10,0–15,0	10,0–15,0	10,0–15,0	10,0–15,0
Deca (zelena)	14,0–21,5	14,0–21,5	14,0–21,5	14,0–21,5	14,0–21,5
Odrasli, mala (plava)	20,5–28,5	20,5–28,5	20,5–28,0	20,5–28,5	20,5–28,0
Odrasli (mornarsko plava)	27,5–36,0	27,5–36,0	27,0–35,0	27,5–36,0	27,0–35,0
Odrasli, dugačka ¹ (mornarsko plava)	27,5–36,0	27,5–36,0	27,0–35,0	27,5–36,0	27,0–35,0
Odrasli, velika (burgundsko crvena)	35,0–45,0	35,0–45,0	34,0–43,0	35,0–45,0	34,0–43,0
Odrasli, velika, dugačka ¹ (burgundsko crvena)		35,0–45,0		35,0–45,0	
Butina (siva)		44,0–56,0	42,0–54,0	44,0–56,0	42,0–54,0

¹ Dugačke manžetne označene su trakom duž ivice. Veoma dugačke (XL ili dugačke) manžetne namenjene su za upotrebu isključivo kada na pacijentu nije moguće koristiti manžetne standardne veličine jer mu ne odgovaraju.

Opaz: Preciznost merenja može biti ograničena uz upotrebu veoma dugačkih manžetni izvan preporučenog opsega jer manžetna koja pokušava da obuhvati obim veći od preporučenog može da dovede do očitavanja lažno visokih vrednosti.

Čišćenje i dezinfekcija – Listu odobrenih sredstava za čišćenje i dezinfekciju, kao i odgovarajuće procedure potražite u dokumentu *NBP Cuffs Care and Air Hoses Addendum - Care, Cleaning, and Disinfection* (Dopuna za manžetne za neinvazivno merenje krvnog pritiska i creva za vazduh – Nega, čišćenje i dezinfekcija). **Odlaganje na otpad** – Bacite sve oštećene ili pohabane manžetne. Pratite odobrene metode za odlaganje medicinskog otpada u vašoj ustanovi (bolnici, klinici) ili lokalne propise. **Prijavlivanje problema** – Svaki ozbiljan problem do kog je došlo sa ovim uređajem treba da se prijavi kompaniji Philips i nadležnom državnim organu u zemljama unutar Evropskog ekonomskog prostora (EEP), uključujući Švajcarsku i Tursku, u kojima se nalazi korisnik ili pacijent. Specifikacije u vezi sa okruženjem – **Skladištenje**: Temperatura: od -20 °C do +60 °C pri relativnoj vlažnosti vazduha od 0 do 90% na 60 °C; **Rad** – **Comfort Care, Single Care, Multi Care, Easy Care**: od 0 °C do +50 °C pri relativnoj vlažnosti vazduha od 90%; **Rad** – **Gentle Care**: od 0 °C do +40 °C pri relativnoj vlažnosti vazduha od 90%. **Informacije o ponovnom naručivanju** – Sledeći proizvodi nose CE oznaku i dostupni su u Evropskoj uniji. **Easy Care**: M4552B – Easy Care manžetna za novorođenčad; M4553B – Easy Care manžetna za pedijatrijske pacijente; M4554B – Easy Care manžetna za sitne odrasle; M4555B – Easy Care manžetna za odrasle M4556B – Easy Care manžetna za odrasle, XL veličine; M4557B – Easy Care manžetna za krupne odrasle; M4558B – Easy Care manžetna za krupne odrasle, XL veličine; M4559B – Easy Care manžetna za butinu; **Gentle Care**: M4572B – Gentle Care manžetna za novorođenčad; M4573B – Gentle Care manžetna za pedijatrijske pacijente; M4574B – Gentle Care manžetna za sitne odrasle; M4575B – Gentle Care manžetna za odrasle; M4576B – Gentle Care manžetna za odrasle, XL veličine; M4577B – Gentle Care manžetna za krupne odrasle; M4578B – Gentle Care manžetna za krupne odrasle, XL veličine; **Comfort Care**: M1571A – Comfort Care manžetna za novorođenčad; M1572A – Comfort Care manžetna za pedijatrijske pacijente; M1573A – Comfort Care manžetna za sitne odrasle; M1574A – Comfort Care manžetna za odrasle; M1575A – Comfort Care manžetna za krupne odrasle; M1574XL – Comfort Care manžetna za krupne odrasle, XL veličine; M1576A – Comfort Care manžetna za butinu.

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Manschettstorlek (färg)	Extremitetens omkrets (cm)				
	Single Care	Gentle Care	Multi Care	Easy Care	Comfort Care
Spädbarn (orange)		10,0–15,0	10,0–15,0	10,0–15,0	10,0–15,0
Barn (grön)	14,0–21,5	14,0–21,5	14,0–21,5	14,0–21,5	14,0–21,5
Liten vuxen (blå)	20,5–28,5	20,5–28,5	20,5–28,0	20,5–28,5	20,5–28,0
Vuxen (marinblå)	27,5–36,0	27,5–36,0	27,0–35,0	27,5–36,0	27,0–35,0
Vuxen, lång ¹ (marinblå)	27,5–36,0	27,5–36,0	27,0–35,0	27,5–36,0	27,0–35,0
Stor vuxen (vinröd)	35,0–45,0	35,0–45,0	34,0–43,0	35,0–45,0	34,0–43,0
Stor vuxen, lång ¹ (vinröd)		35,0–45,0		35,0–45,0	
Lär (grå)		44,0–56,0	42,0–54,0	44,0–56,0	42,0–54,0

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Copyright © 2021
Nederland B.V.