Medtronic

Microstream™ Luer

Adult Oral-Nasal CO₂ Sampling Line

Pediatric Oral-Nasal CO₂ Sampling Line

Adult Nasal CO₂ Sampling Line


Pediatric Nasal CO₂ Sampling Line

en	Instructions for Use	REF MLA
fr	Mode d'emploi	REF MLAO
de	Gebrauchsanweisung	REF MLA100U
nl	Gebruiksaanwijzing	REF MLAO100U
it	Istruzioni per l'uso	REF MLPO
es	Instrucciones de uso	REF MLANO
pt	Instruções de uso	REF MLPNO
ru	Инструкция по эксплуатации	
da	Brugsanvisning	
sv	Bruksanvisning	
el	Οδηγίες χρήσης	
no	Bruksanvisning	
hu	Használati útmutató	
pl	Instrukcja użycia	
cs	Návod k použití	
fi	Käyttöohjeet	
tr	Kullanma Talimatları	
sk	Pokyny na používanie	
sl	Navodila za uporabo	
ro	Instrucțiuni de utilizare	
bg	Инструкции за употреба	
et	Kasutusjuhised	
lt	Naudojimo instrukcijos	
lv	Lietošanas norādījumi	
hr	Upute za uporabu	
sr	Uputstva za upotrebu	

Table of Contents

English	6
French	7
German	8
Dutch	9
Italian	10
Spanish	11
Portuguese	12
Russian.....	12
Danish.....	13
Swedish	14
Greek.....	15
Norwegian	16
Hungarian	16
Polish.....	17
Czech.....	18
Finnish	19
Turkish	20
Slovak	20
Slovenian.....	21
Romanian.....	22
Bulgarian	23
Estonian.....	24
Lithuanian.....	25
Latvian	26
Croatian.....	27
Serbian.....	28

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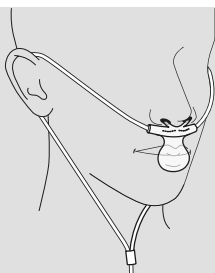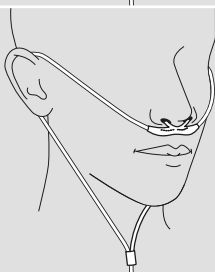
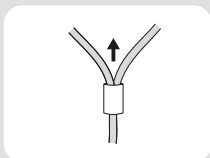
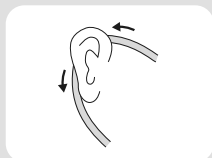
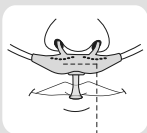
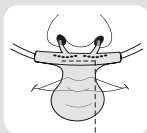
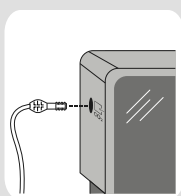
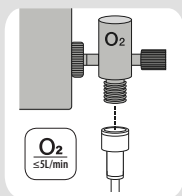
Microstream™ Luer

Adult Oral-Nasal CO₂ Sampling Line

Pediatric Oral-Nasal CO₂ Sampling Line

Adult Nasal CO₂ Sampling Line

Pediatric Nasal CO₂ Sampling Line

Indications for Use:

Used whenever the physician needs to collect a sample of the patient's breathing to measure CO₂ with a capnograph while simultaneously administering supplemental oxygen near the nose and mouth for inhalation.

Microstream Luer products covered in this DFU are intended to be used with capnography monitors with the following requirements:

- Have a standard luer connector that meets the requirements of ISO 594-2
- Comply with capnography standard (80601-2-55)
- Operate at a sampling flow rate of 90-180 ml/min
- Can withstand a sampling line pressure drop of up to 70mBar @ 180ml/min.

The product is intended for single patient use only.

WARNINGS

- **Loose or damaged connections may compromise ventilation or cause an inaccurate measurement of respiratory gases. Securely connect all components, turning the sampling line connector clockwise into the monitor CO₂ port until it can no longer be turned. Check connections for leaks according to standard clinical procedures, and ensure CO₂ values appear.**
- **Ensure no kinks are present in CO₂ and O₂ tubing, as kinked tubing may cause inaccurate CO₂ sampling or affect O₂ delivery.**
- **Carefully route the sampling line to reduce the possibility of patient entanglement or strangulation.**
- **For MLPO and MLPNO only: This product can expose you to chemicals, including DINP, which are known to the State of California to cause cancer. For more information, go to www.P65Warnings.ca.gov.**

CAUTIONS

- Do not attempt to clean, disinfect, sterilize or flush any part of the sampling line as this can cause damage to the monitor.
- Sampling lines are recommended for use with oxygen provided at up to 5l/min. At higher levels of oxygen provision, dilution of CO₂ readings may occur, leading to lower CO₂ values.
- Dispose of sampling lines according to standard operating procedures or local regulations for the disposal of contaminated medical waste.

Notes

- The response time of the standard length sampling line is ≤3.5 seconds, 4m sampling lines ≤5.9 seconds.
- Sampling lines with H in their names include a moisture reduction component (Nafion® or its equivalent) for use in higher humidity environments where long duration use of CO₂ sampling is required.
- Replace the sampling line according to hospital protocol or when a blockage is indicated by the device. Excessive patient secretions or a build-up of liquids in the airway tubing may occlude the sampling line, requiring more frequent replacement.

TM* Trademark of its respective owner.

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Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic.
Other brands are trademarks of a Medtronic company.
© 2019 Medtronic. All rights reserved.
Oridion Medical 1987 Ltd., 7 Hamarpe St.,