Microstream™ Advance


CO₂ Filter Line MRI XL

en	Instructions for Use
fr	Mode d'emploi
de	Gebrauchsanweisung
nl	Gebruiksaanwijzing
it	Istruzioni per l'uso
es	Instrucciones de uso
pt	Instruções de uso
ru	Инструкция по эксплуатации
da	Brugsanvisning
sv	Bruksanvisning
el	Οδηγίες χρήσης
no	Bruksanvisning
hu	Használati útmutató
pl	Instrukcja użycia
cs	Návod k použití
fi	Käyttöohjeet
tr	Kullanma Talimatları
sk	Pokyny na používanie
sl	Navodila za uporabo
ro	Instrucţiuni de utilizare
bg	Инструкции за употреба
et	Kasutusjuhised
lt	Naudojimo instrukcijos
lv	Lietošanas norādījumi
hr	Upute za uporabu
sr	Uputstva za upotrebu


Table of Contents

English	6
French	6
German	7
Dutch	7
Italian	8
Spanish	8
Portuguese	9
Russian.....	9
Danish.....	10
Swedish	10
Greek.....	11
Norwegian	11
Hungarian	12
Polish.....	12
Czech.....	13
Finnish	13
Turkish	14
Slovak	14
Slovenian.....	15
Romanian.....	15
Bulgarian	16
Estonian.....	16
Lithuanian.....	17
Latvian.....	17
Croatian.....	18
Serbian.....	18

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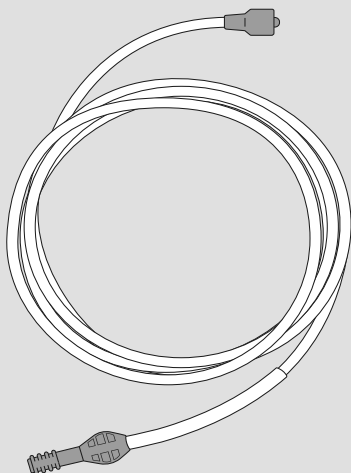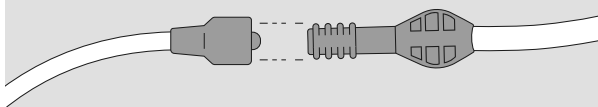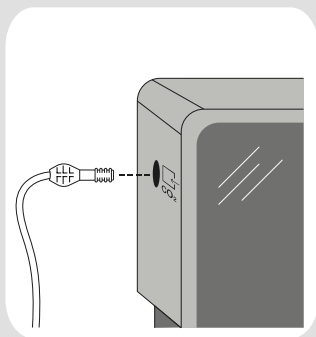
CO₂ Filter Line MRI XL

Indications for Use:

Intended to be used when longer CO₂ tubing is required, to allow CO₂ sampling for environments such as an MRI suite.

For use with all Microstream™-enabled monitors

The product is intended for single patient use only.

The product is intended for use with adult, pediatric and infant/neonatal patients.

WARNINGS

- **Loose or damaged connections may compromise ventilation or cause an inaccurate measurement of respiratory gases. Securely connect all components, turning the sampling line connector clockwise into the monitor CO₂ port until it can no longer be turned. Check connections for leaks according to standard clinical procedures, and ensure CO₂ values appear.**
- **Do not attempt to clean, disinfect, sterilize or flush any part of the sampling line. Re-use of single-use accessories could pose a cross-contamination risk to the patient or damage the functioning of the monitor.**

Notes

- During nebulization, lavage, or suction, follow monitor instructions for use (IFU) to suspend CO₂ pump activity, in order to avoid moisture buildup and sampling line occlusion. In addition, especially if unable to suspend the CO₂ pump, disconnect the sampling line connector from the CO₂ port on the monitor.
- A longer delay time, slower response time and decreased frequency response will occur when using the Filter Line MRI XL, due to its extended length.
- Any serious incident related to device use that may occur should be reported immediately to the manufacturer, the local competent authority, and any other regulators as required.

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Together are trademarks of Medtronic.
Other brands are trademarks of a
© 2019 Medtronic. All rights reserved.
Oridion Medical 1987 Ltd., 7 Hamarpe St.,