
Nellcor™

Sensor Wraps

en us	Instructions For Use
fr	Mode d'emploi
de	Gebrauchsanweisung
it	Istruzioni per l'uso
es	Instrucciones de uso

en us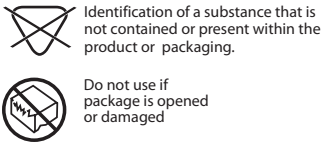

This product cannot be adequately cleaned and/or sterilized by the user in order to facilitate safe reuse, and is therefore intended for single use. Attempts to clean or sterilize these devices may result in a bio-incompatibility, infection or product failure risks to the patient.

Description

The **Nellcor™** Posey® Sensor Wrap is intended for use with the following **Nellcor** reusable oxygen transducers (sensors) when short-term or spot-check monitoring of arterial oxygen saturation and pulse rate is required: the D-YS, the OXI-A/N, and the OXI-P/I. The **Nellcor** Sensor Wrap attaches the sensor to the patient and maintains positioning of the sensor's optical components.


Instructions for Use

Wrap Size:

- 1) The **Nellcor** Sensor Wrap has two pairs of evenly-spaced holes, as shown in Figure **A**. The right pair of holes is intended for adult and neonatal patients. The left pair of holes is intended for infant and pediatric patients.
- 2) When applying a sensor to an adult or neonatal patient, use the right pair of holes. If desired, cut along the straight perforated lines to make the sensor wrap narrower.
- 3) When applying a sensor to an infant or pediatric patient, cut along the curved and the straight perforated lines to make the sensor wrap both smaller and narrower.

D-YS Sensor Use:

- 1) Hold the **Nellcor** Sensor Wrap with the blue side up and the closure tab on the left. Secure the sensor to the wrap by pushing the "button" on the black sensor pad through the right hole of the right pair of holes in the wrap, as shown in Figure **B**.
- 2) Push the "button" on the white sensor pad through the left hole of the right pair of holes in the wrap, as shown in Figure **C**.
- 3) Apply the sensor to the patient such that the optical components are directly opposite each other on the sensor site, and the blue side of the wrap is against the patient's skin, as shown in Figure **D**.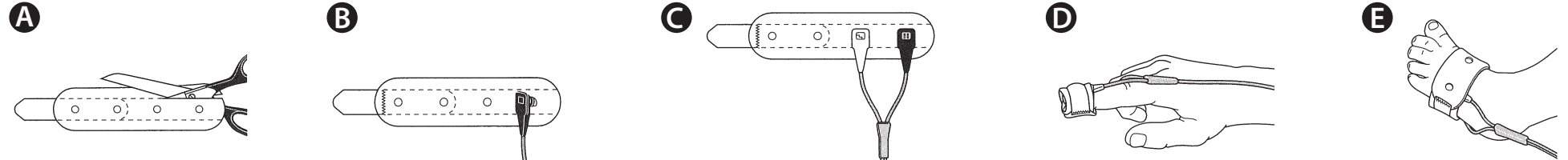

- 4) Secure the sensor in place by pressing the closure tab against the white fabric side of the wrap. The tightness of the wrap should be sufficient to keep the sensor in place, but not enough to constrict blood flow to the site or cause venous pulsations.

OXI-A/N and OXI-P/I Sensor Use:

- 1) Position the sensor on the patient application site so that the optical components are directly opposite each other. Then wrap the **Nellcor** Sensor Wrap around the sensor such that the holes of the wrap engage the alignment bumps on the sensor, and the blue side of the wrap is against the patient's skin. See Figure **E**.
- 2) Secure the sensor in place by pressing the closure tab against the white fabric side of the wrap. The tightness of the wrap should be sufficient to keep the sensor in place, but not enough to constrict blood flow to the site or cause venous pulsations.

Refer to the sensor **Directions for Use** for additional instructions regarding selection of a sensor site and proper attachment of a **Nellcor** sensor to the patient.

Removal


Remove the **Nellcor** Sensor Wrap from the patient by gently pulling the closure tab away from the white fabric side of the wrap.

- Remove the **D-YS** sensor from the wrap by gently pulling the alignment buttons out of the holes in the wrap.
- Remove an **OXI-A/N** and **OXI-P/I** sensor from the wrap by gently pulling the wrap away from the sensor.

The **Nellcor** Sensor Wrap is intended for single-patient use. Discard after use.

Warnings

- 1) Failure to apply the sensor according to the sensor **Directions for Use** may lead to incorrect measurements.
- 2) Do not wrap the sensor too tightly. Venous pulsations may lead to inaccurate saturation measurements.
- 3) Inspect the sensor site every 4 hours to ensure skin integrity. If skin integrity changes, move the sensor to another site.

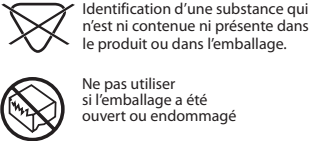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© 2011 Covidien.
COVIDIEN, COVIDIEN with logo and Covidien logo are U.S. and internationally registered trademarks of Covidien AG. Posey® is a registered trademark of J.T. Posey Company. Other brands are trademarks of a Covidien company.
Covidien LLC, 15 Hampshire Street, Mansfield, MA 02048 USA.
Covidien Ireland Limited, IDA Business & Technology Park, Tullamore.