iQ[®] Calibrator

TABLE OF CONTENTS

English (EN)	1
Français (FR)	4
Deutsch (DE)	7
Italiano (IT)	10
Español (ES)	13
Português, continental (PT)	16
Português, Brazilian (PTBR)	19
Dansk (DA)	22
Svenska (SV)	25
Norsk (NO)	28
Ελληνικά (EL)	31
中文 (ZH)	34
Lietuviškai (LT)	37
Magyar (HU)	40
Polski (PL)	43
Čeština (CS)	46
Slovenčina (SK)	49
Türkçe (TR)	52
Български (BG)	55
Eesti keel (ET)	58
Latviešu (LV)	61
Română (RO)	64
Русский (RU)	67

REF: 800-3103

iQ[®] Calibrator is intended for *in vitro* diagnostic use with iQ[®]200 Series

REF: 800-3103

INTENDED USE: iQ[®] Calibrator is intended for *in vitro* diagnostic use with iQ[®]200 Series

INTENDED USER: Clinical Laboratorian

CONTENTS: iQ Calibrator Pack (REF 800-3103): 4 X 125 mL bottles iQ Calibrator (REF 475-0059)

PRINCIPLES AND CHEMICAL COMPOSITION

iQ Calibrator (REF 475-0059) is a suspension of fixed human red blood cells in a particulate-free solution. It is used to automatically calibrate the iQ200 Series Automated Urine Microscopy Analyzer. An accurate count is certified and the expected value (accurate to approximately 8.3%) is printed on each label. Barcodes are provided that contain Lot number, Expiration date and Expected value

iQ CALIBRATOR NOMINAL ASSAY VALUES AND RANGES

CONTROL	NOMINAL ASSAY VALUE
iQ CALIBRATOR	1200 (MEAN; particles/μL)

Important Note: The nominal assay values for the iQ Calibrator may change from LOT number to LOT number

COMPOSITION OF REAGENTS

iQ Calibrator: Human erythrocytes, water, salts, dyes and preservatives. The concentration of stabilized human erythrocytes is <1% in an aqueous suspension. Non-reactive for HbsAg, HCV 3.0 Ab, HIV 1 / 2, HIV-1 p24 Ag, RPR and ALT <82 IU/mL. The stabilized human erythrocytes are chemically fixed, which thus minimizes any biohazardous risks

GHS HAZARD CLASSIFICATION

iQ[®] Calibrator

WARNING

H317

May cause an allergic skin reaction.

H361

Suspected of damaging fertility or the unborn child.

P201

Obtain special instructions before use.

P280

Wear protective gloves, protective clothing and eye/ face protection.

P308+P313

IF exposed or concerned: Get medical advice/ attention.

P333+P313

If skin irritation or rash occurs: Get medical advice/ attention.

P362+P364

Take off contaminated clothing and wash it before use. N-(3-Chloroallyl) hexaminium Chloride 0.1 - 0.5%

SDS Safety Data Sheet is available at techdocs.beckmancoulter.com.

STORAGE / HANDLING CONDITIONS:

- Store at 2 ° C to 8 ° C (35.6 ° F to 46.4 ° F)
- Open expiration: 24 hours
- Do not freeze
- Do not store in direct sunlight
- Each bottle is expected to yield single calibration and should not be used again
- Do not mix the calibrator from different bottles
- Bring to room temperature before use
- Glass or validated plastic tubes may be used when running the iQ Calibrator
- iQ Calibrator is stable for eight months from date of manufacture
- Any unopened solution in its original packaging may be used until the expiration date
- Do not use beyond the date of use after opening as stated on the bottle

TRAINING PREREQUISITE: iQ200 Service Engineer Training or iQ200 Product Training

PROCEDURE:**1. MATERIAL PROVIDED**

- 4 X 125 mL bottles of iQ Calibrator
- One set of barcode labels

2. MATERIAL REQUIRED BUT NOT PROVIDED

- Glass or validated plastic tubes 10 mL (diameter 16mm / height 100mm) for processing on the automated analyzer
- iQ200 Series Automated Urine Microscopy Analyzer

3. METHODOLOGY

Shake the bottle as directed in the iQ200 Series operators manual before use and follow the directions for calibration. The included barcodes provide the lot number, expiration date, and expected value to the instrument. The passing results are displayed on screen as soon as they are available

LIMITATIONS OF THE PROCEDURE iQ CALIBRATOR

- **Expected Values:** The expected value is 1200 ± 100 cells/ μ L (see label on the bottle)
- **Limitations:** No interferences are reported

PRECAUTIONS

- The iQ Calibrator has been determined to be nonhazardous under the guidelines issued by OSHA in 29 CFR 1910.1200(d). Appropriate precautions in the collection, handling, storage and disposal of specimens should be exercised accordingly
- Wear all Personal Protective Equipment when working with the iQ200 Automated Urine Microscopy Analyzer (Gloves, Lab Coat, Goggles)
- Personal Precautions: Avoid skin and eye contact. Use good laboratory procedures
- Spill and Leak Procedures: No special hazards determined
- Environmental Precautions: Absorb into inert solid or dilute with a large quantity of water and flush into an wastewater treatment system. Comply with all applicable legal regulations

TECHNICAL SUPPORT

- If you have any questions regarding the use of this product, please contact Iris Diagnostics Clinical Support at 1-818-527-7000, or www.beckmancoulter.com. If you are located outside the US, please contact your local distributor
- If you need additional copies of this document, you can obtain it free of charge:

1. From the Iris Diagnostics Web-site:

techdocs.beckmancoulter.com

2. By calling up the following phone number:

1-818-527-7000, 1-800-776-4747 in order to have the package insert faxed, mailed or e-mailed

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© 2019 Iris Diagnostics - a Division of IRIS International, Inc. All rights reserved.
Iris Diagnostics - a Division of IRIS International, Inc.
mdi Europa GmbH