Iris Diluent™

TABLE OF CONTENTS

English (EN)	1
Français (FR)	4
Deutsch (DE)	7
Italiano (IT)	10
Español (ES)	13
Português, continental (PT)	16
Português, Brazilian (PTBR)	19
Dansk (DA)	22
Svenska (SV)	25
Norsk (NO)	28
Ελληνικά (EL)	31
中文 (ZH)	34
Lietuviškai (LT)	37
Magyar (HU)	40
Polski (PL)	43
Čeština (CS)	46
Slovenčina (SK)	49
Türkçe (TR)	52
Български (BG)	55
Eesti keel (ET)	58
Latviešu (LV)	61
Română (RO)	64
Русский (RU)	67

PACKAGE INSERT

REF: 800-3202

Iris Diluent™ is intended for *in vitro* diagnostic use with iQ®200 Series and Iris Diagnostic Instruments

REF: 800-3202

INTENDED USE: Iris Diluent™ is intended for *in vitro* diagnostic use with iQ®200 Series and Iris Diagnostic Instruments

INTENDED USER: Clinical Laboratorian

CONTENTS: Iris Diluent Pack (REF 800-3202): 4 X 475 mL bottle of Iris Diluent (REF 475-0021)

PRINCIPLES AND CHEMICAL COMPOSITION

Iris Diluent is a particle free aqueous salt solution which is used for diluting urine and body fluid samples. It is also used to rinse the system to prevent carryover of system cleanser and samples

Composition: 2-Phenoxyethanol 1% in a salt solution

SDS Safety Data Sheet is available at techdocs.beckmancoulter.com

STORAGE / HANDLING CONDITIONS:

- Store at 20°C to 28°C (68°F to 82.4°F)
- Do not freeze
- Do not store in direct sunlight
- Unused solutions that remain in the original capped container are stable until the expiration date. Do not use after the expiration date printed on the bottle
- Iris Diluent is stable for twelve months from date of manufacture
- Do not use beyond the date of use after opening is stated on the bottle

PROCEDURE:

1. MATERIAL PROVIDED

- 4 X 475 mL containers of Iris Diluent
- 4 – spouts

2. MATERIAL REQUIRED BUT NOT PROVIDED

- Glass or validated plastic tubes 10 mL (diameter 16mm / height 100mm) for running on the instrument

3. METHODOLOGY

- Follow the directions in the iQ200 Series Operators Manual

TRAINING PREREQUISITE: iQ200 Service Engineer
Training or iQ200 Product Training

PRECAUTIONS

- Iris Diluent™ has been determined to be nonhazardous under the guidelines issued by OSHA in 29 CFR 1910. 1200(d). Appropriate precautions in the collection, handling, storage and disposal of specimens should be exercised accordingly. Discard any discolored solutions as they may have deteriorated
- Wear all Personal Protective Equipment when working with the iQ200 Automated Urine Microscopy Analyzer (Gloves, Lab Coat, Goggles)
- Personal Precautions: Avoid skin and eye contact. Use good laboratory procedures
- Spill and Leak Procedures: No special hazards determined
- Environmental Precautions: Absorb into inert solid or dilute with a large quantity of water and flush into a wastewater treatment system. Comply with all applicable legal regulations

TECHNICAL SUPPORT

- If you have any questions regarding the use of this product, please contact Iris Diagnostic Technical Support at +1-818-527-7000, +1-800-776-4747, or www.beckmancoulter.com. If you are located outside the US, please contact your local distributor
- If you need additional copies of this document, you can obtain it free of charge:
 1. **From the Beckman Coulter Web-site:**
<http://www.techdocs.beckmancoulter.com>
 2. **By calling the following phone number:**
+1-818-527-7000, +1-800-776-4747 in order to have the package insert faxed, mailed or e-mailed

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© 2019 Iris Diagnostics - a Division of IRIS International, Inc. All rights reserved.
Iris Diagnostics - a Division of IRIS International, Inc.
mdi Europa GmbH