



# Technical Data

## Mycoprep (for 10 tests)

FD173B

A combination mixture for decontamination and liquefaction of specimen. N-Acetyl L-cysteine (NALC) is a mucolytic agent that can rapidly digest sputum when used at a concentration of 0.5 to 2.0%. Decontamination is achieved by the addition of sodium hydroxide, while sodium citrate gives stabilizing effect on the NALC by chelating heavy metal ions present in the specimen and which may inactivate the NALC. The phosphate buffer minimizes the continuing action of NaOH and also allows the efficient sedimentation of mycobacteria by centrifugation, by lowering the specific gravity of the specimen.

### Composition

Per kit sufficient for 10 tests

#### Ingredients

Kit contents (5 bottles of following)

R032: Sodium citrate - hydroxide buffer

R033 : Phosphate buffer

R034 : NALC (N-acetyl-L-cysteine)

#### Concentration

20ml

100ml

0.100g

#### Formula:

##### R032 : (per vial, 20ml)

Trisodium citrate

0.290g

Sodium hydroxide

0.400g

Distilled water

20ml

##### R034 : (per vial 0.1gm)

N-Acetyl-L-cysteine

0.100g

##### R033 : (per vial, 100 ml)

Disodium hydrogen phosphate

0.470g

Potassium dihydrogen phosphate

0.470g

Distilled water

100ml

### Directions:

Aseptically mix 0.1 gm of N-Acetyl-L-cysteine (R034) in 20 ml of Sodium citrate-hydroxide buffer (R032). Use immediately and within 24 hours only. Transfer a maximum volume of 10 ml of specimen to a sterile graduated 50 ml plastic centrifuge tube having a leak proof cap. Add an equal volume of the above solution. Mix the contents thoroughly by inverting with the cap tightened. Mix on vortex mixer for approximately 20s until the contents are liquefied. Allow the mixture to stand for 15 minutes at 20 to 25°C with occasional gentle shaking by hand. Do not overprocess as this will reduced the recovery of mycobacteria.

Add phosphate buffer (R033) up to the 50 ml mark on the tube. Recap the tube and swirl it by hand to mix the contents well. Centrifuge the solution for at least 15 minutes at  $\geq 3000$  xg. Preferably use a refrigerated centrifuge. Carefully decant the supernatant fluid into a splash-proof discard container containing suitable disinfectant.

Add to the sediment, 1 to 2 ml of phosphate buffer pH 6.8 with a sterile pipette and resuspend the sediment with the pipette or by gently shaking the tube. Inoculate onto solid culture media (M162-L J Medium Base, M199-Middlebrook 7H10 Agar etc.) or use for preparing a smear for staining.

Remarks : Good laboratory practices and hazard precaution must be observed.

### Storage and Shelf Life

Store at 2-8°C. Use before the expiry date on the label.

Revision : 1 / 2012

#### Disclaimer :

User must ensure suitability of the product(s) in their application prior to use. Products conform solely to the information contained in this and other related HiMedia™ publications. The information contained in this publication is based on our research and development work and is to the best of our knowledge true and accurate. HiMedia™ Laboratories Pvt Ltd reserves the right to make changes to specifications and information related to the products at any time. Products are not intended for human or animal diagnostic or therapeutic use but for laboratory, research or further manufacturing use only, unless otherwise specified. Statements contained herein should not be considered as a warranty of any kind, expressed or implied, and no liability is accepted for infringement of any patents.