

Periodic acid 1% solution method for PAS

IVD In-vitro diagnostic medical device **CE**

CND Code: W01030799

Catalog number	Unit size
05-M05030	500 ml

Packaging	- 05-M05030 Primary container: white bottle in High Density Polyethylene (HDPE). Useful capacity 500 ml. HDPE cap. Tamper evident cap Wear, water, alcohol and solvents resistant PVC label. Scratchproof ink resistant to water and alcohol.														
Expected aim	Product for the preparation of cyto-histological samples for optical microscopy.														
Application	Reagent used as oxidant in the PAS method.														
Principle	In the PAS method, to show the neutral mucins, the periodic acid oxidizes selectively groups: 1.2 - Glycolic; primary aminic (or 1-hydroxy-2-aminic); secondary aminic (or 1-hydroxy-2-alkylaminic) and 1-hydroxy-2-ketonic. During oxidating process the links between carbon atoms in 1,2 position break and consequently aldehydic groups are formed. In the following reaction, sulphurous fuchsin in Schiff reagent changes these 2 contiguous aldehydic groups into a insoluble stained compound.														
Method	1) Sections to water 2) Periodic acid solution 3) Wash in distilled water 4) Schiff reagent 5) Wash in distilled water 6) Mayer's Hemalum 7) Blue in tap water 8) Dehydrate in alcohol 9) Xylene and mount	10 minutes 20 minutes 3 minutes 5 minutes													
Results	Positive PAS substances.....Red magenta NucleiBlue														
Components	<table border="1"> <thead> <tr> <th>Components</th><th>CAS</th><th>CE</th><th>Index</th></tr> </thead> <tbody> <tr> <td>Periodic acid</td><td>10450-60-9</td><td>2339370</td><td>-</td></tr> <tr> <td>Deionized water</td><td></td><td></td><td></td></tr> </tbody> </table>			Components	CAS	CE	Index	Periodic acid	10450-60-9	2339370	-	Deionized water			
Components	CAS	CE	Index												
Periodic acid	10450-60-9	2339370	-												
Deionized water															
Warning and precaution	The product must be used exclusively by specialized technical operators. Carefully read the information on the classification of dangerous substances on the label. Always refer to the safety data sheet where are available the information on the risks presented by the mixture, the precautionary measures during use, the measures first aid and the intervention in the event of accidental release. Do not use if the primary container is damaged.														
Storage	Store the preparation at 2-8°C. Keep the containers tightly closed.														
Stability	After the first opening, the product is usable until the expiry date, if correctly stored. Product validity: 1 year.														
Disposal	Hazardous preparation: observe all state and local environmental regulations regarding waste disposal.														

Date of issue: July 2015

Producer: Bio-Optica Milano s.p.a.

Page 1 di 1