


REVISIONS		
REV	ECO NUMBER	NAME
A	PR056827	M. GROBMAN

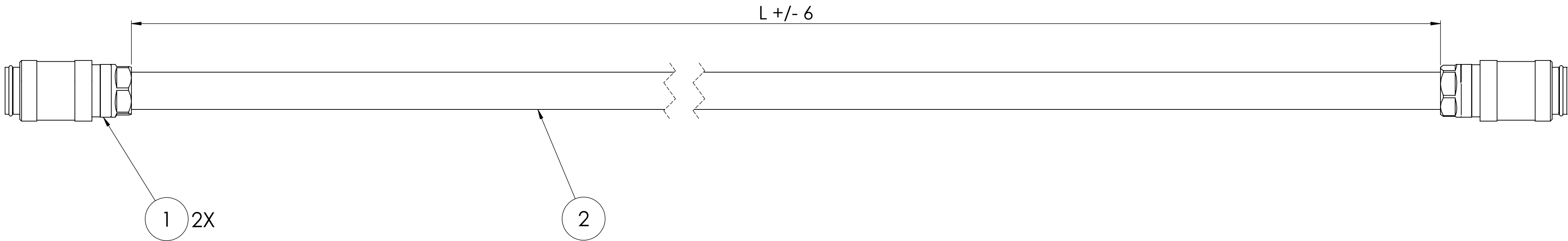

P/N	L (INCH)
712-0767-02	144
712-0900-02	108

ASSEMBLY INSTRUCTIONS:

1. INSERT BARBED END OF FITTINGS INTO THE HOSE. HOSE ENDS FIT FLUSH TO END OF FITTINGS.
2. THE AIR LEAKAGE SHALL NOT EXCEED A PRESSURE DROP OF 4 mmHg AT 300 mmHg FOR 1 MINUTE.
3. NEATLY COIL PRODUCT FOR PACKAGING.

THIS DOCUMENT CONTAINS PROPRIETARY TRADE SECRET AND CONFIDENTIAL INFORMATION WHICH IS THE PROPERTY OF SPACELABS HEALTHCARE. THIS DOCUMENT AND THE INFORMATION IT CONTAINS ARE NOT TO BE COPIED, DISTRIBUTED, OR DISCLOSED TO OTHERS, OR USED IN ANY MANNER OUTSIDE OF SPACELABS HEALTHCARE WITHOUT PRIOR WRITTEN APPROVAL OF SPACELABS HEALTHCARE.		Spacelabs Healthcare	
UNLESS OTHERWISE SPECIFIED DIMENSIONS ARE IN INCHES. TOLERANCES AS SHOWN BELOW		DESCRIPTION DRAWING FOR ADULT NIBP HOSE	
LINEAR X.X ± 0.25 X.XX ± 0.25 X.XXX ± 0.125 X.XXXX ± 0.0625		ANGULAR ± 1 °	
THIRD ANGLE PROJECTION		INTERPRET DIMENSIONS AND TOLERANCES PER ISO OR ASME Y14.5M-1994 DIMENSIONS SHOWN ARE FINISHED SIZES AND SHALL ALLOW FOR PLATING AND COATING, BUT NOT PAINTING GENERAL THREAD FIT 6H 6g	
SIZE A1	PART NUMBER 653-1659-00	REV A	
VERSION:		SHEET 1 OF 1	