MicroScan

MicroScan Tray Lids Procedure Manual

REF B1018-18

TABLE OF CONTENTS

English	2
Français (fr-fr)	4
Deutsche (DE)	6
Italiano (IT)	8
Español (ES)	10
Português Portugal (PT-PT)	12
Norsk (NO)	14
Ελληνικά (GR)	16
中文 (中国) (zh-cn)	18
Lietuvių (LT)	20
Magyar (HU)	22
Polski (PL)	24
Čeština (cs-cz)	26
Slovak (SK)	28
한국어 (KR)	30
Türkçe (TR)	32
Русский (RU)	34
Hrvatski (hr-hr)	37
Български (bg-bg)	39
中文 (台灣) (zh-tw)	41
Română (RO)	43
Srpski (Latinski) (SP)	45
Português Brasil (PT-BR)	47
Nederlands (NL)	49
Tiếng Việt (VN)	51
Македонски (MK)	53

MicroScan

MicroScan Tray Lids Procedure Manual

REF B1018-18

INTENDED USE

MicroScan WalkAway Tray Lids are for use with the MicroScan Panels when used on the WalkAway instrumentation.

INTENDED USER

This product is intended for laboratory professional use.

CLINICAL RELEVANCE

Not applicable, product is an IVD accessory.

SUMMARY AND PRINCIPLE

MicroScan WalkAway Tray Lids are an *in vitro* diagnostic accessory which aids in panel incubation for quantitative and qualitative AST and bacterial identification. Refer to MicroScan panel instructions for methods of inoculation, incubation and results determination which may include manual, semiautomated, or full automation on MicroScan instrumentation.

WARNING AND PRECAUTIONS

1. For *in vitro* diagnostic use only.
2. Observe aseptic techniques and established precautions against microbiological hazards throughout all procedures, with special awareness that the inoculated panels contain potentially pathogenic organisms.
3. Single Use Only. All MicroScan consumables should not be re-used unless otherwise noted (e.g., cover trays).
4. Do not use MicroScan Tray Lids if warped, broken or chipped.

STORAGE

MicroScan Tray Lids should be stored at 1 - 50°C. Do not use beyond the expiration date.

SHELF LIFE/EXPIRATION DATE

Expiration Date: The MicroScan Tray Lids are stable for 5 years after the date of manufacture.

Shelf Life: The MicroScan Tray Lids are stable for 5 years after the date of manufacture when stored properly.

GHS HAZARD CLASSIFICATION

Not classified as hazardous

Safety Data Sheet is available at beckmancoulter.com/techdocs

EVIDENCE OF DETERIORATION

Not applicable.

SPECIMEN COLLECTION AND PREPARATION

Not applicable.

MATERIALS PROVIDED

MicroScan Trays Lids, catalog number B1018-18, contains 240 tray lids per package

MATERIALS REQUIRED, BUT NOT PROVIDED

Applicable MicroScan panel Procedural Manual

COMPOSITION

Polystyrene

This product does not contain materials of animal origin.

PROCEDURE

Refer to the appropriate MicroScan panel Procedural Manual for testing details.

RESULTS

Not applicable.

LIMITATION OF THE PROCEDURE

Not applicable.

QUALITY CONTROL

Not applicable.

PERFORMANCE CHARACTERISTICS

Not applicable.

WARRANTY STATEMENT

The system is covered by and subject to the provisions of the warranty included in your contractual agreement for the system or its reagents. The customer is responsible for routine preventive maintenance procedures. Repairs arising from the failure to perform these maintenance procedures at the indicated time intervals are made at the discretion of Beckman Coulter, and at the customer’s expense.

NOTICE TO USER

Any serious incident that has occurred in relation to MicroScan panels shall be reported to Beckman Coulter and the competent authority of the Member State in which the user and/or the patient is established.

TRADEMARKS

Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.

May be covered by one or more pat. -see www.beckmancoulter.com/patents.

REVISION HISTORY

REVISION	DATE	DESCRIPTION
AA	December 2021	Initial Release
AB	August 2023	Updated: Symbols Key section. Added trademark section, Patent Information Statement.

Symbols Key

Glossary of Symbols for Beckman Coulter Microbiology Products is available at beckmancoulter.com/techdocs (Document Number 3240-3095).

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Beckman Coulter Ireland Inc.
Beckman Coulter, Inc., 250 S. Kraemer Blvd., Brea, CA 92821 U.S.A.