

## Disposable Pouch for Breakout Box (PN 031622 Rev A)

### Intended use:

The disposable pouch is a carrying pouch intended to carry a breakout box unit and associated battery pack for individual patient use while providing visualization of applicable indicator lights and reducing patient contact.

### Instructions for use:

- To be used with existing standard (black modular) pouch when not actively using belt or lanyard carrying option.
- After attaching the electrodes to the patient, place the breakout box into the standard (black modular) pouch with the electrode wires exiting at the top or on the sides, as required. If an external battery pack is used, insert it into the standard (black modular) pouch beside the breakout box.
- Close the zippers as much as possible so that the breakout box is secured in the standard (black modular) pouch.
- Pull apart the disposable drawstring pouch opening to loosen the drawstring and place the entire assembly, from the above steps, into the drawstring disposable pouch with electrode leads, patient event button and additional accessories (e.g. pulse oximeter), if used, exiting at the drawstring opening.
- Secure the drawstring to desired tightness
- Use drawstring loops over patient shoulders to position comfortably

**Caution:** If the breakout box positioned to be on back of patient remove from the patient's back if patient is in sitting position on a chair with a back or laying down.

**Warning:** As with all medical equipment, there is a risk of injury if the lanyard/belt and pouch(es) are used without ensuring they are secured to the patient properly. Refer to the breakout box user manual for details.