
AQT90 FLEX

Empty tube kit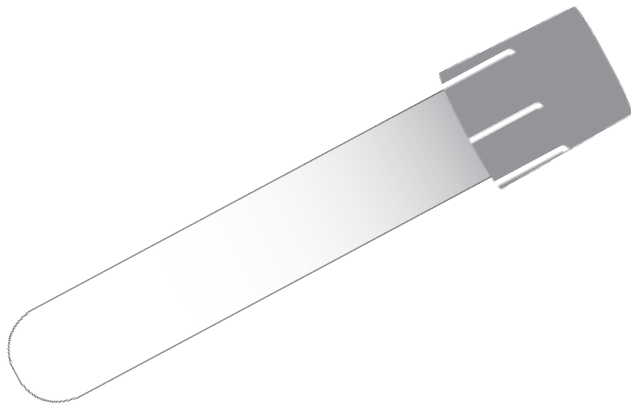

Contents

en	Empty tube kit.....	2
bg	Набор празни епруветки.....	3
cs	Sada prázdných zkumavek.....	4
da	Sæt med tomme rør.....	5
de	Leerröhrchen-Kit.....	6
el	ΚΙΤ κενών σωληναρίων.....	7
es	Kit de tubos vacíos.....	8
et	Tühi katsutikomplekt.....	9
fi	Pakkaus tyhjiä putkia.....	10
fr	Kit de tubes vides.....	11
hr	Komplet praznih epruveta.....	12
hu	Üres csőkészlet.....	13
it	Kit provette vuote.....	14
ja	空のチューブキット.....	15
lt	Tuščių mėgintuvėlių rinkinys.....	16
lv	Tukšo stobriņu komplekts.....	17
nl	Lege buizenkit.....	18
no	Sett med tomme rør.....	19
pl	Zestaw pustych probówek.....	20
pt	Kit de tubos vazios.....	21
ro	Kit de tuburi goale.....	22
ru	Набор пустых пробирок.....	23
sk	Súprava prázdných skúmaviek.....	24
sr	Komplet praznih epruveta.....	25
sv	Förpackning med tomma rör.....	26
tr	Boş tüp kiti.....	27
uk	Комплект пустих пробірок.....	28
zh	空试管套件.....	29

en

Empty tube kit

REF 944-230

For single use

Intended use

For *in vitro* diagnostic use.

To be used for liquid quality controls on the AQT90 FLEX analyzer.

Shelf life

The Empty Tube is stable until the expiry date printed on the box label.

Liquid quality control (LQC) solution

During tests to verify quality control on the analyzer using the Empty Tube as a container for LQC solutions, serum-based Bio-Rad's Liquichek Cardiac Markers Plus Control LT Levels 1, 2 and 3 were used. If other LQC solutions are used, new verification tests must be performed.

For detailed information about liquid quality control, see the *AQT90 FLEX Instructions for use*.

Contents

50 empty tubes

Components

- Outer plastic tube
- Inner glass tube
- Rubber septum
- Opal plastic cap
- Label

Preparing a tube for use in an LQC test procedure

NOTE: The procedure is illustrated inside the lid of the Empty Tube Kit box.

1. First remove the opal cap from the outer tube and then remove the rubber septum from the inner tube.
2. Use a pipette to transfer the relevant volume of LQC solution into the inner tube - the volume depends on the number of parameters you wish to analyze:
 - One parameter: 450 µL
 - Two parameters: 500 µL
 - Three parameters: 550 µL
 - Four parameters: 600 µL
 - Five parameters: 650 µL

3. Place the rubber septum over the inner tube.
4. Press it into place.
5. Place the opal cap over the outer tube and press it down until it closes with a click. The tube is now ready for use in a LQC test procedure on the AQT90 FLEX analyzer.

For detailed information about the procedure, see the *AQT90 FLEX Instructions for use*.

NOTE: Do not use the Empty Tubes to store samples.

The Empty Tube Kit does not include measuring components.

Explanation of graphical symbols

For detailed information about graphical symbols, see the *AQT90 FLEX Instructions for use*.

Disposal

Used materials must be disposed of in accordance with local procedures. Refer to recommendations of the manufacturer for disposal of liquid quality control (LQC) material.

Report to manufacturer

For the EU: If, during the use of this device, or as a result of its use, a serious incident has occurred, please report it to the manufacturer and the competent authority in the Member State.

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Radiometer, the Radiometer logo, ABL, AQT, TCM, RADIANCE, AQUIRE, PICO, CLINITUBES and QUALICHECK are trademarks of or used under license by Radiometer Medical ApS.
© 2022 Radiometer Medical ApS. All rights reserved.