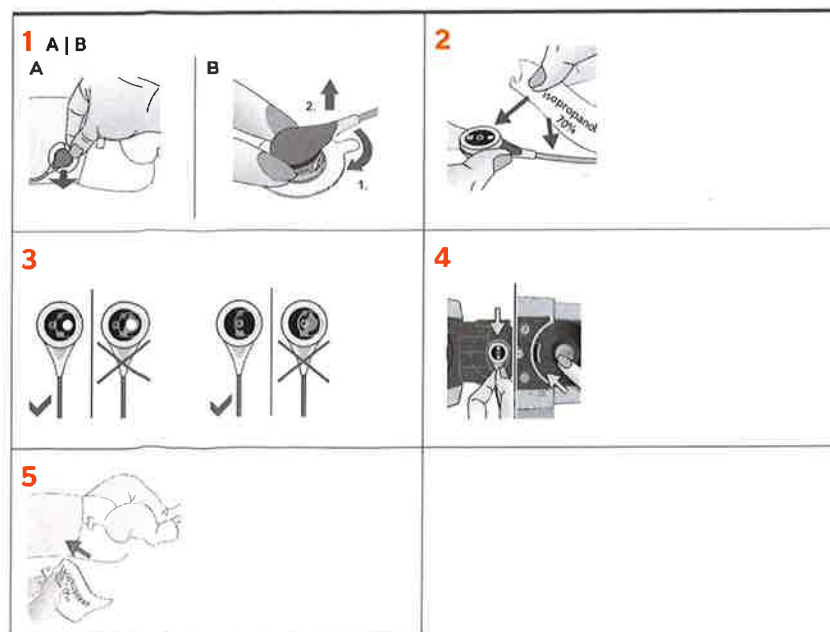
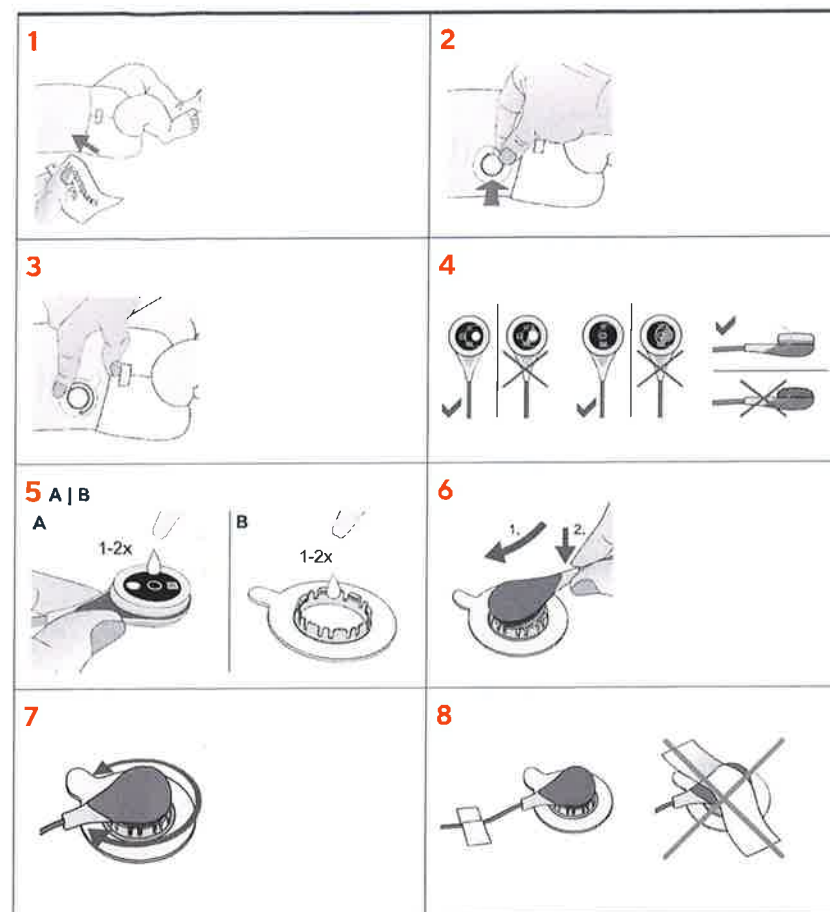
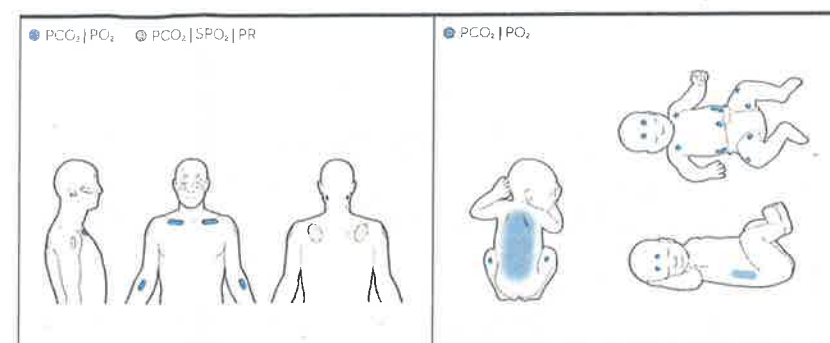
Sensor application/removal using a Multi-Site Attachment Ring

[illegible]

Warning: Please consult the Instruction Manual of the Sentec Digital Monitoring System for complete descriptions of warnings, cautions, recommendations of temperature setting and site selection, combination, and additional information.

Warnings/Cautions: Ensure that all parts are applied correctly and the physically intact and that the sensor is not damaged by sharp-edged objects or fingernails. Incorrect application and damage can cause incorrect measurements. Do not use sensor attachment accessories or Contact Gel on impaired skin on patients who exhibit allergic reactions to these disposable. The use of temperatures higher than 41 °C requires special attention to patients with susceptible skin, e.g. neonates, pediatric patients, burn victims, patients with skin diseases. A flat, well-perfused area of intact skin is needed. Do not place over large superficial veins or areas of skin breakdown or edema. Do not re-attach the sensor to the same measurement site if any skin irritations are noted. Do not remove the membrane of the sensor for cleaning and disinfection. Use 70% Isopropanol (for external use) or approved cleaning agent listed on www.sentec.com/ifu. Do not clean the sensor in an ultrasonic cleaner. Do not sterilize any parts described herein (e.g. by irradiation, steam, ethylene oxide or plasma methods). Do not remove the connector of the sensor cable in any liquid solution.

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