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English: These directions for use may be updated without further notice. Copies of the current version are available from the manufacturer. Ambu® is registered trademark of Ambu A/S, Denmark. Ambu A/S is certified according to ISO 9001 and ISO 13485.

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English

Ambu Disposable Pressure Manometer for single patient use

1. Introduction

- The Ambu Disposable Pressure Manometer is intended to be used for monitoring the patients airway pressure
- The Ambu Disposable Pressure Manometer is suitable for use with Ambu resuscitators or other resuscitators hyper-inflation bags, CPAP masks or circuits, if specified by the manufacturer.

2. Warnings/precautions

- For single patient use only. Reuse can cause cross-infection. Do not soak, rinse, or sterilize this device as these procedures may leave harmful residues. Discard after use.
- Do not connect pressure sources exceeding the limits (0-60 cm H₂O (hPa)) to the Ambu Disposable Pressure Manometer.
- Ambu Disposable Pressure Manometer must only be used by personnel trained in resuscitation and/or airway management.
- Make sure the Ambu Disposable Pressure Manometer is firmly attached to avoid leaking.

3. Specifications

Connector size: ID 3.7 mm (.15")

Measuring limits: 0-60 cm H₂O (hPa)

Accuracy (black line at end of piston located in the middle of number):

+/- 2 cm H₂O (hPa) at 5, 10, 15, 20 and 30 H₂O (hPa)

+/- 3 cm H₂O (hPa) at 40 H₂O (hPa)

+/- 5 cm H₂O (hPa) at 60 H₂O (hPa)

Dimensions: length 55 mm, diameter 22 mm

Weight: 6.4 g

Operating temperature: - 18°C to + 50°C

Storage: Tested at -40°C and +60° C according to EN ISO 10651-4:2002 and ASTM F 920-93

Latex and PVC free

4. Operating instructions

- Remove the cap from the manometerport on the device on which the manometer is to be attached. Retain the cap to cover the port later if manometer is no longer connected.
- Attach the manometer with its flexible connector onto the device´s manometer port.
- Occlude the patient port on the attached device and pres-surize the manometer to the maximum scale reading on the manometer.
- Release the pressure and check that the piston returns smoothly back below the "5" cm H₂O reading.
- Monitor pressure during ventilation.
- Store together with the resuscitator between uses on the same patient.

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