The SmartCan™

Instructions For Use

en	THE INTERSURGICAL SmartCan DISPOSABLE CO ₂ ABSORBER	4-5
fr	L'ABSORBEUR DE CO ₂ JETABLE INTERSURGICAL SmartCan	6-7
de	DER SmartCan EINWEG-CO ₂ -ABSORBER VON INTERSURGICAL	8-9
es	CARTUCHO ABSORBENTE DE CO ₂ DESECHABLE SmartCan DE INTERSURGICAL	10-11
pt	ABSORVENTE DE CO ₂ DESCARTÁVEL SmartCan DA INTERSURGICAL	12-13
it	ASSORBITORE DI CO ₂ MONOUSO SmartCan DI INTERSURGICAL	14-15
nl	INTERSURGICAL SmartCan DISPOSABLE CO ₂ ABSORBER	16-17
no	INTERSURGICAL SmartCan ENGANGS CO ₂ -ABSORBATOR	18-19
fi	KERTAKÄYTTÖINEN INTERSURGICAL SmartCan - CO ₂ - ABSORBERI	20-21
sv	INTERSURGICAL SmartCan CO ₂ - ABSORBER FÖR ENGÅNGSBRUK	22-23
da	INTERSURGICAL SmartCan CO ₂ - ENGANGS ABSORBER	24-25
el	ΑΠΟΡΡΟΦΗΤΗΣ CO ₂ ΜΙΑΣ ΧΡΗΣΗΣ SmartCan ΤΗΣ INTERSURGICAL	26-27
lt	„INTERSURGICAL SmartCan“ VIENKARTINIO NAUDOJIMO CO ₂ ABSORBENTAS	28-29
pl	JEDNORAZOWY POCHŁANIACZ CO ₂ SmartCan FIRMY INTERSURGICAL	30-31
ru	ОДНОРАЗОВЫЙ АБСОРБЕР CO ₂ SmartCan КОМПАНИИ INTERSURGICAL	32-33
cs	JEDNORÁZOVÝ ABSORBÉR CO ₂ INTERSURGICAL SmartCan	34-35
hu	INTERSURGICAL SmartCan ELDOBHATÓ CO ₂ - ABSZORBER	36-37
sl	ABSORBER CO ₂ ZA ENKRATNO UPORABO INTERSURGICAL SmartCan	38-39
lv	INTERSURGICAL SmartCan VIENREIZĒJAS LIETOŠANAS CO ₂ ABSORBĒTĀJS	40-41
et	INTERSURGICAL SmartCan- I ÜHEKORDELT KASUTATAV CO ₂ -ABSORBER	42-43
bg	АБСОРБЕР НА ВЪГЛЕРОДЕН ДИОКСИД (CO ₂) ЗА ЕДНОКРАТНА УПОТРЕБА SmartCan HA INTERSURGICAL	44-45
sr	INTERSURGICAL SmartCan ABSORBER UGLJEN-DIOKSIDA ZA JEDNOKRATNU UPOTREBU	46-47
ro	ABSORBANT DE CO ₂ DE UNICĂ FOLOSINȚĂ INTERSURGICAL SmartCan	48-49
sk	JEDNORÁZOVÝ ABSORBÉR CO ₂ INTERSURGICAL SmartCan	50-51
hr	JEDNOKRATNI APSORBER UGLJIKOVA DIOKSIDA (CO ₂) INTERSURGICAL SmartCan	52-53
tr	THE INTERSURGICAL SmartCan TEK KULLANIMLIK CO ₂ ABSORBANI	54-55
ja	INTERSURGICAL SMARTCAN デイスポーザブルCO ₂ 吸収剤	56-57
zh-s	使用说明INTERSURGICAL SMARTCAN 抛弃式 CO ₂ 吸收器	58-59
zh-t	使用說明INTERSURGICAL SMARTCAN 拋棄式 CO ₂ 吸收器	60-61
ar	العربية تعليمات الاستخدام: ماص ثاني أكسيد الكربون (CO ₂) الذي يُستخدم مرة واحدة من SmartCan INTERSURGICAL	62-63

INSTRUCTIONS FOR USE.

THE INTERSURGICAL SmartCan™ DISPOSABLE CO₂ ABSORBER

These instructions must be carefully read before using the Intersurgical SmartCan absorber. Also refer to the instructions of use provided by the anaesthetic machine manufacturer.

WARNING: A WARNING statement provides important information about a potentially hazardous situation which, if not avoided could result in death or serious injury

CAUTION: A CAUTION statement provides important information about a potentially hazardous situation which, if not avoided could result in minor or moderate injury.

TECHNICAL DATA

Product code	Absorbent weight/ volume	Resistance to flow of canister at 60 L/min	Casing	Dust Scrims	Typical pH	Absorbent details
2169001	1.60L / 1.4 kg / 3.1 lb	2.4 cm H ₂ O	Acrylonitrile Butadiene Styrene (ABS)	Polypropylene	Used below 10 Unused 12-14 depending on absorbent	For full details please contact Intersurgical
2169002 (Not available in the US market)	1.60L / 1.4 kg / 3.1 lb	2.4 cm H ₂ O				
2169003	1.60L / 1.2 kg / 2.8 lb	2.5 cm H ₂ O				

INTENDED USE:

To remove carbon dioxide from anaesthetic and respiratory gases delivered to a patient.

The Intersurgical SmartCan disposable CO₂ absorber, is for the absorption of carbon dioxide within an anaesthetic breathing system.

The SmartCan is compatible with the GE Healthcare® Carestation® 600 series, 620™, 650™, 650c Pendant™, 650c Wall Mount™ and 750™.

INTENDED USER:

For use by qualified medical personnel

CHOICES AVAILABLE:

Product codes:

2169001 SmartCan Spherasorb™ disposable CO₂ absorber, white to violet colour change.

2169002 (Not available in the US market) SmartCan Spherasorb™ disposable CO₂ absorber, pink to white colour change.

2169003 SmartCan LoFloSorb™ disposable CO₂ absorber, green to violet colour change.

For full details of Spherasorb and LoFloSorb please contact Intersurgical or refer to the Intersurgical Absorbents catalogue.

PRINCIPLE OF OPERATION

The absorbent within the SmartCan absorber undergoes a chemical reaction with the CO₂ within the breathing system. The hydroxides within the absorbent are converted to carbonates and water in an exothermic process. The CO₂ is therefore, bound within the absorbent.

WARNING If the moisture content of the absorbent is outside that set during the manufacturing process, this will affect the absorbing performance. For more information on this issue please see Warning No5 of the Safety Information section or contact Intersurgical.

FITTING AND REPLACING THE INTERSURGICAL SmartCan ABSORBER

to the GE Healthcare® Carestation®, 620™, 650™, 650c Pendant™, 650c Wall Mount™ and 750™.

Before attaching the SmartCan absorber, it is beneficial to ensure that the absorbent granules are well settled and level. This can be achieved by tapping gently on the side of the absorber.

WARNING Do not use the SmartCan if the absorber is cracked or damaged, or if the green sealing cap is missing. Also do not use if there is any unusual discolouration of the granules.

CAUTION The SmartCan absorber must complete and pass the machine pre-use leakage test before use. This must be done in accordance with the instruction for use of the GE Healthcare® Carestation®, on which it is being used (Carestation®, 620™, 650™, 650c Pendant™, 650c Wall Mount™ and 750™). Also make sure that the breathing circuit is free from any occlusions and all connections are clean, undamaged and free from dust.

CAUTION The SmartCan absorber must be correctly fitted to the GE Healthcare® Carestation® before starting the leakage test.

TO FIT AND REPLACE THE SmartCan:

Remove the green sealing cap from the top of the new SmartCan absorber.

Complete the following steps, 1 to 4.

Step 1: Lower the handle of the lifter dish so it is in the lower position.

Step 2: Slide the SmartCan absorber into place on the lifter dish - Figure 1.

Step 3: With the SmartCan absorber in position, raise the handle of the lifter dish, to lock the SmartCan into the breathing system - Figure 2.

Step 4: Conduct the machine pre-use leakage test in accordance with the instruction for use of the GE Healthcare® Carestation®, on which it is being used (Carestation®, 620™, 650™, 650c Pendant™, 650c Wall Mount™ and 750™).

Please note that failure of the pre-use leakage test may be due to leakage elsewhere in the machine or breathing system. In the event of failure of the pre-use test, also examine other possible causes for leakage in addition to the SmartCan.

Step 5: Replacing the SmartCan. To remove the SmartCan, pull down the handle of the lifter dish to unlock the absorber from the breathing system. When the lifter dish is fully lowered, lift the SmartCan and remove it from the lifter dish.

Step 6: Remove the seal from a new Intersurgical SmartCan disposable absorber and complete steps 1 to 4 as above, including the pre-use leakage test in accordance with the instruction for use of the GE Healthcare® Carestation®, on which it is being used (Carestation®, 620™, 650™, 650c Pendant™, 650c Wall Mount™ and 750™).

CAUTIONS

1. When exhaustion has been identified, the SmartCan absorber must be removed and replaced to ensure that CO₂ absorption in the breathing system is continued.

2. CO₂ is not being absorbed when the SmartCan absorber is not attached to the GE Healthcare® Carestation®, 620™, 650™, 650c Pendant™ 650c Wall Mount™ and 750™).

3. A new filter or HMEF should be used on the patient Y piece of the breathing system for every patient. When this is done, the SmartCan is not restricted to single patient use and can be used until exhaustion is identified. It is advised that a filter or HMEF is also placed at the expiratory port of the anaesthetic machine.

4. If the SmartCan has been attached to the machine for one month, it is recommended to replace it regardless of extent of usage.

Figure 1.
Presenting SmartCan to
lifter dish

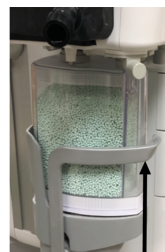
Figure 2.
Raising handle and locking
SmartCan into position

5. The degree of colour change at exhaustion depends on the conditions of use and is an indication only. The SmartCan absorber must be used in conjunction with inspiratory FiCO_2 monitoring. The SmartCan absorber needs replacing when the inspiratory FiCO_2 concentration exceeds 5mmHg (approximately 0.5% volume FiCO_2)

NOTE: With some absorbents, the colour indicator can sometimes regenerate slowly in between uses and the absorbent may revert back to its original colour. However, this is not a regeneration of the absorption capacity. Therefore, the Intersurgical SmartCan absorber should be discarded in the first instance that exhaustion has been determined.

Always refer to and follow the Instructions For Use of the GE Healthcare® Carestation®, 600 series being used (Carestation®, 620™, 650™, 650c Pendant™, 650c Wall Mount™ and 750™).

WARNINGS

1. DO NOT use the SmartCan if the pre-use leakage test fails.
2. The GE Healthcare® Carestation® 600 series have a CO_2 bypass system allowing replacement of the absorber during ventilation, without disturbing gas flow to the patient. CO_2 is not being removed while this CO_2 bypass is active.

SAFETY INFORMATION

The SmartCan absorber may be used with oxygen, nitrous oxide, Xenon, Enflurane, Halothane, Isoflurane, Desflurane and Sevoflurane.

CAUTIONS

1. Before using the SmartCan it is essential to become familiar with and follow the Instructions For Use of the GE Healthcare® Carestation®, 600 series being used (Carestation®, 620™, 650™, 650c Pendant™, 650c Wall Mount™ and 750™).
2. The SmartCan absorber must be used in conjunction with monitoring of FiCO_2 inspiratory carbon dioxide.
3. Once exhausted, the SmartCan cannot be reused. Do not leave attached to the machine once exhaustion is identified.

WARNINGS

1. Do not allow contact with alcohol based cleaning products.
2. Always conduct a pre-use leakage test, according to the Instructions For Use of the GE Healthcare® Carestation®, 600 series being used (Carestation®, 620™, 650™, 650c Pendant™, 650c Wall Mount™ and 750™). Do not use the SmartCan if the pre-use leakage test fails.
3. Do not use with trichloroethylene or chloroform because toxic products may be produced by chemical reactions
4. Do not use if the scrims are missing or out of place or if granules or dust have escaped beyond these scrims.
5. The carbon dioxide absorbent inside the SmartCan absorber has a moisture content set in the manufacturing process. Do not flush the SmartCan absorber with dry gas for long periods. This may result in loss of moisture and reduced CO_2 absorption efficiency. Excessive loss of moisture can lead to failure to absorb CO_2 and to clinical problems.
6. Do not sterilize, disinfect or clean the SmartCan absorber before or during use. Cleaning can lead to device malfunction and breakage.
7. Do not use the SmartCan after expiry date.
8. Never allow pure carbon dioxide to pass through any Carbon Dioxide Absorbents or disposable absorbers, as this will generate high temperatures.
9. Do not use if the SmartCan absorber becomes excessively hot.
10. Do not use if it is believed that dry gas has been left running during long periods of non-use at high flows.
11. Do not use if there is a delay in the increase of inspired agent concentration.
12. It is recommended to use water traps in the patient circuit during low flow anaesthesia (fresh gas flows of 1 L/min or less) and when excessive condensed moisture accumulates in the limbs of the breathing system. Use water traps in both the expiratory and inspiratory limbs to remove water condensation.

For more information please refer to the Intersurgical Absorbents Catalogue or contact Intersurgical.

RISK AND SAFETY PHRASES RELATING TO THE ABSORBENTS WITHIN THE SmartCan ABSORBER.

H315: Causes skin irritation.

H318 Causes serious eye damage.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P302/P352 IF ON SKIN: Wash with plenty of soap and water.

P305/351/338. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P332/313: If skin irritation occurs: Get medical advice/attention.

STORAGE

The SmartCan absorber must be stored, sealed, in a clean, dry environment away from direct sunlight and not under heavy load.

Recommended storage at room temperature over the indicated product shelf-life.

DISPOSAL

Intersurgical advises that after clinical use, the waste SmartCan is treated at source with minimal handling; by incineration within a mixture of other clinical waste. In any case, the SmartCan must be disposed of according to local regulations. Refer to the MSDS for further details.

SAFETY DATA SHEET

The Material Safety Data Sheet is available on request.

Rx ONLY - Federal law restricts this device to sale by or on the order of a physician.

*GE Healthcare is a registered trademark of General Electric Company

Carestation is a registered trademark of Datex-Ohmeda, Inc.

Datex-Ohmeda, Inc., is a General Electric Company.

620, 650, 650c Pendant, 650c Wall Mount and 750 are trademarks of General Electric Company.

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