NIBP Cuff

Instructions for Use

CM12XX/CM13XX/CM17XX Series

Shanghai International Holding Corp. GmbH (Europe)
Eiffestraße 80, 20537 Hamburg, Germany

mindray

Cuff Type	Model	Limb Circumference	Applicable to	Application Site	Others
Reusable NIBP cuff	CM12XX Series (detachable bladder)	CM1200	7 - 13 cm	Small infant	Without Lid
		CM1201	10 - 19 cm	Infant	
		CM1202	18 - 26 cm	Pediatric	
		CM1203	25 - 35 cm	Adult	
		CM1204	33 - 47 cm	Large adult	
	CM13XX Series (integral bladder)	CM1205	46 - 66 cm	Adult	Without Lid
		CM1300	7 - 13 cm	Small infant	
		CM1301	10 - 19 cm	Infant	
		CM1302	18 - 26 cm	Pediatric	
		CM1303	25 - 35 cm	Adult	
Reusable NIBP cuff	CM13XX Series (integral bladder)	CM1304	33 - 47 cm	Adult	With Lid
		CM1305	46 - 66 cm	Adult	
		CM1306	25 - 35 cm	Adult	
		CM1307	33 - 47 cm	Large adult	
		CM1702	18 - 26 cm	Pediatric	
	CM17XX Series (detachable bladder)	CM1703	25 - 35 cm	Adult	Without Lid
		CM1704	33 - 47 cm	Adult	
		CM1702A	18 - 26 cm	Pediatric	
		CM1703A	25 - 35 cm	Adult	
		CM1704A	33 - 47 cm	Large adult	
		CM1707A	33 - 47 cm	Large adult	

Warnings

- Federal law (USA) restricts this device to sale by or on the order of a physician.
- Check the cuff before using. Do not use the cuff for NIBP monitoring in case of damage, block, deterioration or visible contamination.
- For disposal of accessories, follow local regulations or your hospital's policy regarding disposal of such accessories. Do not dispose randomly.
- Choose the correct cuff size as per the patient category.
- The user shall be responsible for ensuring the compatibility between this cuff and other equipment by either referring to the instructions for using this cuff or contacting us before use. Otherwise, product performance may be compromised or personal injury may occur.
- For monitors performing NIBP measurements during inflation, select cuffs with the **TrueBP™** symbol. Using other cuffs may lead to incorrect or failed measurements.
- Do not place the cuff on a limb being used for intravenous infusion or arterial catheter. This could cause tissue damage around the catheter when infusion is slowed down or blocked during cuff inflation.
- Keep the extension hose connecting the cuff and the monitoring equipment unclogged and prevent the hose from kinking.
- Do not press the hose on the cuff.
- Do not place the product in an environment that exceeds the stated range.
- Do not allow liquid to enter the pipelines and bladder/or integral bladder) of the cuff when cleaning. Using the cuff with liquid inside for NIBP measurement will lead to the risk of equipment damage.

Be sure to understand all the precautions and warnings listed in previous sections before applying any cuff to a patient.

- Choose correct cuff size as per patient's limb circumference.
- Check that the cuff is completely inflated. Refer to the instructions below to apply the cuff around the patient's arm (or thigh) and make sure that the marking  on the cuff is over the brachial artery (or femoral artery) location.
- Do not wrap the cuff too tightly around the limb. Excessive tightness may cause discoloration and ischemia of the limb distal to the hose.
- Connect the cuff with the extension hose.

Caution:

- Improper cuff placement may cause inaccurate measurements.
- The limb taking NIBP measurement shall be at the same level as the patient's heart.

Cleaning/Disinfection

Cautions

- Do not dry clean the cuff.
- Do not press the cuff with a hot iron.
- Never use cleaning agents/disinfectants other than the recommended.
- Clean and disinfect the cuff according to the instructions.
- Do not clean or disinfect the metal parts of this product. Otherwise, metal part corrosion or product reliability deterioration could result.

NIBP CUFF

INSTRUCTIONS FOR USE

Statement

SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD. (hereinafter called Mindray) owns the intellectual property rights to this product and this manual. Disclosure of the information in this manual in any manner whatsoever without the written permission of Mindray is strictly forbidden.

This manual provides the instructions necessary to operate the product in accordance with its function and intended use. Observance of this manual is a prerequisite for proper performance and correct operation, and ensures patient and operator safety.

- Mindray is responsible for the effects on safety, reliability and performance of this product, only if:
- the product is used in accordance with the instructions for use.
 - this product is not damaged by human factors. Human factors refer to unintentional falling, intentional damaging, etc.

In the event that it becomes necessary to return a unit to Mindray, please contact the Mindray Service Department and obtain a Mindray Customer Service Authorization Number. The Mindray Customer Service Authorization Number should appear on the outside of the shipping container. Return shipments will not be accepted if the Mindray Customer Service Authorization Number is not clearly visible. Please provide the model number, serial number, and a brief description of the reason for return. The customer is responsible for freight charges when this product is shipped to Mindray for service (including any relevant customs fees or other freight related charges).

mindray  **MINDRAY** and  are the registered trademarks or trademarks owned by Mindray in China and other countries.

The issue date of this manual is 2022-11.
© 2006-2023 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights reserved.

Notification of Adverse Events

As a health care provider, you may report the occurrence of certain events to SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD., and accordingly to the competent authority of the Member state in which the user and/or patient is established.

These events, include device-related death and serious injury or illness. In addition, as part of our Quality Assurance Program, SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD. requests to be notified of device failures or malfunctions. This information is required to ensure that SHENZHEN MINDRAY BIO-MEDICAL ELECTRONICS CO., LTD. provides only the highest quality products.

Intended Use

Intended Purpose Statement

The NIBP accessory is intended for connecting with Mindray medical devices that support NIBP monitoring.

Indication for Use

The reusable NIBP accessory is intended for manual measurement and automatic non-invasive blood pressure monitoring.

The NIBP accessory must be used in connection with the monitoring device (including accessories) specified by our company, including the MEC, IMEC, PM, VS, and T series patient monitors of Mindray that supports the NIBP measurement function. The user shall be responsible for verifying the responsible for verifying the compatibility of this accessory and the equipment before use. This cuff must be used by trained clinical professionals. Otherwise, product performance may be compromised or personal injury may occur.

Intended Users

The NIBP accessory must be used by trained clinical professionals.

Intended Patient Population

The NIBP accessory is intended for manual measurement and automatic non-invasive blood pressure monitoring of adult, pediatric, and neonatal patients.

Intended Medical Conditions

The product is expected to be used in medical institutions.

Contraindications

None.

Side Effects

According to the conclusion of clinical evaluation and residual risk evaluation, for the intended patients, there is no known side effects that can occur during or after the use of the medical device. And there is no need for the operator to make extra preparations. Thus, no residual risk associated with using the medical device should be disclosed due to the risk management report.

Product Introduction

The NIBP cuff is intended for collecting, transmitting human's blood pressure signals, and implement-ing NIBP measurements by connecting human body and the patient monitor.

The CM12XX/CM13XX/CM17XX series reusable NIBP cuffs shall only be used for NIBP measurement of large adult/adult/pediatric/infant /small infant patients.

All the cuffs must be used in connection with the monitoring device (including accessories) specified by our company. The user shall be responsible for ensuring the compatibility between the NIBP cuff and the monitoring device (including accessories) by either referring to the instructions for using this cuff or contacting us before use. The cuffs must be used by trained clinical professionals. Otherwise, product performance may be compromised or personal injury may occur.

Cuffs with the **TrueBP™** symbol can be used with monitoring devices for NIBP measurements during deflation or inflation.

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