

# 8x11cm 80g 双胶纸双面印刷

## Anesthesia Mask



REF CP650014 LOT 23001082 2022-09-15 2025-09-14

Size: ☐ 0-X# ☐ 1-X# ☐ 2-X#  
☐ 3-X# ☒ 4-X# ☐ 5-X#  
☐ 6-X#



(01)03663996018588  
(10)23001082  
(17)250914

### Directions For Use:

1. Adjust mask cushion with syringe (for the mask with check-valve). Check the cushion by gently pressing it with hand.
2. Place mask cushion on the patient's chin first, roll cushion up the face with gentle but firm pressure.
3. If provided, the Hook Ring can be used in conjunction with a headstrap or removed when not needed.
4. Connect the mask to other device.

### Caution:

1. For single use. Discard after use.
2. Used by professional doctor or trained medical staff.
3. This device cannot be used for the patients with facial trauma and PVC allergy.
4. Do not use if package or device is open or damaged.
5. Do not store at extreme temperature and humidity. Store in cool and dry place.
6. Ensure that all connections are secure.



35 rue du Sausset - 93290 Tremblay en France - France  
Tel.: +33 (0)1 42 43 50 50 - Fax: +33 (0)1 42 43 64 44  
info@comepa.com - www.comepa.com



Xiamen Compower Medical Tech. Co., Ltd.  
Unit 301, No.16, Xianghong Road, Xiang'an Torch  
Industrial Zone, 361101 Xiamen P.R. China  
Tel: +86-592-5799319 Fax: +86-592-5796765



Shanghai International Holding Corp. GmbH (Europe)  
Eiffestrasse 80, 20537 Hamburg, Germany  
Tel: +49-40-2513175 Fax: +49-40-255726



Version 2.3 2024.03.10

## Masque d'Anesthésie

REF CP650014 LOT 23001082 2022-09-15 2025-09-14

Size: ☐ 0-X# ☐ 1-X# ☐ 2-X#  
☐ 3-X# ☒ 4-X# ☐ 5-X#  
☐ 6-X#



(01)03663996018588  
(10)23001082  
(17)250914

### Instructions d'utilisation :

1. Ajustez le coussin du masque avec une seringue (pour le masque avec valve de contrôle). Vérifiez le coussin en appuyant doucement dessus avec la main.
2. Placez d'abord le coussin sur le menton du patient, puis placez le masque sur le visage avec une pression douce mais ferme.
3. Si fourni, l'anneau de maintien peut être utilisé en conjonction avec un serre-tête élastique ou retiré lorsqu'il n'est pas nécessaire.
4. Connectez le masque à l'autre dispositif.

### Précaution :

1. Pour usage unique. Jetez après utilisation.
2. À utiliser par un médecin ou du personnel médical formé.
3. Ce dispositif ne peut pas être utilisé pour les patients ayant subi un traumatisme facial et qui ont des soins actifs.
4. N'utilisez pas si l'emballage ou le dispositif est ouvert ou endommagé.
5. Ne pas stocker à l'extrême température et humidité. Stockez dans un endroit frais et sec.
6. Assurez-vous que toutes les connexions sont sécurisées.