COMEPA Bougie IFU
Size: A5
1 IFU in box

Black
黑色

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EN Tracheal Tube Introducer (Bougie)
Used for Endotracheal Tube Intubation

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

Product description:

The Tracheal Tube Introducer (Bougie) designed for intubation assistance, provide good directional control during routine or difficult oral intubations when the bad visualization of vocal cords. The Bougie made from medical polyethylene (PE), provides proper stiffness for ease of insertion. Length marking on the introducer could indicate the distance of insertion easily observed for safety.

Intended Use:

The Tracheal Tube Introducer is used to guide the endotracheal tube during difficult intubations. Short term use.

Instructions for use:

- 1) Preload an appropriately sized, lubricated endotracheal tube onto the proximal part of the introducer.
- 2) Using laryngoscopy, introduce the tip of the introducer beyond the epiglottis and advance it in a straight line toward the glottis.
- 3) Advance the introducer into the trachea. If resistance is encountered, do not force the introducer; gently rotate and advance the introducer into the trachea.
NOTE: During introduction, you may feel tracheal rings.
- 4) Confirm introducer position using standard methods (e.g., capnography, breath sounds, X-ray).
- 5) Maintaining position of the introducer, advance the preloaded endotracheal tube into the trachea to the appropriate distance.
- 6) Maintaining position of the endotracheal tube, remove the introducer and laryngoscope.
- 7) Confirm endotracheal tube position using standard methods (e.g., capnography, breath sounds, X-ray).
NOTE: The vented type with connector for use with a ventilatory device, the connector can be attached on the tube end or removed from the tube.

Caution:

- 1) Sterilized by Ethylene Oxide (EO).
- 2) For single use only, do not re-sterilize or reuse.
- 3) Do not use if the package or product has been damaged or contaminated.
- 4) Do not store at extreme temperatures and humidity, avoid direct sunlight.
- 5) The device should only be used by personnel trained in it's use.

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Tel. +33 (0) 1 42 43 50 50 - Fax: +33 (0) 1 42 43 64 44

info@comepa.com - www.comepa.com

Hangzhou Formed Medical Devices Co., Ltd.

Add: Building 3, No. 292 Gonghe North Road, Xindeng Town, Fuyang District, 311404 Hangzhou, People's Republic of China

Shanghai International Holding Corp. GmbH(Europe)

Add: Eiffestrasse 80, 20537 Hamburg, Germany

V5-231223