Respironics Inc.
1001 Murry Ridge Lane
Murrysville, PA 15668 USA

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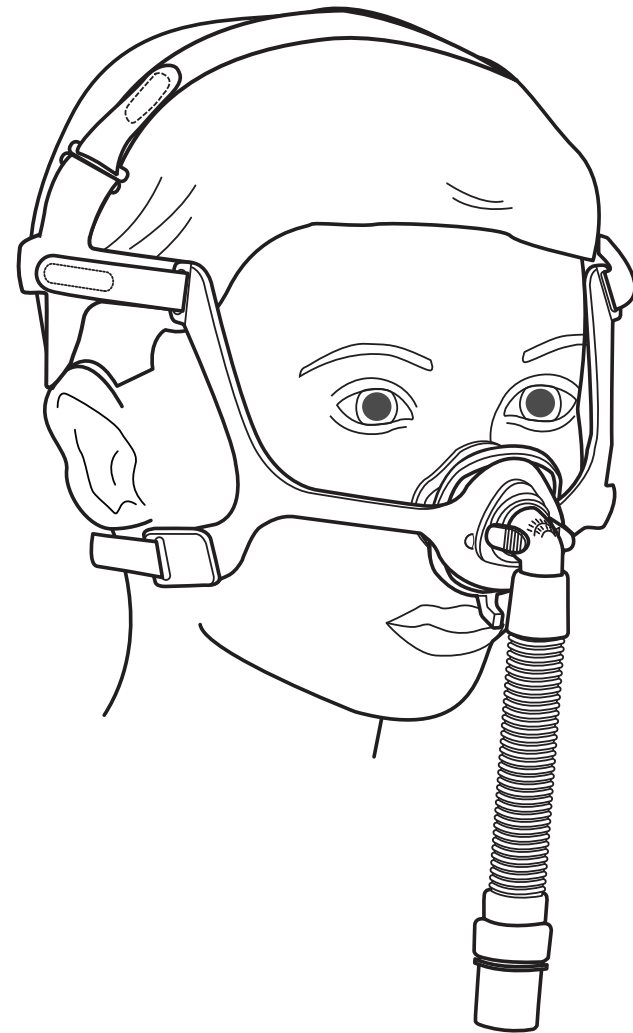
AUSTRALIAN SPONSOR DETAILS:

Philips Electronics Australia Ltd.
65 Epping Road, North Ryde, NSW 2113
Australia

1130591 R01
LZ 8/26/2016

©2016 Koninklijke Philips N.V. All rights reserved.

PN841 Leak 3
Pediatric
Nasal Mask

PHILIPS
RESPIRONICS

PN841 Leak 3 Pediatric Nasal Mask

EN	003	INSTRUCTIONS FOR USE	SK	108	NÁVOD NA POUŽITIE
FR	010	MODE D'EMPLOI	SL	115	NAVODILA ZA UPORABO
IT	017	ISTRUZIONI PER L'USO	LV	122	LIETOŠANAS INSTRUKCIJA
DE	024	GEBRAUCHSANWEISUNG	LT	129	NAUDOJIMO INSTRUKCIJA
ES	031	INSTRUCCIONES DE USO	ET	136	KASUTUSJUHEND
PT	038	INSTRUÇÕES DE UTILIZAÇÃO	RO	143	INSTRUCȚIUNI DE UTILIZARE
EL	045	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	BG	150	ИНСТРУКЦИИ ЗА УПОТРЕБА
DA	052	BRUGSANVISNING	UK	157	ІНСТРУКЦІЇ ІЗ ЗАСТОСУВАННЯ
NL	059	GEBRUIKSAANWIJZING	МК	164	УПАТСТВА ЗА УПОТРЕБА
SV	066	BRUKSANVISNING	HR	171	UPUTE ZA UPORABU
FI	073	KÄYTTÖOHJEET	ZHTW	178	使用說明
NO	080	BRUKSANVISNING	RU	185	ИНСТРУКЦИИ ПО ПРИМЕНЕНИЮ
PL	087	INSTRUKCJA STOSOWANIA	KK	192	ҚОЛДАНУ НҮСҚАУЛАРЫ
CS	094	NÁVOD K POUŽITÍ	SR	199	UPUTSTVO ZA UPOTREBU
HU	101	HASZNÁLATI ÚTMUTATÓ	TR	206	KULLANIM TALIMATLARI

PHILIPS

RESPIRONICS

PN841 Leak 3 Pediatric Nasal Mask

Instructions for use - **EN**

Safety Information

Intended Use

This mask is intended to provide an interface for application of non-invasive ventilation to patients. The mask is for single patient use in the hospital/institutional environment only. The mask is to be used on patients 5 kg/11 lbs or greater (≥ 5 kg/11 lbs) for whom non-invasive ventilation has been prescribed. Use of this mask is limited by the indications for use of the compatible therapy device with respect to patient weight.

⚠ Caution: U.S. Federal law restricts this device to sale by or on the order of a physician.

ⓘ Note: This mask is not made with natural rubber latex, DEHP, or BPA.

ⓘ Note: This mask can be used with a variety of therapy devices which may have their own limitations on patient population. If the therapy device itself is limited to patients greater than 5 kg, then the use of this mask does not expand the intended use of that therapy device.

Contraindications

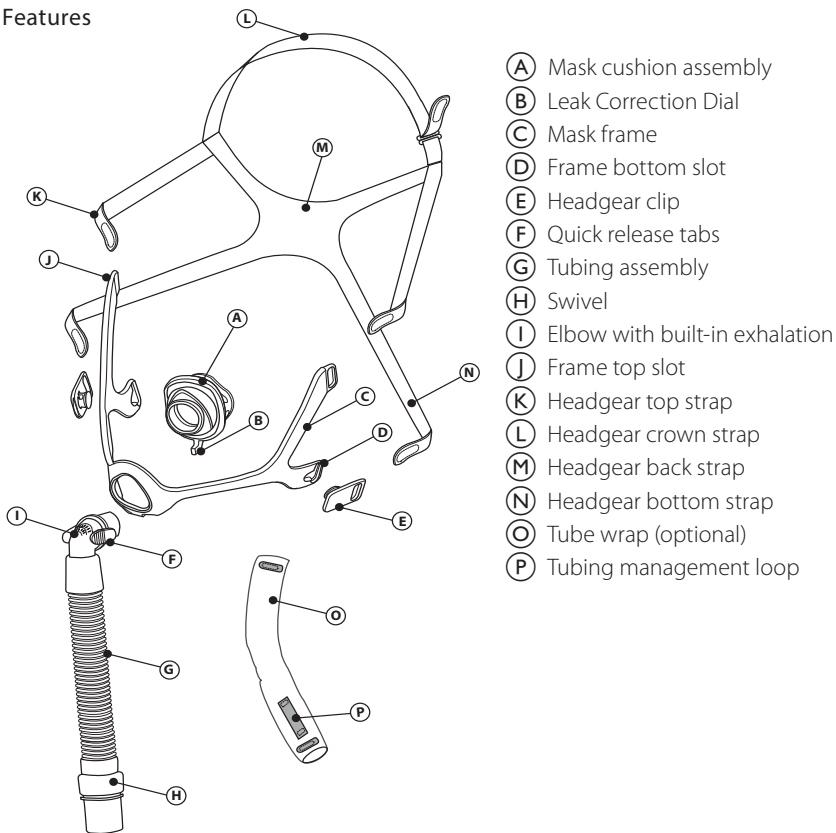
This mask is not intended for patients who are dependent on mechanical ventilation for their life support.

⚠ Warnings

- When using this mask, it is the responsibility of the healthcare professional to ensure proper device setup in accordance with patient condition, including the use of proper alarms.
- Appropriate patient monitoring and alarms should be used for patients who are unable to remove the mask themselves to assure the mask and tubing remain in place during therapy.
- The use of nasal or full face masks applies forces on the head and may affect cranial facial growth patterns. Regular healthcare professional assessment is recommended when using this mask.
- Use of this mask while the system is not on and operating may cause rebreathing of exhaled air. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation.
- Monitor or intervene if the patient experiences skin redness, irritation, discomfort, blurred vision, or drying of the eyes.
- Monitor or intervene if any of the following symptoms occur: unusual chest discomfort, shortness of breath, severe headache, eye pain or eye infections.
- Monitor or intervene if any of the following symptoms occur: tooth, gum, or jaw soreness. Use of a mask may aggravate a patient's existing dental conditions.
- The mask material should be away from eyes.
- Do not overtighten the headgear straps. Watch for signs of overtightening, such as excessive redness, sores or bulging skin around the edges of the mask. Loosen the headgear straps to alleviate symptoms.
- Do not block or seal off the exhalation ports.
- This mask is for single patient use only.
- This mask is not designed for multi-patient use. Respironics cannot guarantee the performance specifications once this mask has been disinfected.

Mask Parts

Features

Before Use

- Wash the patient’s face. Do not use moisturizer/lotion on your hands or the patient’s face.
- Inspect the mask. Replace the mask or components if broken or worn.

Mask Care

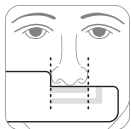
Cleaning Instructions for Non-Fabric Parts

1. Wash in warm water with liquid dishwashing soap.
 2. Rinse thoroughly.
 3. Air dry completely before use.
 4. Replace or remove the fabric parts after 1 week. Do not clean the fabric.
- ⚠ Caution:** Do not clean the mask with bleach, alcohol, solutions containing bleach or alcohol, or solutions containing conditioners or moisturizers.
- ⚠ Caution:** Any deviation from these instructions may impact the performance of the product.
- ⚠ Caution:** Inspect the mask for damage or wear (cracking, crazing, tears, etc). Discard and replace components or mask as necessary.




Achieving the Right Fit and Using the Mask

Before Putting on the Mask

Sizing: Select the cushion that fits the width of the nose without blocking the nostrils. The mask sizing gauge can be used as a guide by holding the gauge under the nose (see illustration).






Putting on the Mask

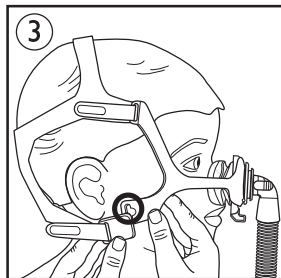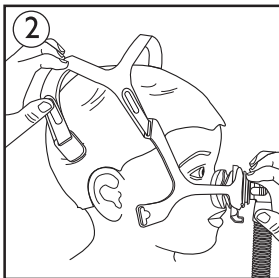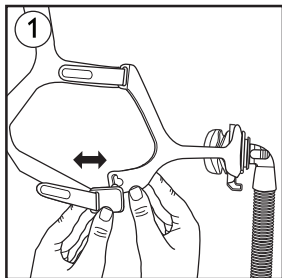1. Headgear clip: Disconnect one or both by twisting the clip outward. ①
2. Headgear: Pull headgear on over head. ②
3. Cushion: With the Leak Correction Dial facing downward, place cushion over nose. ③ The top of the cushion should sit at the bridge of the nose.
4. Headgear clip: Press the clip against the frame slot to reattach. ③

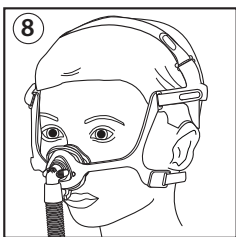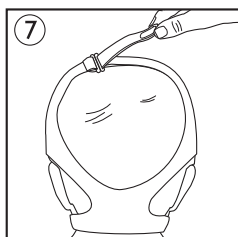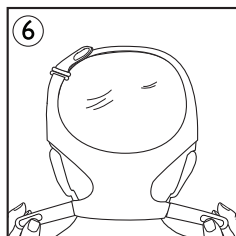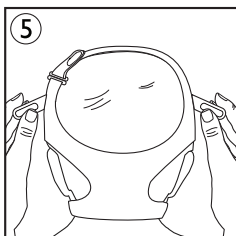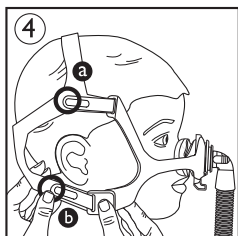Adjusting the Mask


1. Headgear: Peel the headgear tabs away from the headgear straps then press tabs against the headgear straps to reattach. ④
 - Adjust the top straps to the headgear to achieve a comfortable fit.
 - Adjust the bottom strap to the headgear clip to achieve a comfortable fit.

2. Headgear: Adjust top straps evenly. ⑤
3. Headgear: Adjust bottom straps evenly. ⑥
4. Headgear: Adjust crown strap so that the headgear and frame do not touch the ears. ⑦
5. The mask should rest comfortably against the face. ⑧

If there are excessive air leaks, adjust the headgear. For small mask leaks during sleep, see the **Leak Correction Dial** section.

Note: Do not overtighten the headgear.

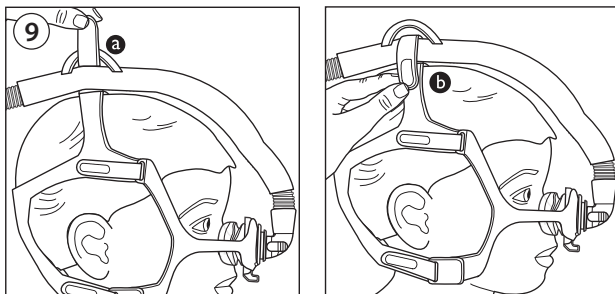6. Using tube management (optional): The flexible tubing may be placed over the head ⑨.
 - a Slide the end of the crown strap through the loop on the tube wrap.
 - b Route the end of the crown strap through the slot in the headgear and fasten into place.


Using the Mask

1. Connect the patient circuit from the ventilator to the swivel.
2. Turn the therapy device on. Make sure the child is breathing normally.

Leak Correction Dial

The Leak Correction Dial should point straight downward each time the mask is placed on the child.

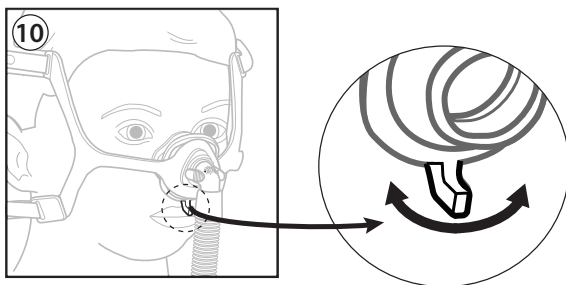If the child experiences small mask leaks while sleeping, move the dial to the left or right ⑩. If leaks cannot be corrected using the dial, re-seat the mask by pulling it directly away from the face and gently setting it back into place.




Removing the Entire Mask

Disconnect bottom headgear clips and slide the mask upward over the head.

Disassembly

Removing the Elbow and Tubing

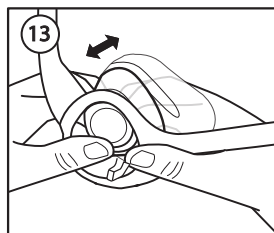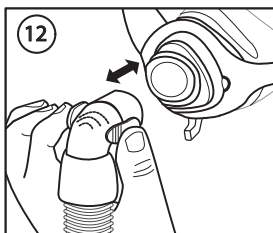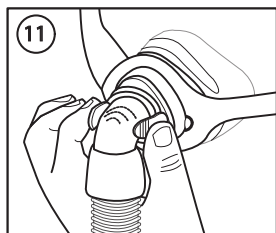Squeeze the quick release tabs and pull the elbow away from the cushion ⑪⑫.

Removing the Cushion

Push the cushion backwards through the frame ⑬.

Removing the Headgear

1. Detach the headgear bottom clips from the frame or undo the bottom tabs and pull.
2. Undo the headgear top tabs and pull the straps through the frame.

Assembly

Attaching the Cushion

With the Leak Correction Dial pointing downward, push the cushion into the frame towards the front and press into place.

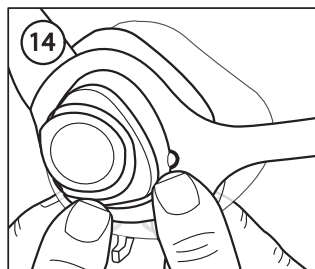
The sides of the cushion will match the alignment cutouts in the mask frame ⑭.

Attaching the Elbow and Tubing

Push the elbow into the cushion until the quick release tabs click into place.

Attaching the Headgear

1. Hold the crown strap upward.
2. Make sure the back strap that displays "Philips Respironics" is facing outward and is right side up.
3. Slide the headgear top strap into the frame top slots.
4. Connect the headgear clips to the frame bottom slot.

Comfort Tips for Caregivers

- If the child has a preference for a certain sleeping position, consider this when choosing placement of the ventilator.
- The most common mistake is overtightening the headgear. The headgear should fit loosely and comfortably. If the skin bulges around the mask or if red marks appear, loosen the headgear.
- If necessary, adjust the crown strap to move the headgear off of the ears.
- The quick release elbow allows the mask to be temporarily disconnected from the tubing without the child having to remove the mask. The elbow can be quickly clicked back into place to resume therapy.
- Use of the optional tube management feature moves the tube away from the front of the body and reroutes it over the child's head. The quick release feature cannot be used when tube is routed over the top of the head using the tubing management loop on the tube wrap.

Health Care Provider & Clinician Information

Leak Symbol and Port Settings

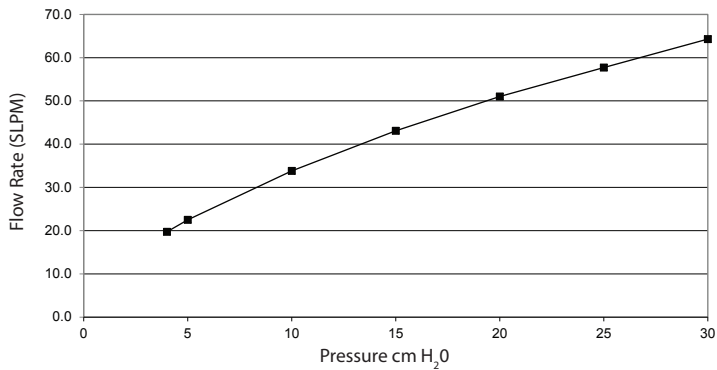
Some ventilators use a leak value and symbol in the mask selection control. The leak value and symbol represents the intentional leak of the mask. On ventilators equipped with a mask selection control, enter **K3**.

Specifications

The technical specifications of the mask are provided to determine if it is compatible with your therapy device, and that alerts and alarms function properly.

⚠ Warning: Use outside of these specifications may result in ineffective therapy. The pressure flow curve shown below is an approximation of expected performance. Exact measurements may vary.

Pressure Flow Curve

Resistance

Drop in Pressure at:	
50 SLPM	100 SLPM
All Sizes 0.7 cm H ₂ O	2.8 cm H ₂ O

Deadspace

SCS	18.9 mL
SCM	24.3 mL
SCL	35.9 mL

Sound Levels

A-weighted Sound Power Level	29.8 dBA
A-weighted Sound Pressure Level at 1m	22.1 dBA

Disposal

Dispose of in accordance with local regulations.

Storage Conditions

Temperature: -4° F to 140° F (-20° C to 60° C)
Relative Humidity: 15% to 95%, non-condensing

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