Respironics Inc.
1001 Murry Ridge Lane
Murrysville, PA 15668 USA

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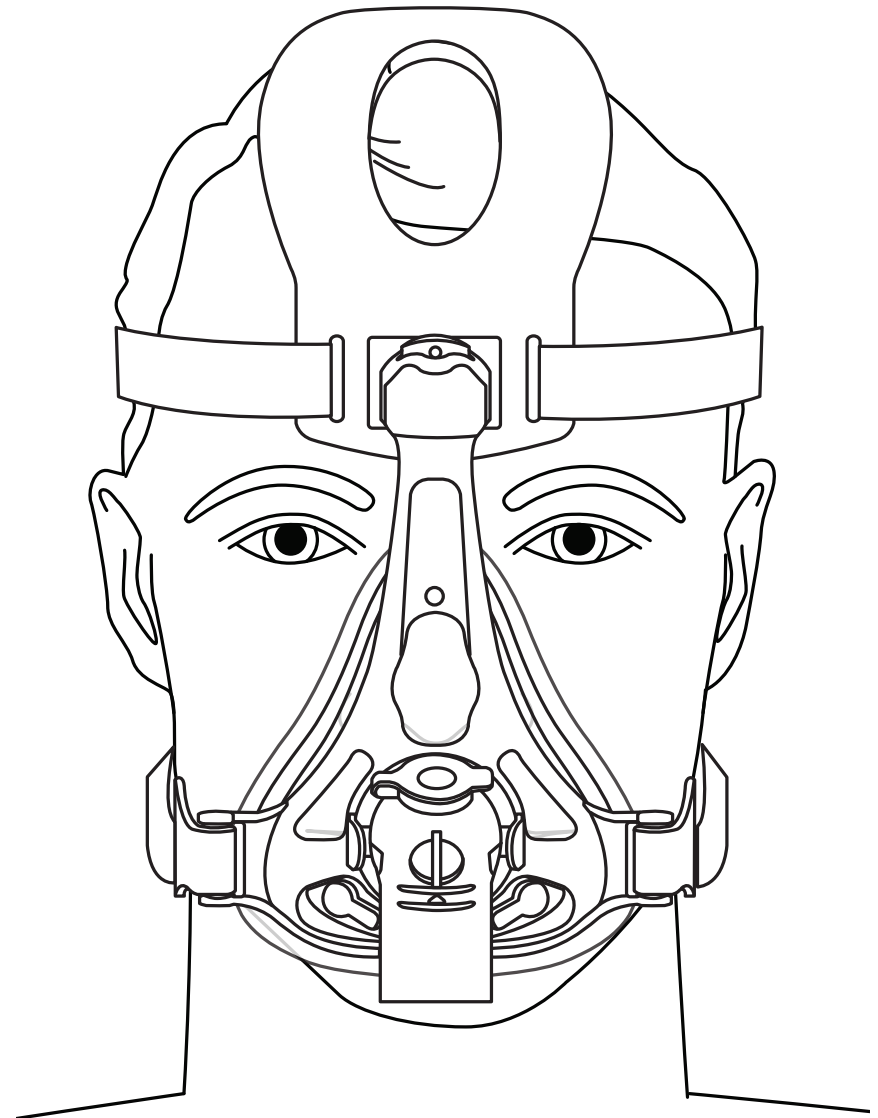
EC REP

AUSTRALIAN SPONSOR DETAILS:

Philips Electronics Australia Ltd.
65 Epping Road, North Ryde, NSW 2113
Australia

1120845 R06
LZ 2/8/2017

©2017 Koninklijke Philips N.V. All rights reserved.

AF541 EE Leak 1 Oro-Nasal Mask

PHILIPS

RESPIRONICS

AF541 EE Leak 1 Oro-Nasal Mask

EN	001	INSTRUCTIONS FOR USE	SK	119	NÁVOD NA POUŽITIE
FR	008	MODE D'EMPLOI	SL	126	NAVODILA ZA UPORABO
IT	016	ISTRUZIONI PER L'USO	LV	134	LIETOŠANAS INSTRUKCIJA
DE	024	GEBRAUCHSANWEISUNG	LT	141	NAUDOJIMO INSTRUKCIJOS
ES	032	INSTRUCCIONES DE USO	ET	149	KASUTUSJUHEND
PT	040	INSTRUÇÕES DE UTILIZAÇÃO	RO	156	INSTRUCȚIUNI DE UTILIZARE
EL	048	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	BG	164	ИНСТРУКЦИИ ЗА УПОТРЕБА
DA	057	BRUGSANVISNING	UK	172	ІНСТРУКЦІЇ ІЗ ЗАСТОСУВАННЯ
NL	065	GEBRUIKSAANWIJZING	МК	180	УПАТСТВО ЗА УПОТРЕБА
SV	073	BRUKSANVISNING	HR	188	UPUTE ZA UPOTREBU
FI	081	KÄYTTÖOHJE	ZH-TW	195	使用說明
NO	088	BRUKSANVISNING	RU	202	РУКОВОДСТВО ПО ЭКСПЛУАТАЦИИ
PL	096	INSTRUKCJA OBSŁUGI	KK	211	ҚОЛДАНУ НҰСҚАУЛАРЫ
CS	103	NÁVOD K POUŽITÍ	SR	219	UPUTSTVO ZA UPOTREBU
HU	111	HASZNÁLATI UTASÍTÁS	TR	226	KULLANIM TALIMATLARI

RESPIRONICS

AF541 EE Leak 1 Oro-Nasal Mask

Instructions for Use - **EN**

Safety information

Intended Use

This mask is intended to provide an interface for application of CPAP or bi-level therapy to patients. The mask is for single use in the hospital/institutional environment only. The mask is to be used on patients (>40lbs/20kg) for whom CPAP or bi-level therapy has been prescribed.

Note: This mask is not made with natural rubber latex, BPA or DEHP.

Symbols

Manufacturer

Humidity Limits

Not made with natural rubber latex

Temperature Limits

Do Not Reuse

Mask Leak Symbol and Value

Contraindications

This mask is not for use on patients with the following conditions: recent eye surgery or dry eyes, hiatal hernia, excessive reflux, impaired cough reflex, and impaired cardiac sphincter function. This mask should also not be used on patients who need life support ventilation, who are taking a prescription drug that may cause vomiting, or on patients that are uncooperative, unresponsive, or unable to remove the mask by themselves.

⚠ Warnings

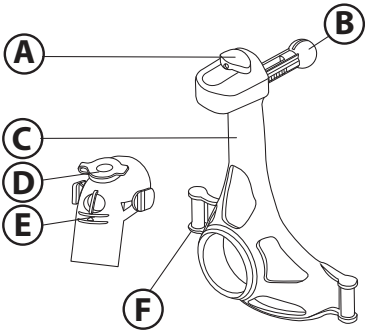
- This mask requires a separate exhalation device.
- Use of this mask while the system is not on and operating may cause rebreathing of exhaled air. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocation.
- Monitor or intervene if the patient experiences skin redness, irritation, discomfort, blurred vision, or drying of the eyes.
- Monitor or intervene if any of the following symptoms occur: Unusual chest discomfort, shortness of breath, stomach distension, belching, severe headache, eye pain or eye infections.
- Monitor or intervene if any of the following symptoms occur: tooth, gum, or jaw soreness. Use of a mask may aggravate a patient's existing dental conditions.
- This mask is for single use only and is not intended to be disinfected or cleaned after patient use. Dispose of the mask after patient use.
- This mask is not designed for multi-patient use. Respirationics cannot guarantee the performance specifications once this mask has been disinfected.
- A minimum of 3cm H₂O (hPa) must be maintained when using this mask.
- Attaching an exhalation device requires therapy pressure level adjustment to compensate for increased leak.

PHILIPS

- Do not overtighten the headgear straps. Watch for signs of overtightening, such as excessive redness, sores or bulging skin around the edges of the mask. Loosen the headgear straps to alleviate symptoms.
- Do not block the anti-asphyxia valve.


Mask Parts and Mask Options

Mask Frame and Elbow


- (A) Forehead Adjuster Button
- (B) Forehead Adjuster Ball
- (C) Mask Frame
- (D) NIVO Nebulizer Port
- (E) Entrainment Elbow with Anti-Asphyxia Valve
- (F) Headgear Clip Post

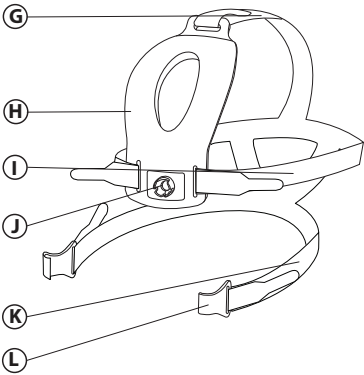CapStrap Headgear


- (G) CapStrap Headgear Crown Strap
- (H) CapStrap Forehead Pad
- (I) Headgear Top Straps
- (J) Headgear Ball Socket
- (K) Headgear Bottom Straps
- (L) Headgear Clips

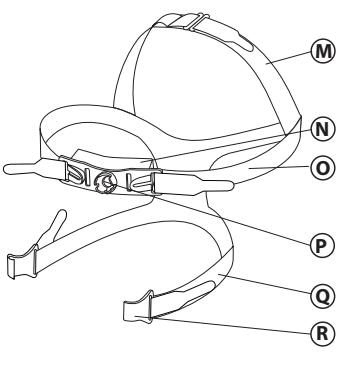Four Point Headgear

- (M) Four Point Headgear Crown Strap
- (N) Four Point Headgear Forehead Pad
- (O) Headgear Top Straps
- (P) Headgear Ball Socket
- (Q) Headgear Bottom Straps
- (R) Headgear Clips

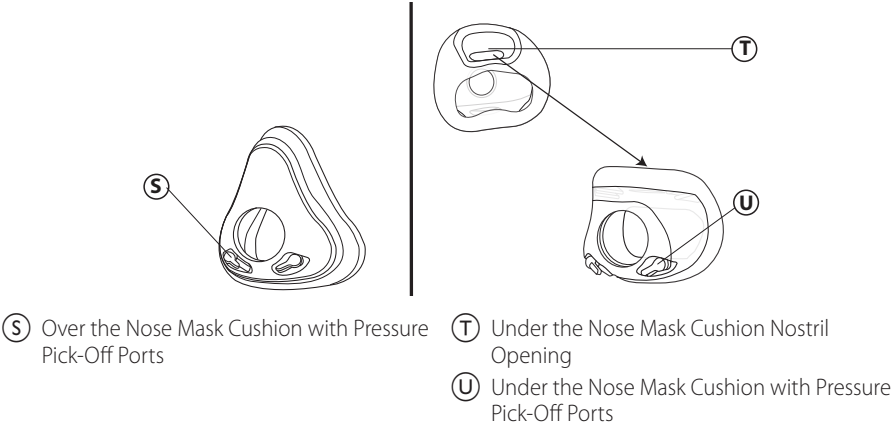Cushion Options

Before Use

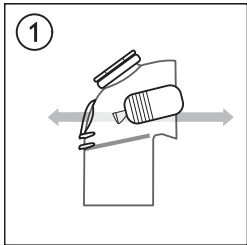
- Wash the patient’s face. Do not use moisturizer/lotion on your hands or the patient’s face.

Safety Checks

1. Visually inspect the anti-asphyxia valve (flapper). The flapper inside the elbow must lay flat when the device is shut off ①. This allows the patient to breathe fresh air. Replace the mask if the flapper does not lay flat when the device is shut off.

⚠Warning: Do not block or seal the anti-asphyxia valve.

2. Inspect the mask for damage or wear (cracking, crazing, tears, etc.). Discard and replace as necessary.


Achieving the Right Fit

Mask Cushion and Headgear Options

The AF541 Oro-Nasal EE Leak 1 Full Face Mask can be used with either the CapStrap Headgear or the Four Point Headgear. The mask can also be used with either the Over the Nose Mask Cushion or the Under the Nose Mask Cushion. Use the following instructions to achieve the right fit with the desired cushion and headgear configurations.

Fitting the Over the Nose Mask Cushion

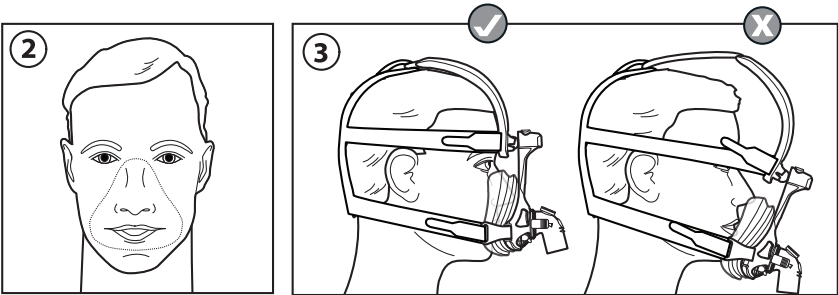
The mask cushion should rest comfortably against the face. The bottom of the mask cushion should rest just above the chin with the mouth slightly open ②③.

Fitting the Under the Nose Mask Cushion

The mask should rest comfortably against the face. The cushion nostril opening goes under the nostrils. The bottom of the mask cushion should rest just above the chin with the mouth slightly open (4)(5).

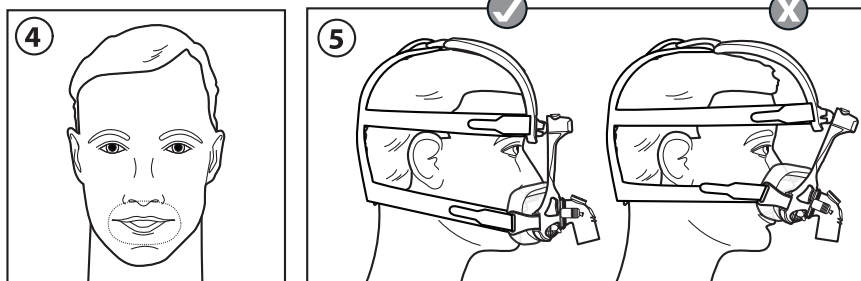ⓘ **Note:** Do not place your nose in the cushion nostril opening.


Using the Mask with the CapStrap Headgear or Four Point Headgear


ⓘ **Note:** Refer to the Fitting Illustrations below based on the headgear used (CapStrap Headgear or Four Point Headgear).

1. Grip and twist the headgear clips away from the headgear clip posts.
2. Loosen the headgear straps as needed.
3. Pull the mask over head (6)(12). Hold the mask against the face.
4. Press the headgear clips onto the headgear clip posts.
5. Pull back the headgear tabs to evenly adjust the crown (7)(13), top (8)(14), and bottom headgear straps (9)(15).
6. Press and hold the forehead adjuster button to slide the frame toward or away from the face (10)(16). This will decrease or increase the pressure on the nose and chin.
7. Connect the mask to the patient circuit disposable exhalation port (DEP).
8. Have the patient lie down. Turn the therapy device on. Instruct the patient to breathe normally and to keep their mouth slightly open (11)(17).
9. Monitor patient comfort. If there are any excessive air leaks, make final adjustments. Some air leaking is normal.

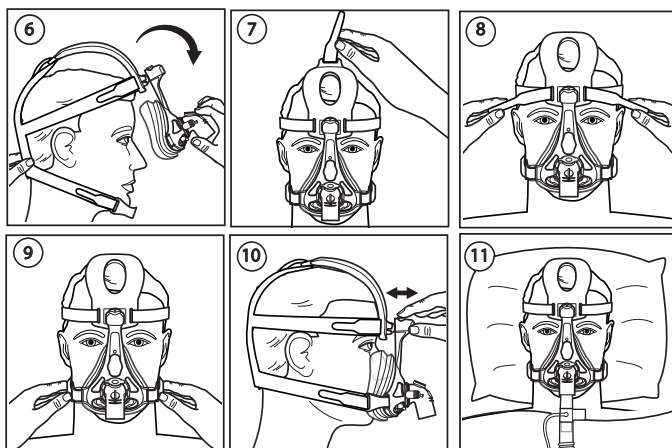CapStrap Headgear Fitting Illustrations

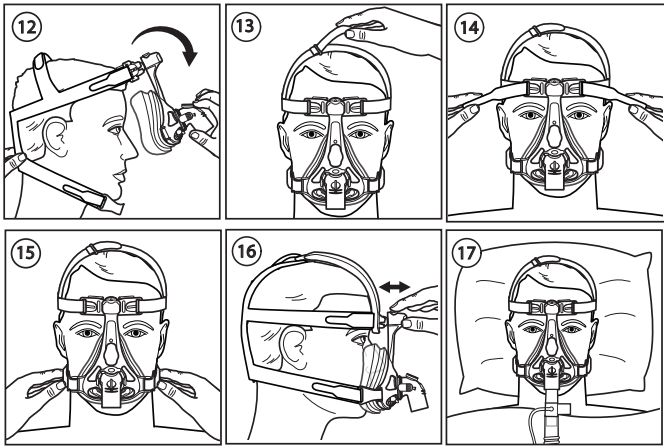Four Point Headgear Fitting Illustrations


Oral Access Without Removing the Headgear

1. Disconnect the headgear clips.
2. Swing the mask upward (18)(19).

Removing the Mask and Headgear

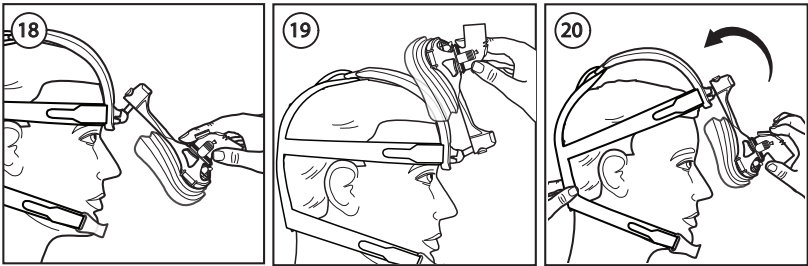1. Turn the headgear clips away from the headgear clip posts.
2. Pull the mask over the top of the head (20).



Using the Clean Clip Shell

The mask may be placed in the Clean Clip Shell when not in use for therapy. The shell helps the mask stay clean when not in use.

1. Attach the shell to the post of the clean clip bracket (optional) on the ventilator stand (21).
2. Place the mask in the Clean Clip Shell. The bottom headgear posts snap into the sides of the Clean Clip Shell. The top of the mask frame rests in the bottom of the Clean Clip Shell (22).
3. After therapy, pull the shell off of the clean clip bracket. Dispose of both the Clean Clip Shell and the mask.

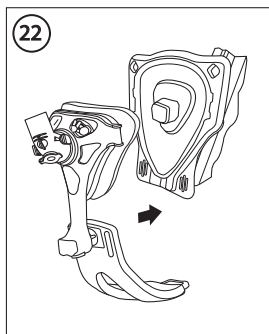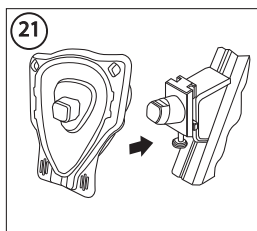Note: Never re-use shell between patients. If any parts become soiled, dispose of shell and mask.

* Comfort Tips

- The headgear should fit loose and comfortable. If the patient's skin bulges around the mask or if you see red marks on the patient's face, loosen the headgear.
- Adjust the headgear lower side straps to manage leaks.
- Set forehead adjustment to manage comfort and mask leak.

Leak Symbol and Port Settings

Some ventilators use a leak value and symbol in the mask selection control. The leak value and symbol represents the intentional leak of the mask. On ventilators equipped with a mask selection control, enter  1.

Disassembly

Removing the Capstrap Headgear

1. Turn the headgear clips away from the headgear clip posts.
2. Pull the CapStrap headgear ball socket away from the forehead adjuster ball (23).

Removing the Four Point Headgear

1. Turn the headgear clips away from the headgear clip posts.
2. Pull the Four Point headgear forehead pad ball socket away from the forehead adjuster ball.

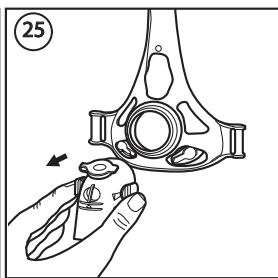
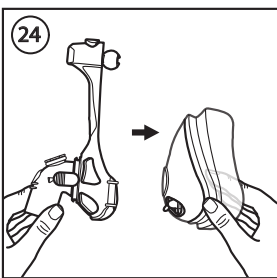
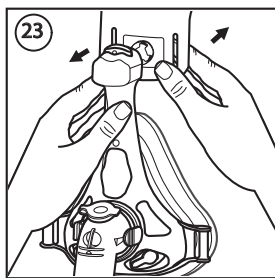
Removing the Mask Cushion

1. Grasp the sides of the mask cushion, and pull it off the mask frame (24).

Note: Do not pull the under the nose mask cushion by the cushion nostril opening.

Removing the Mask Elbow

1. Grasp and squeeze the elbow tabs and pull the elbow away from the mask frame (25).

Assembly

Replacing the Mask Elbow

1. Push the elbow onto the mask frame until the tabs snap into place (audible click).

Replacing the Mask Cushion

1. Grasp the sides of the mask cushion, and push it onto the mask frame (26).

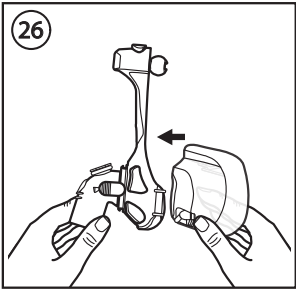
Note: Do not pull the under the nose mask cushion by the cushion nostril opening.

Replacing the CapStrap Headgear

- 1. Push the CapStrap headgear ball socket onto the forehead adjuster ball.
- 2. Press the headgear clips onto the headgear clip posts.

Replacing the Four Point Headgear

- 1. Push the Four Point headgear forehead pad ball socket onto the forehead adjuster ball.
- 2. Press the headgear clips onto the headgear clip posts.


Specifications

The technical specifications of the mask are provided to determine if it is compatible with your therapy device, and that alerts and alarms function properly.

Warning: Use outside of these specifications may result in ineffective therapy. The pressure flow curve shown below is an approximation of expected performance. Exact measurements may vary.

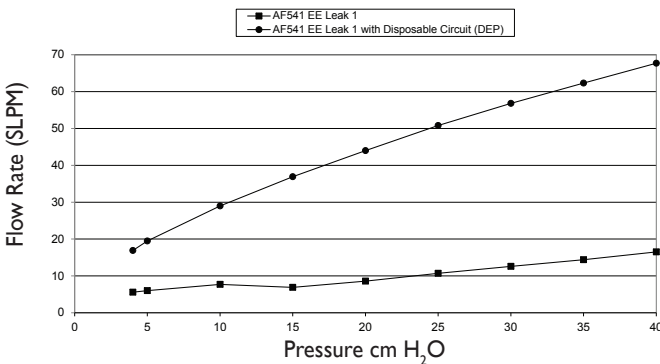Pressure Flow Curve

Resistance with Anti-Asphyxia Valve Closed to Atmosphere
Drop in Pressure at

	50 SLPM	100 SLPM
All sizes	0.3 cm H ₂ O	0.6 cm H ₂ O

Closed to Atmosphere: A minimum of 3 cm H₂O must be maintained to ensure that the anti-asphyxia valve closes.

Open to Atmosphere: In the absence of positive airway pressure (0 cm H₂O), the anti-asphyxia valve will open.

Inspiratory and Expiratory Resistance with Anti-Asphyxia Valve Open to Atmosphere at 50 SLPM

Inspiratory Resistance	1.6 cm H ₂ O
Expiratory Resistance	1.0 cm H ₂ O

Deadspace

Over-the-Nose Mask Cushion

S	277 mL
M	288 mL
L	303 mL
XL	315 mL

Under-the-Nose Mask Cushion

XS	218 mL
A	215 mL
B	213 mL
C	230 mL

Sound Levels at 10 cm H₂O

A-weighted Sound Power Level	18.5 dBA
A-weighted Sound Pressure Level at 1m	11.4 dBA

Disposal

Dispose of in accordance with local regulations.

Storage Conditions

Temperature: -4° F to 140° F (-20° C to 60° C)
Relative Humidity: 15% to 95%, non-condensing

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