Abbott

EDTA CAPILLARY TUBES

REF

7D2227

IVD

LOT

XXXXXXXXXX

YYYY-MM-DD

YYYY-MM-DD

Abbott Diagnostics Medical Co., Ltd.
357 Matsuhidai, Matsudo-shi, Chiba,
270-2214, Japan Tel +81 47 311 5750

EC REP

Abbott Rapid Dx International Limited
Parkmore East Business Park, Ballybrit,
Galway, Ireland, H91 VK7E

UDI

(01)0457122647430
(10)XXXXXXXXXX
(17)231230
(11)221230
(240)7D2227R

UK Responsible Person

Unipath Limited
(trading as Alere International)
Clearblue Innovation Centre, Priority
BusinessPark, Stannard Way, Bedford,
Bedfordshire, England, MK443UP,
United Kingdom

© 2022 Abbott. All rights reserved.

All trademarks referenced are trademarks of either the Abbott group
of companies or their respective owners.

Made in USA

242123/R11

EN

Intended Purpose

EDTA Capillary Tubes are intended for the fingerstick collection of whole blood specimens for use in Determine™ in vitro diagnostic rapid test devices. EDTA Capillary Tubes are specially designed to transfer sufficient volume of blood to ensure that the Determine™ test performs correctly and minimizes blood clotting. EDTA Capillary Tubes are intended for professional use in a laboratory environment and near-patient testing. They are not intended for self-testing.

For full procedure refer the diagnostic assay IFUs:

Determine™ HIV 1/2 (7D23)

Determine™ HIV Early Detect (7D28)

Determine™ HBsAg 2 (7D29)

Determine™ Syphilis TP (7D24)

USER TRAINING, QUALIFICATION AND EXPERIENCE REQUIREMENTS

These EDTA Capillary Tubes are designed for use by professionals in both laboratory and near patient settings including laboratory technologists. Suitable personnel shall have a consummate level of training and experience specimen preparation, the running and interpretation of specimens, management of all supplies and waste disposal as per local regulations and generation of reports.

Caution: Do not eat, drink, smoke or apply make-up while using this product.

Warning: Each pipette is single use only. Do not re-use.

Warning: handle with care to avoid breakage. Broken Capillary Tubes may cause cuts leading to exposure to biohazardous material.

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/746/EU on In vitro Diagnostic Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Contents

EDTA CAPILLARY TUBES 100 x EDTA Capillary Tubes

Storage Instruction

Bring to room temperature before opening the cap and store at 2-30 °C until expiration date.

Store in packaging until use to avoid risk of contamination.

Advice Line:

For further information, please contact your distributor, or call Abbott Technical Specialists:

Europe: Tel: +44 161 483 9032

Email: EME.TechSupport@abbott.com

Glossary of Symbols					
	Contains sufficient for 100 tests		Authorized representative in the European Community		Manufacturer
	Catalogue number		Importer		Unique Device Identification
	Consult instructions for use		Device for Near-Patient Testing		In vitro diagnostic medical device
	Use-by date		Store between 2-30°C		Do not reuse
	CE marking according to IVD Medical Devices Regulation (EU) 2017/746		Do not use if package is damaged		Keep away from sunlight
	Not for Self-Testing		Date of manufacture		Lot Number
		EDTA Capillary Tube			

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