Bioline™ Malaria Ag P.f/Pan

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PREPARATION / PRÉPARATION / PREPARACIÓN / PREPARAÇÃO

1

EN Open the package and look for the following:

1. Test device with desiccant in individual foil pouch
2. Assay diluent
3. Disposable specimen applicator (5 µl) (Capillary pipette or Disposable inverted cup)
4. Sterile lancet
5. Alcohol swab
6. Instructions for use

2

EN First, carefully read the instructions for using Bioline™ Malaria Ag P.f/Pan test kit.

3

EN Next, look at the expiration date on the back of the foil pouch. If the expiration date has passed, use another lot. To avoid false results, ensure that the assay diluent used is from the same kit as the new test device.

4

EN Open the foil pouch and look for the following:

1. Result window
2. Specimen well
3. Assay diluent well

Then, label the device with the patient identifier.

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1

EN Clean the area to be lanced with an alcohol swab.

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PT Limpe a área a ser lancetada com swab de álcool.

2

EN Prick the lateral side of the finger with the sterile lancet provided. Then, safely dispose of the lancet immediately after.

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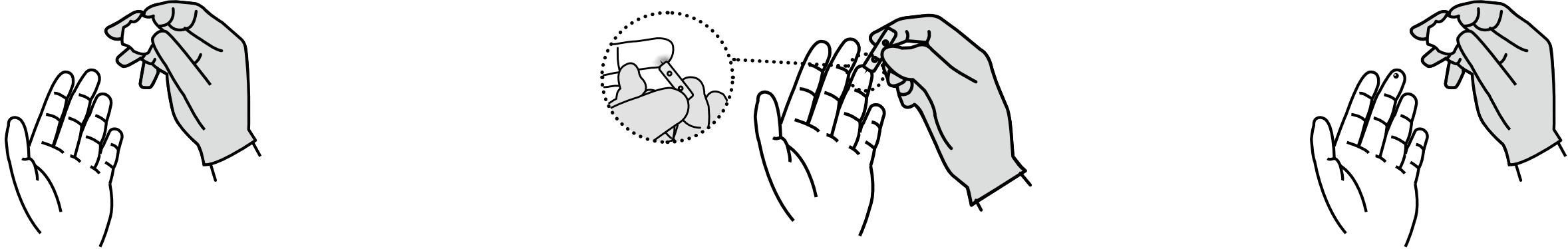
3

EN Wipe away the first drop of blood with sterile gauze or cotton.

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4

EN Using a disposable inverted cup (5 µl) provided, dip the circular end of a inverted cup into the blood specimen.

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5

EN Dispense 5 µl of drawn blood into round specimen well.

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6

EN Hold assay diluent bottle vertically and dispense 4 drops of assay diluent into the assay diluent well. Exactly, 4 drops should be added. Do not let bottle tip touch device in order to avoid cross-contamination.

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6

EN Read the test result after 15 minutes.

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6

EN Use Inverted cup : Let the circular end of the inverted cup touch the pad, then press down lightly.

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P.f

POSITIVE / POSITIF / POSTIVO

⚠ Caution : The presence of any test line, no matter how faint, the result is considered positive.

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EN The presence of two colored lines ("P.f" test line and "C" control line) or three colored lines ("P.f", "Pan" test lines and "C" control line) within the result window indicate P.f positive result.

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EN The presence of two colored lines ("Pan" test line and "C" control line) within the result window, regardless of which line appears first, indicate Pan (P.v or P.m or P.o) positive result.

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EN The presence of three colored lines ("P.f", "Pan" test lines and "C" control line) within the result window, regardless of which line appears first, indicate P.f positive or mixed infection of P.f and P.v or P.m or P.o.

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EN If the Control Line "C" is not visible within the result window after performing the test, the result is considered invalid. Instructions may not have been followed correctly or the test may have deteriorated beyond the expiration date. It is recommended that the specimen be retested using a new test device.

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