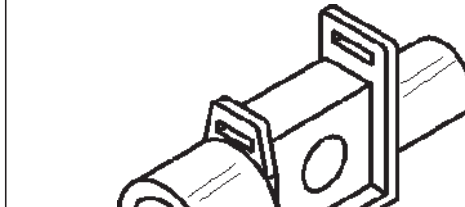
RESPIRONICS

REF 7007-01

REF 7053-01

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Intended use/Indications for use

7007-01: Pediatric/Adult Airway Adapter
For use on pediatric and adult patients in a variety of clinical settings including, but not limited to, the Intensive Care Unit (ICU) and the Operating Room (OR). The Pediatric/Adult Airway Adapter is for use on patients with ET tube sizes greater than 4.0 mm.

7053-01: Neonatal Airway Adapter

For use on intubated neonatal patients in a variety of clinical settings including, but not limited to, the Neonatal Intensive Care Unit (NICU) and the Operating Room (OR). The Neonatal Airway Adapter is for use on patients with ET tube sizes 4.0 mm and smaller.

Warnings
– Inspect the CO2 airway adapter for damage prior to use and after reprocessing. Do not use the adapter if it appears to be damaged or broken.
– Periodically check the airway adapter and replace if clogged by secretions or if the CO2 waveform changes.

Instructions for Use
1. Reusable airways adapters are compatible only with the CAPNOSTAT CO2 Sensor.
2. Press the CAPNOSTAT CO2 sensor onto the airway adapter. It will “click” into place when properly sealed.

Cautions

– Clean, disinfect and sterilize airway adapters before reuse. Handle per institutional protocols for contaminated devices.
– Reusable airway adapters are not compatible with automated cleaning or disinfection methods. Sterilize per the instructions below.
– To avoid damaging the windows, do not insert any object into the airway adapter.
– Before reusing an adapter, ensure the windows are dry and residue free and that the adapter has not been damaged during the cleaning, disinfection and sterilization processes.

Cleaning, disinfection and sterilization - maximum 100 cycles
(See also: Reprocessing Guide PN 1154848 at www.philips.com/IFU)

Manual cleaning instructions

1. In a clean sink or basin, add sufficient warm tap water (30–40°C) to fully immerse the adapter.
2. Soak for 5 minutes fully submerged, with no bubbles on inner surfaces.
3. After five (5) minutes of soaking, agitate the adapter by moving it back and forth under the water, allowing the water to flow through the device for one (1) minute. **Note:** In case of excess sputum, immerse a sterile, needle-less syringe in the water with the adapter. Use it to flush surfaces such as seams, grooves, and narrow spaces.
4. While the adapter is immersed, place a non-linting wipe in the water and gently wipe external surfaces. Ensure that all surfaces are wiped at least one (1) time. Pay attention to surfaces such as seams, grooves, and narrow spaces during the wiping. Avoid wiping the optical windows. If wipes become visibly soiled, use additional wipes.
5. Rinse the adapter under room temperature, running tap water for at least 30 seconds, paying special attention to seams, grooves, and narrow spaces.

Manual disinfection instructions

Note: Follow manufacturers instructions for use when using the approved disinfecting agents.

6. Fill a tray or basin with one of the disinfecting agents listed below. Fully immerse the adapter for the required Time shown below.

Approved disinfectants	Time
Ready-to-use Cidex OPA Solution (0.55% ortho-phthalaldehyde) at room temperature (25°C ±5°C / 68-86°F)	12 minutes
10% aqueous solution of 5% chlorine bleach	20 minutes
Glutaraldehyde 2.5% solution with Metcicide	90 minutes

7. Donato the adapter is immersed in disinfecting agent, popotene sterilia sprinčavica bez igla. Iskoristavajte je za pranje i/ili za dezinfekciju u čistim i tesnim prostorima s dezinfekantom.
8. Rinse thoroughly for at least 30 minutes by immersing the adapter in a large volume of distilled or deionized water. While the device is immersed, use a sterile, needle-less syringe to flush inner surfaces and narrow spaces. Do not reuse the device.
9. Repeat the rinse step two more times, using a large volume of fresh distilled or deionized water each time, for a total of three rinse cycles.

10. Gently dry the adapters with clean, non-linting wipes then allow to continue to air dry at room temperature. Avoid wiping the optical windows.

Sterilization instructions

Clean and disinfect airway adapters before sterilizing them with one of the methods below. Segregate and label parts after sterilization. **NOTE: Neonatal reusable airway adapters are not intended for use with steam sterilization.**

Method	Exposure Time	Exposure Temp.	Minimum Dry Time
Gravity Displacement (ETO)	3 hours	38°C - 39°C with 12 hours aeration	N/A
3.4% Glutaraldehyde	10 hours	20°C	Air dry, visual inspection
PeraSafe	10 hours	20°C	Air dry, visual inspection

Disposeal

Dispose of in accordance with all applicable Federal, State, and local Medical/Hazardous waste practices.

NOTE: Any serious incident that has occurred in relation to this product should be reported to Respironics Novametric and the competent authority of the country in which the user and/or patient is established.

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