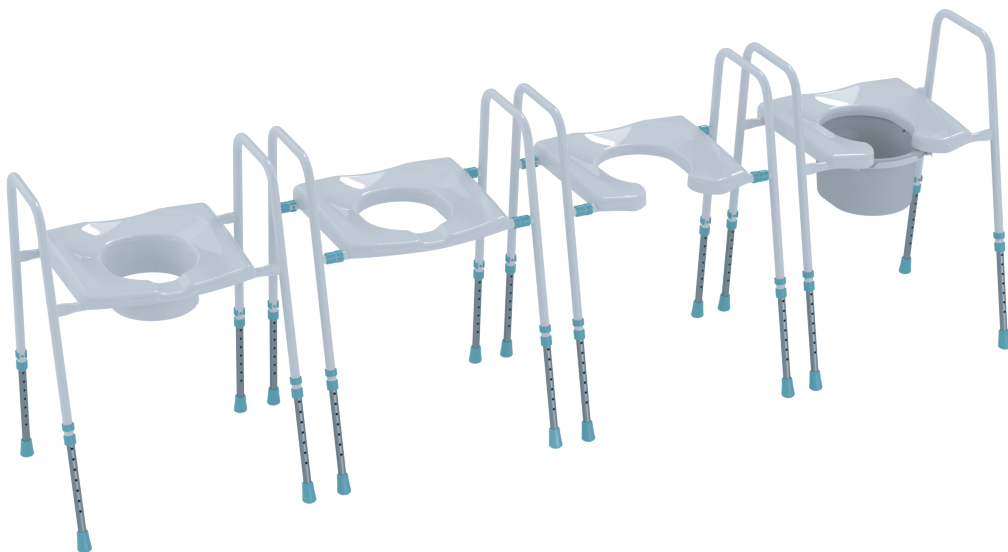
**GORDON
ELLIS
& CO**

PRIMA[®]

TOILET AIDS AND COMMODES CLASSIC, SUPERIOR & HORSESHOE

Welcome to your new Gordon Ellis product.
We hope that it proves a useful aid in your daily life.

Your PRIMA[®] TOILET AIDS AND COMMODES were specifically developed by our UK Design Team with the intention of compensating for an injury, or overcoming a temporary or permanent disability. Hopefully making daily living that little bit easier.

GORDON ELLIS & CO.
TRENT LANE
CASTLE DONINGTON
DERBY. ENGLAND DE74 2AT

AUTHORISED REPRESENTATIVE
ADVENA LTD, TOWER BUSINESS
CENTRE, 2ND FLOOR, TOWER STREET,
SWATAR, BKR 4013 MALTA

MD

IFU-REF 68000/1020-3

PART REF

68000, 68004, 68005, 68010, 68020/H, 68025/H, 68001/FP, 68003/FP, 68001/FP/B,
68001/FP/R 68021/H/FP, 68023/H/FP, 6832, 6833, 6835/FP, 6837/H, 6839/H, 6840/H/FP

MADE IN CHINA

VIEW THE ENTIRE RANGE ONLINE AT
GORDONELLISHEALTHCARE.CO.UK

[gordonellis.healthcare](https://www.gordonellis.healthcare)

@EllisHealthcare

Instructions

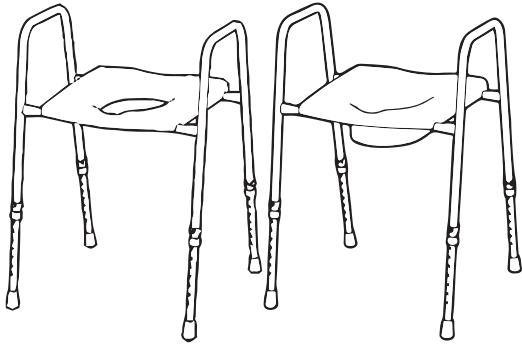
Part number:

Classic


68000, 6832

Superior



68010, 6834

A

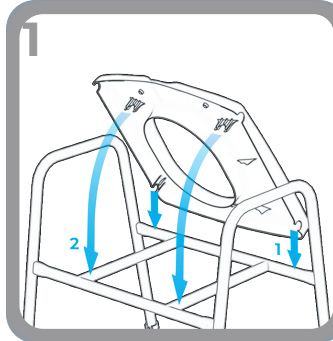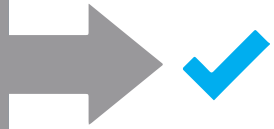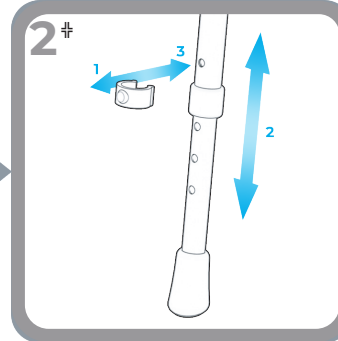
† **NOTE:** ONE LEG FEATURES EXTRA HOLES FOR FINER HEIGHT ADJUSTMENT AND GREATER STABILITY.

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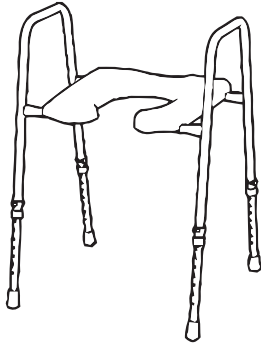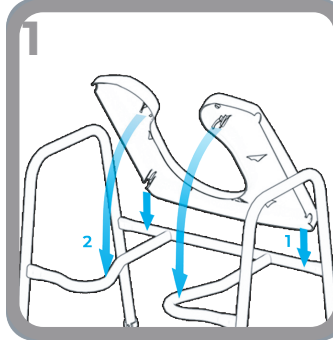
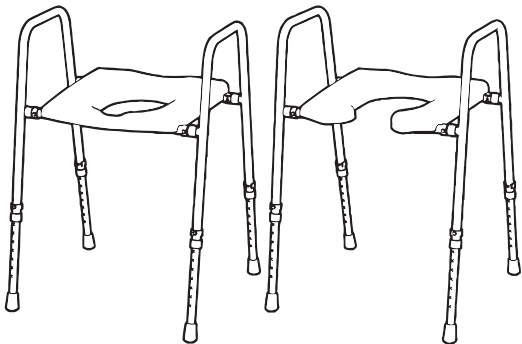
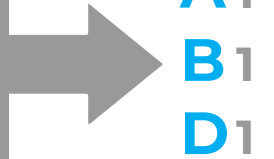
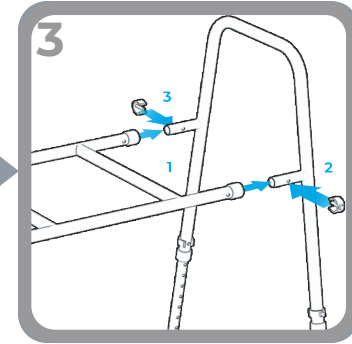
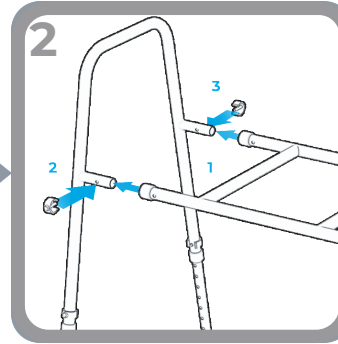
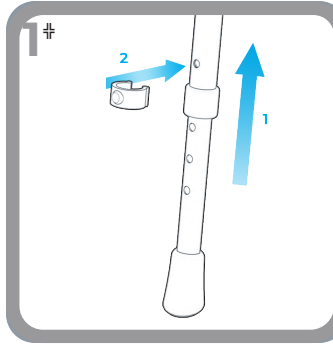
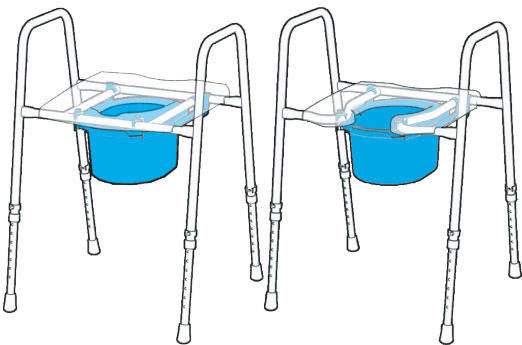
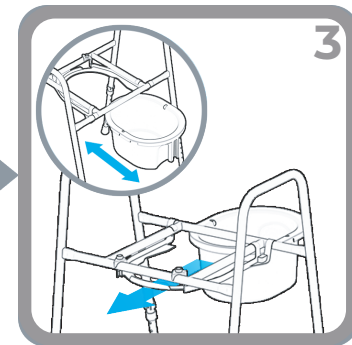
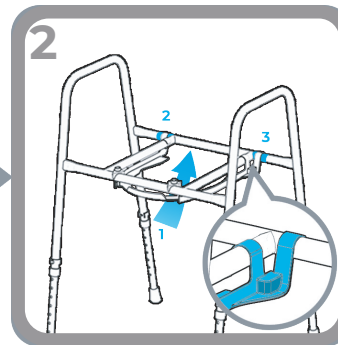
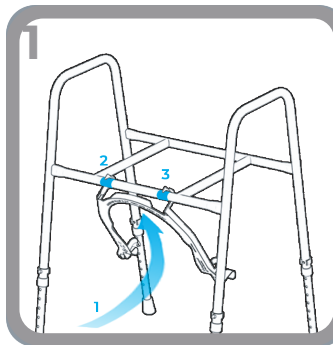
EN

1. PUSH THE SEAT ON TO THE FRAME UNTIL IT CLIPS SECURELY (FIG:A1/B1) 2. THE SEAT HEIGHT IS ADJUSTABLE TO SUIT THE USER. ADJUST BY REMOVING THE CLIPS THAT HOLD THE LEGS AND SLIDING THE ADJUSTABLE LEGS UP OR DOWN (FIG:A2/C1). SECURE BY REFITTING THE CLIPS. NOTE: ONE LEG FEATURES EXTRA HOLES FOR FINER HEIGHT ADJUSTMENT AND GREATER STABILITY. 3. FOR TOILET AID: POSITION THE TOILET AID OVER THE TOILET BOWL AND CHECK THAT IT IS STABLE TO USE (FIG:A3). 4. FOR COMMODE: ATTACH BRACKET (FIG:D1+D2), SLIDE PAN ONTO BRACKET LEDGE (FIG:D3). IT IS THE RESPONSIBILITY OF THE USER/ OWNER TO CARRY OUT AN INITIAL AND REGULAR LOCAL RISK ASSESSMENTS. CHECK FOR STABILITY BEFORE USE. FOR STATIC USE ONLY. ENSURE E-CLIPS ARE FULLY PROTRUDING THROUGH ADJUSTMENT HOLES. ENSURE ALL LEGS ARE RAISED TO THE SAME HEIGHT.

A = MAXIMUM USER WEIGHT.
B = WIPE WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.

ANY SERIOUS ACCIDENT OCCURRING THROUGH USE SHOULD BE REPORTED TO THE EC REP (EU CUSTOMERS) OR THE MANUFACTURER.

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