
WarmTouch™

Pediatric Surgical Blanket


en	Instructions For Use
fr	Mode d'emploi
de	Gebrauchsanweisung
nl	Gebruiksaanwijzing
it	Istruzioni per l'uso
es	Instrucciones de uso
sv	Bruksanvisning
da	Brugsanvisning
no	Bruksanvisning
fi	Käyttöohjeet
pt	Instruções de uso
ru	Инструкция по применению
zh	使用说明
pl	Instrukcja użytkowania
cs	Návod k použití
sk	Návod na použitie
sl	Navodila za uporabo
hu	Használati útmutató
el	Οδηγίες χρήσης
tr	Kullanma Talimatı
ar	إرشادات الاستعمال
bg	Указания за употреба
ro	Instrucțiuni de folosire
et	Kasutusjuhend
lt	Naudojimo instrukcija
lv	Lietošanas norādījumi
hr	Upute za uporabu
sr	Uputstva za upotrebu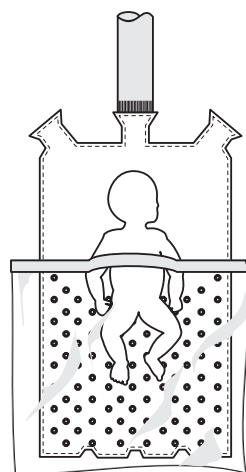


Blanket under patient
 Couverture en-dessous du patient
 Decke unter dem Patienten
 Dekken onder patiënt
 Coperta sotto il paziente
 Manta debajo del paciente
 Filt under patient
 Tæppe under patienten
 Teppe under patienten
 Peite potilaan alle
 Manta sob o paciente
 Одеяло под телом пациента

患者身下用毯
 Koc pod tułów
 Pokrývka pod pacienta
 Prikrývka na umiestnenie pod pacienta
 Pregrinjalo, položeno pod bolnika
 Páciens alatti takaró
 Κουβέρτα κάτω από τον ασθενή
 Örtü hastanın altında
 الغطاء تحت المريض
 Одеяло под пациента


Pătură sub pacient
 Lina patsiendi all
 Apklotas po pacientu
 Sega zem pacienta
 Prekrivač pod pacientom
 Čebe ispod pacijenta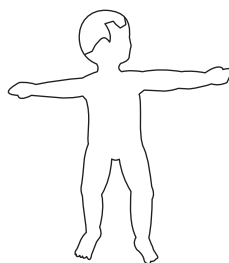


1

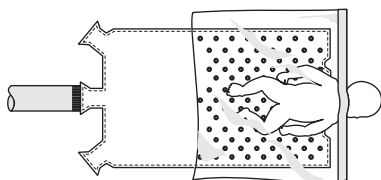
4

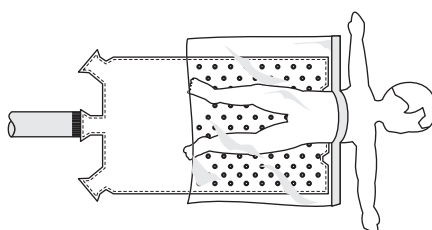
2

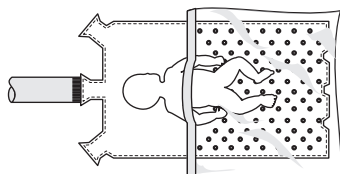5

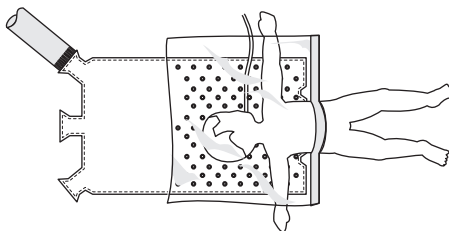3

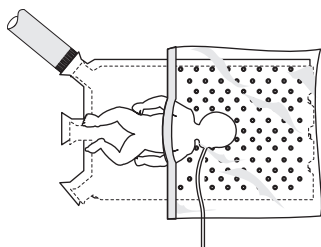
Blanket over patient
 Couverture au-dessus du patient
 Decke auf dem Patienten
 Deken over patiënt
 Coperta sopra il paziente
 Manta sobre el paciente
 Filt över patient
 Tæppe over patienten
 Teppe over pasienten
 Peite potilaan päälle
 Manta sobre o paciente
 Одеяло на теле пациента
 患者身上用毯
 Koc na tułów
 Pokrývka přes pacienta
 Prikrývka na umiestnenie na pacienta
 Pregrinjalo, položeno na bolnika

Páciens feletti takaró
 Κουβέρτα επάνω από τον ασθενή
 Örtü hastanın üstünde
 لغطاء فوق المريض
 Одеяло над пациента
 Pătură peste pacient
 Lina patsiendi peal
 Apklotas virš paciento
 Segá uz pacienta
 Prekrivač na pacijentu
 Čebe preko pacijenta


6


7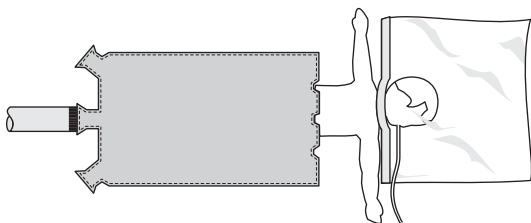

8

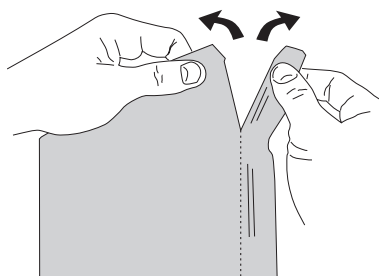9

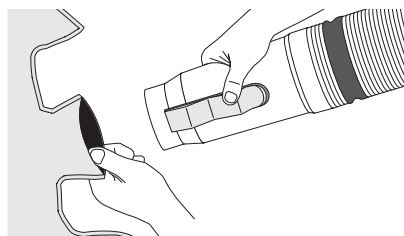Page left intentionally blank

WarmTouch™

Pediatric Surgical Blanket

Blanket for use with WarmTouch™ warming units only

Identification of a substance that is not contained or present within the product or packaging.

No Free Hosing

Must consult instructions for use

This product cannot be adequately cleaned and/or sterilized by the user in order to facilitate safe reuse, and is therefore intended for single use. Attempts to clean or sterilize these devices may result in a bio-incompatibility, infection or product failure risks to the patient.

The WarmTouch™ Pediatric Surgical Blanket is to be used only with WarmTouch™ convective warming units with model numbers 5015300A, 5015800, 5015900 and 5016000.

Directions for Use

Indications

The WarmTouch™ Pediatric Surgical Blanket is a versatile system consisting of an inflated warming blanket and a clear plastic drape with tape. The blanket can be used in various configurations under or over a patient to help regulate patient temperature during surgery. The blanket has both perforated and non-perforated areas. Warm air passes through tiny holes in the perforated area to warm the patient directly. In the non-perforated area, warmth is transferred conductively through contact with the patient's skin. The clear plastic drape is used to trap warm air from the perforated area over and around the patient and to help restrict the flow of air into the surgical site.

Contraindications

None known.

WARNINGS

- **Possible explosion hazard. Do not use the device in the presence of flammable anesthetics or in an oxygen-rich environment.**
- **No free-hosing. Keep hose nozzle connected to the WarmTouch blanket at all times or thermal injury may occur.**
- **Possible fire hazard. Prevent the blanket material from coming into contact with a laser or an electrosurgical active electrode, or combustion may result.**
- **Possible burn or infection hazard. Do not allow blanket to come into contact with open wounds or mucous membranes. All patients' wounds should be covered while using the warming system.**
- **Possible patient burns. Use caution and consider discontinuing use on patients during vascular surgery when an artery to an extremity is clamped. Do not apply the warming system to ischemic limbs. Use caution and monitor closely if used on patients with severe peripheral vascular disease.**
- **Do not cover the patient's face with the head cover unless the patient is intubated and ventilated.**
- **The WarmTouch Blanket should not be used on patients who exhibit allergic reactions to adhesive tape.**
- **Use only WarmTouch blankets with the WarmTouch Convective Warming System. The performance of the WarmTouch Convective Warming System has only been evaluated and validated using WarmTouch blankets.**
- **Using the warming system on patients with transdermal medication patches may increase the rate of drug delivery, potentially causing harm to the patient.**
- **If a fault or sudden change in performance occurs in the warming system, discontinue use. Notify your sales/ service center. The unit must be serviced according to the Service Manual by qualified personnel.**
- **Use caution to avoid suffocation when patient is positioned face down on blanket. Do not leave patient unattended.**
- **Do not open more than one hose access inlet when connecting the nozzle to the blanket.**
- **Restrain the patient while the blanket is inflated under the patient, using accepted methods for intraoperative restraint during anesthesia.**
- **Monitor the patient for any movement or displacement while the blanket is inflated under the patient.**

CAUTIONS

- Federal (U.S.A.) law restricts the use of the warming system to sale by or on the order of a physician.
- Ensure the patient is dry or the warming system may be ineffective.
- Use WarmTouch blankets only as directed. Carefully follow these instructions regarding handling and positioning of the blanket.
- WarmTouch Pediatric Surgical Blanket is a single-patient non-sterile item.
- When handling blanket in the vicinity of the sterile field, observe proper sterile technique.
- Do not allow objects that are heavy enough to constrict airflow (such as cables) to drape over the blanket at the blower inlet.
- Do not allow sharp objects to come into contact with the blanket, as they may puncture or tear it.
- As blanket initially inflates and throughout duration of application, check that the blanket fills evenly over the entire blanket surface and replace if cuts or tears are present.

- Ensure that other blanket inlets besides the one selected for use remain unopened, to prevent warmed air from escaping the blanket.
- Continuously monitor the patient's temperature and vital signs. Reduce air temperature or discontinue therapy when normothermia is reached. Use good clinical judgment in selecting and adjusting temperature settings based on the patient's warming needs and response to treatment.
- If hypotension occurs, consider reducing air temperature or turning off the warming system.
- The WarmTouch Convective Warming System uses an air filter; however, the risk of airborne contamination should not be overlooked when the WarmTouch Convective Warming System is used. Refer to the WarmTouch Convective Warming System Service Manual for filter replacement.
- WarmTouch blankets are for use by trained personnel only.


Instructions for Use

For maximum effectiveness, use the blanket and plastic drape to cover as much skin surface as possible, while at the same time maintaining access to the surgical site and restricting airflow into the surgical site.

1. Always place blanket with perforated side toward patient.
2. Refer to Figures 1-7 for various draping options. **Do not cover head with plastic cover if patient is not intubated and ventilated.**

Note: Do not position surgical site over perforations.

3. Select only one of the three available inlets and tear inlet on perforation line across notches to open. (8)

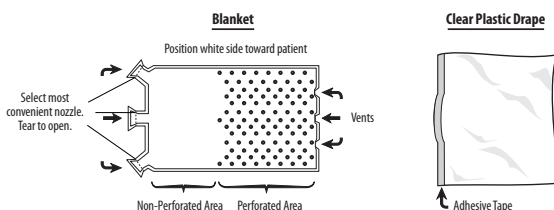4. Insert the WarmTouch Patient Warming System nozzle into the blanket inlet. Ensure that the inlet is clamped under the nozzle clip. (9)
5. Connect the WarmTouch Patient Warming System nozzle blanket clip to the drape or bed sheet.
6. Turn on the WarmTouch Patient Warming System. Refer to the WarmTouch Patient Warming System Operator's Manual.
7. Select the appropriate air temperature setting on the WarmTouch unit. Refer to the WarmTouch System Operator's Manual.
8. Continually monitor the patient's core temperature while the WarmTouch Patient Warming System is in use.

NOTE: The WarmTouch blanket may be used as long as normothermia or recovery from hypothermia can be functionally achieved.

Disposal

Dispose of the blanket after single use. Disposal of the blanket should be made in accordance with applicable national regulations for biological hazardous waste.

Storage and Shipping Conditions

Temperature -40°C to 70°C

Relative Humidity 10% to 95%

Atmospheric Pressure 120hPa to 1060hPa

Additional Copies of Instructions

Additional copies of these instructions can be obtained through your authorized distributor. In addition, permission is hereby granted under Covidien copyrights to purchasers of products obtained from your authorized distributor to make copies of these instructions for use by such purchasers.

Warranty

To obtain information about a warranty, if any, for this product, contact Covidien's Technical Services Department or your local Covidien representative.

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registered trademarks of Covidien AG.
Other brands are trademarks of a
Covidien Inc, 15 Hampshire Street,
Covidien Ireland Limited,