

WarmTouch™

Cardiac Blanket


EN US	Instructions For Use
FR	Mode d'emploi
DE	Gebrauchsanweisung
NL	Gebruiksaanwijzing
IT	Istruzioni per l'uso
ES	Instrucciones de uso
SV	Bruksanvisning
DA	Brugsanvisning
NO	Bruksanvisning
FI	Käyttöohjeet
PT	Instruções de uso
RU	Инструкция по применению
PL	Instrukcja użytkowania
CS	Návod k použití
SK	Návod na použitie
SL	Navodila za uporabo
HU	Használati útmutató
EL	Οδηγίες χρήσης
TR	Kullanma Talimatı
BG	Указания за употреба
RO	Instrucțiuni de folosire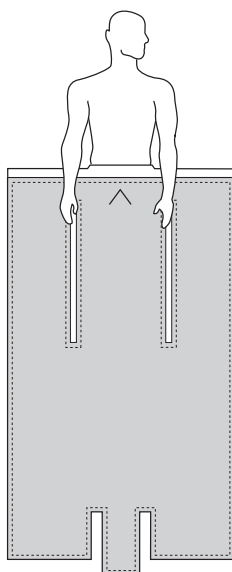


1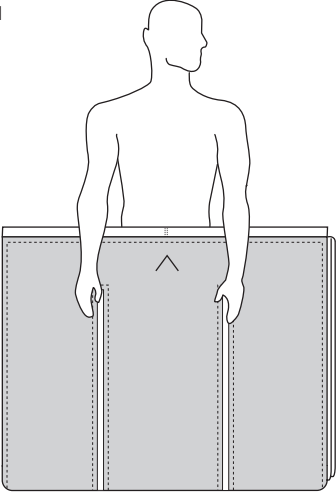


2

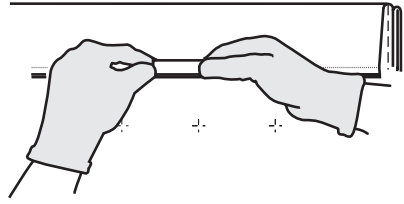
3

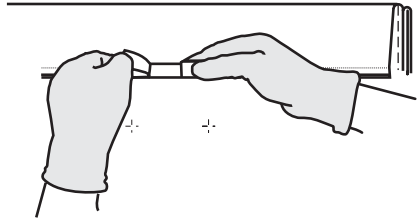
4

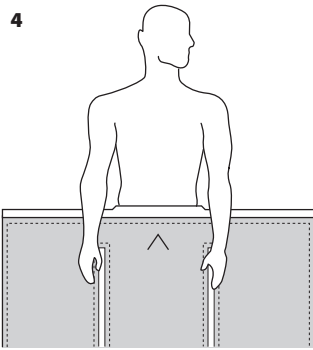
5

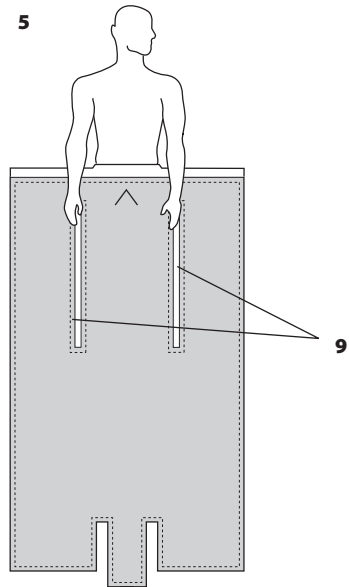
6

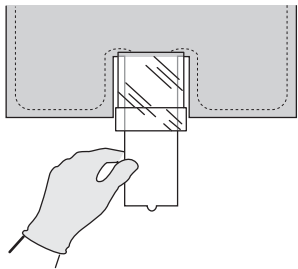7

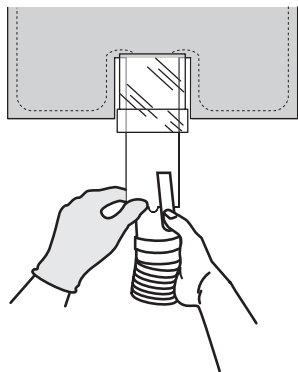8

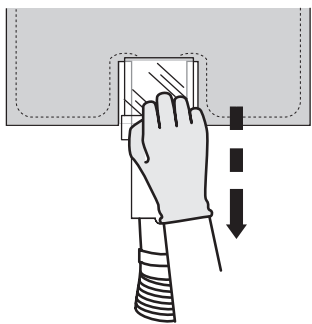WarmTouch™

Cardiac Blanket

Identification of a substance that is not contained or present within the product or packaging.

No Free Hosing

This product cannot be adequately cleaned and/or sterilized by the user in order to facilitate safe reuse, and is therefore intended for single use. Attempts to clean or sterilize these devices may result in a bio-incompatibility, infection or product failure risks to the patient.

Directions for Use

Indications

The WarmTouch™ Patient Warming System (warming unit and blanket) is intended for prevention and treatment of hypothermia, for example, with the surgical patient, the patient in the pre-operative holding area, the pregnant woman who shivers during epidural anesthesia due to hypothermia, or any patient who is uncomfortable anywhere in the cold critical care environment.

The blanket has a lower layer that is perforated. Warm air, generated by the WarmTouch blower, passes through the tiny holes in the perforated layer to warm the patient. Along the top edge of the blanket are three lengths of tape that are used to secure the blanket to the patient and the operating table and to help restrict the flow of air into the surgical site. This sterile blanket has an air inlet that can be extended outside of the sterile field. The extended inlet includes an inlet/nozzle/hose shield that can be used to cover the nonsterile nozzle and hose of the warming unit. Perforated slits are provided over each leg to provide access to the patient's groin and/or leg area.

Instructions for Use

1. Make sure the patient is dry, to ensure maximum effectiveness of the WarmTouch System.
2. Clean the patient's abdominal skin to remove oils or debris to maximize adhesion of the tape to the patient.
3. Position the WarmTouch warming unit at the foot of the patient (under the foot of the operating table, outside of the sterile area) and plug the warming unit into an approved AC power outlet.
4. Remove the Cardiac Blanket from its sterile pouch placing the contents in the sterile field using appropriate sterile techniques. Do NOT use if the sterile package has already been opened or damaged.
5. Remove the sterile wrap from the Cardiac Blanket.
6. Before unfolding, place the Cardiac Blanket over the patient's abdomen with the blanket's perforated side next to the skin. The arrow on the blanket should point toward the patient's head. Arrow indicates center of blanket. **(1)**
7. To remove the tape liner, fold the tape over at the blanket edge. The tape should be folded towards the blanket with liner visible. Roll the tape at the blanket edge. **(2)** Rolling the tape should separate some of the tape liner from the tape. Grip the exposed end of the liner and pull the liner along the edge of the blanket, removing the liner. **(3)**
8. Peel back the tape liner on the left and right tapes and apply the tape to the patient and/or the operating table as appropriate with the arrow on the left-right centerline. **(4)**
9. Unfold the blanket down and out along the length of the patient. **(5)**
10. Fully extend the WarmTouch blanket inlet. **(6)**

Note: Use care when connecting the WarmTouch unit hose to the Cardiac Blanket. The Cardiac Blanket is sterile; however, the WarmTouch warming unit and hose are nonsterile. The tab on the blanket inlet opening must be placed to the right or left of the nozzle clip for maximum penetration of the nozzle into the blanket inlet.

11. Connect the air hose from the warming unit to the Cardiac Blanket inlet. A person in the sterile field should hold the top tab of the blanket inlet. A person outside the sterile field should hold the air hose nozzle, press the nozzle clip, and slip the nozzle into the blanket inlet opening releasing the nozzle clip. **(7)**
12. Pull the blanket plastic hose cover down over the nozzle and hose. **(8)**
13. Turn the WarmTouch System warming unit on. Refer to the WarmTouch System Operator's Manual.
14. Select the appropriate air temperature setting on the WarmTouch unit. Refer to the WarmTouch System Operator's Manual.
15. Perforated slits running along the patient's legs may be pulled open as appropriate for access to the groin area and/or legs. **(9)**
16. Continually monitor patient's temperature while the WarmTouch Patient Warming System is in use.

DANGER: Possible explosion hazard. Do not use the WarmTouch system near flammable anesthetics.

DANGER: Avoid contact of a laser or an electrosurgical active electrode with blanket material or rapid combustion could result.

WARNINGS

- Do not apply heat directly to open wounds. All patient wounds that will be under the blanket should be covered before using the WarmTouch Patient Warming System.
- If a malfunction occurs in the WarmTouch Patient Warming System warming unit, immediately turn off the unit.
- The Cardiac Blanket must be connected to the WarmTouch hose nozzle. Do not direct air from the hose to the patient (free-hosing) or thermal injury may occur. ⚠
- Use only WarmTouch blankets with the WarmTouch Patient Warming System. The performance of the WarmTouch Patient Warming System has only been evaluated and validated using WarmTouch blankets.
- The WarmTouch Patient Warming System blanket should not be used on patients who exhibit allergic reactions to adhesive tape.

CAUTIONS

- Make sure the patient is dry, or the WarmTouch System may be ineffective.
- Continuously monitor the patient's temperature and vital signs. Reduce air temperature or discontinue therapy when normothermia is reached.
- The WarmTouch Patient Warming System uses an air filter, however, the risk of airborne contamination should not be overlooked when the WarmTouch Patient Warming System is used. Refer to the WarmTouch Patient Warming System Service Manual for the filter replacement.
- Use caution and consider discontinuing use during vascular surgery when an artery to an extremity is clamped. Do not apply the WarmTouch Patient Warming System to ischemic limbs. Use caution and monitor closely if used on patients with severe peripheral vascular disease.
- If hypotension occurs, consider reducing air temperature or turning off the WarmTouch Patient Warming System.
- Do not allow objects such as cables to drape over the blanket at the blower inlet that are heavy enough to constrict airflow at the nozzle inlet.
- As blanket initially inflates check that the blanket fills evenly over the entire blanket surface. Do not use if cuts or tears are present which prevent even inflation.

Disposal

Dispose of the blanket after single use. Disposal of the blanket should be made in accordance with applicable national regulations for biologically hazardous waste.

Additional Copies of Instructions

Additional copies of these instructions can be obtained through your authorized distributor. In addition, permission is hereby granted under Covidien copyrights to purchasers of products obtained from your authorized distributor to make copies of these instructions for use by such purchasers.

Warranty

To obtain information about a warranty, if any, for this product, contact Covidien's Technical Services Department, or your local Covidien representative.

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COVIDIEN, COVIDIEN with logo and Covidien logo are U.S. and/or internationally registered trademarks of Covidien AG. Other brands are trademarks of a Covidien company.
Covidien Inc, 15 Hampshire Street,
Covidien Ireland Limited,