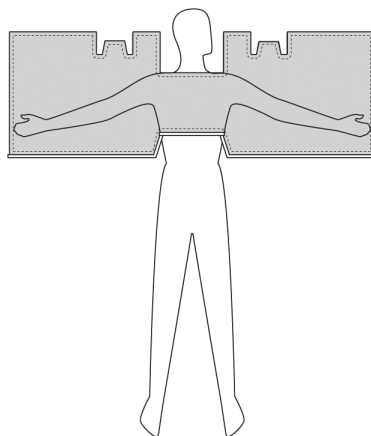
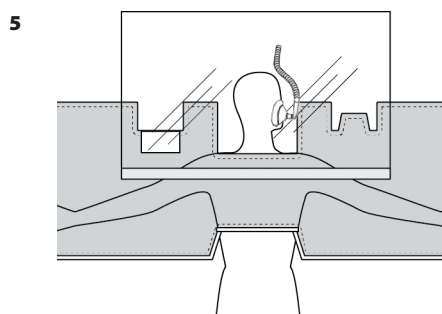
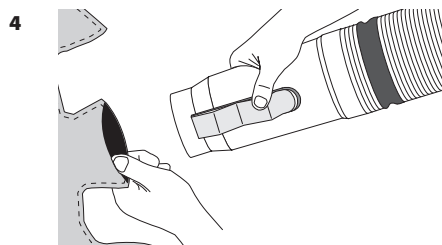
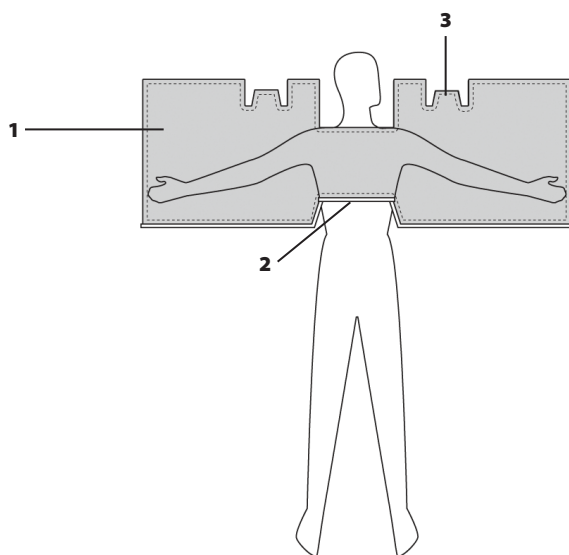
WarmTouch™

Upper Body Blanket

en	Instructions For Use
fr	Mode d'emploi
de	Gebrauchsanweisung
nl	Gebruiksaanwijzing
it	Istruzioni per l'uso
es	Instrucciones de uso
sv	Bruksanvisning
da	Brugsanvisning
no	Bruksanvisning
fi	Käyttöohjeet
pt	Instruções de uso
ru	Инструкция по применению
zh	使用说明
pl	Instrukcja użytkowania
cs	Návod k použití
sk	Návod na použitie
sl	Navodila za uporabo
hu	Használati útmutató
el	Οδηγίες χρήσης
tr	Kullanma Talimatı
ar	إرشادات الاستعمال
bg	Указания за употреба
ro	Instrucțiuni de folosire
et	Kasutusjuhend
lt	Naudojimo instrukcija
lv	Lietošanas norādījumi
hr	Upute za uporabu
sr	Uputstva za upotrebu

WarmTouch™

Upper Body Blanket

Identification of a substance that is not contained or present within the product or packaging

No Free Hosing

Must consult instructions for use

This product cannot be adequately cleaned and/or sterilized by the user in order to facilitate safe reuse, and is therefore intended for single use. Attempts to clean or sterilize these devices may result in a bio-incompatibility, infection or product failure risks to the patient.

The WarmTouch™ Upper Body Blanket is to be used only with WarmTouch™ convective warming units with model numbers 5015300A, 5015800, 5015900 and 5016000.

Directions for Use

Indications

The WarmTouch™ Patient Warming System (warming unit and blanket) is intended for prevention and treatment of hypothermia. For example, the surgical patient, the patient in the pre-operative holding area, the pregnant woman who shivers during epidural anesthesia due to hypothermia, or any patient who is uncomfortable anywhere in the cold critical care environment.

Contraindications

None known.

WARNINGS

- **Possible explosion hazard. Do not use the device in the presence of flammable anesthetics or in an oxygen-rich environment.**
- **No free-hosing. Keep hose nozzle connected to the WarmTouch blanket at all times or thermal injury may occur.**
- **Possible fire hazard. Prevent the blanket material from coming into contact with a laser or an electrosurgical active electrode, or combustion may result.**
- **Possible burn or infection hazard. Do not allow blanket to come into contact with open wounds or mucous membranes. All patients' wounds should be covered while using the warming system.**
- **Possible patient burns. Use caution and consider discontinuing use on patients during vascular surgery when an artery to an extremity is clamped. Do not apply the warming system to ischemic limbs. Use caution and monitor closely if used on patients with severe peripheral vascular disease.**
- **Do not cover the patient's face with the head cover unless the patient is intubated and ventilated.**
- **The WarmTouch Blanket should not be used on patients who exhibit allergic reactions to adhesive tape.**
- **Use only WarmTouch blankets with the WarmTouch Convective Warming System. The performance of the WarmTouch Convective Warming System has only been evaluated and validated using WarmTouch blankets.**
- **Using the warming system on patients with transdermal medication patches may increase the rate of drug delivery, potentially causing harm to the patient.**
- **If a fault or sudden change in performance occurs in the warming system, discontinue use. Notify your sales/service center. The unit must be serviced according to the Service Manual by qualified personnel.**

CAUTIONS

- Federal (U.S.A.) law restricts the use of the warming system to sale by or on the order of a physician.
- Ensure the patient is dry or the warming system may be ineffective.
- Use WarmTouch blankets only as directed. Carefully follow these instructions regarding handling and positioning of the blanket.
- WarmTouch Upper Body Blanket is a single-patient non-sterile item.
- When handling blanket in the vicinity of the sterile field, observe proper sterile technique.
- Do not allow objects that are heavy enough to constrict airflow (such as cables) to drape over the blanket at the blower inlet.
- Do not allow sharp objects to come into contact with the blanket, as they may puncture or tear it.
- As blanket initially inflates and throughout duration of application, check that the blanket fills evenly over the entire blanket surface and replace if cuts or tears are present.
- Ensure that other blanket inlets besides the one selected for use remain unopened, to prevent warmed air from escaping the blanket.
- Continuously monitor the patient's temperature and vital signs. Reduce air temperature or discontinue therapy when normothermia is reached. Use good clinical judgment in selecting and adjusting temperature settings based on the patient's warming needs and response to treatment.
- If hypotension occurs, consider reducing air temperature or turning off the warming system.
- The WarmTouch Convective Warming System uses an air filter; however, the risk of airborne contamination should not be overlooked when the WarmTouch Convective Warming System is used. Refer to the WarmTouch Convective Warming System Service Manual for filter replacement.
- WarmTouch blankets are for use by trained personnel only.

Upper Body Blanket

The Upper Body Blanket has a lower layer that is perforated. Warm air, generated by the WarmTouch Patient Warming System blower, passes through the holes in the perforated layer to warm the patient. Along the bottom edge of the blanket are three lengths of tape that are used to secure the blanket to the patient and the operating table, and to help restrict the flow of air into the surgical site.

Plastic Warming Cover

The WarmTouch Plastic Warming Cover is an accessory consisting of a clear plastic sheet with surgical tape for use with the WarmTouch Patient Warming System. The Plastic Warming cover serves as a passive warming device that traps warm air from the WarmTouch blanket next to and around the patient.

For maximum effectiveness, use the Plastic Warming Cover to cover as much skin surface as possible while at the same time:

- maintaining access to the patient and/or surgical site
- restricting airflow into the surgical site, if applicable

Instructions for Use

1. Make sure the patient is dry, to ensure maximum effectiveness of the WarmTouch Patient Warming System.
2. Clean the patient's skin to remove oils and/or debris to maximize adhesion of the tape to the patient.
3. Position the WarmTouch Patient Warming System next to the patient and plug the warming unit into an approved AC power outlet.

Upper Body Blanket

4. Place the blanket over the patient with the perforated side next to the skin. **(1)**
5. Apply the tape to the patient's chest area to prevent air from blowing into the surgical area. To remove the three pieces of tape liner around the chest area, fold each piece of tape over at the blanket edge. The tape should be folded towards the blanket with the liner visible. Grip the exposed tab and pull the tab along the edge of the blanket, removing the liner. Do this for each piece of tape. **(2)**
6. There are three options for securing the arms of the blanket:
 - a. Fold tape over. Using the tab, remove the entire tape backing. Secure tape to the arm board.
 - b. Fold tape over. Using the tab, remove 2-3 inches of the tape. Wrap one corner of the blanket around the arm board using the 2-3 inches of tape to secure the two ends of the blanket together.
 - c. Do not utilize arm tape. Blanket will conform to patients arms.
7. Choose one of the two hose access inlets, located at the top of the blanket. To open, tear the selected inlet along the perforated line and across the notches. **(3)**
8. Insert the WarmTouch Patient Warming System nozzle into the blanket inlet. Ensure that the inlet is clamped under the nozzle clip. **(4)**
9. Connect the WarmTouch Patient Warming System nozzle blanket clip to the drape or bed sheet.
10. Turn on the WarmTouch Patient Warming System. Refer to the WarmTouch Patient Warming System operation manual.
11. Select the appropriate air temperature setting on the WarmTouch unit. Refer to the WarmTouch system operation manual.
12. Continually monitor the patient's core temperature while the WarmTouch Patient Warming System is in use.

Plastic Warming Cover (optional)

13. Peel back the tape on the Plastic Warming Cover adhesive area.
14. Position the Plastic Warming Cover over the patient's head and secure it to the warming blanket. **(5)**
15. The Plastic Warming Cover should only be used when the patient is intubated and ventilated.

NOTE: The WarmTouch blanket may be used as long as normothermia or recovery from hypothermia can be functionally achieved.

Disposal

Dispose of the blanket after single use. Disposal of the blanket should be made in accordance with applicable national regulations for biologically hazardous waste.

Storage and Shipping Conditions

Temperature -40°C to 70°C

Relative Humidity 10% to 95%

Atmospheric Pressure 120hPa to 1060hPa

Additional Copies of Instructions

Additional copies of these instructions can be obtained through your authorized distributor. In addition, permission is hereby granted under Covidien copyrights to purchasers of products obtained from your authorized distributor to make copies of these instructions for use by such purchasers.

Warranty

To obtain information about a warranty, if any, for this product, contact Covidien's Technical Services Department or your local Covidien representative.

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registered trademarks of Covidien AG.
Other brands are trademarks of a
Covidien Inc, 15 Hampshire Street,
Covidien Ireland Limited,