




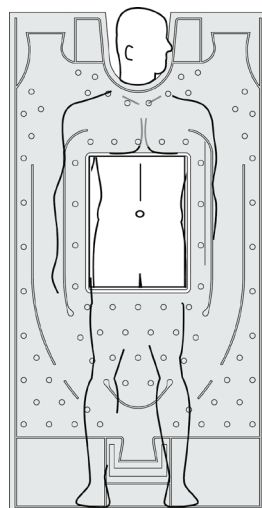
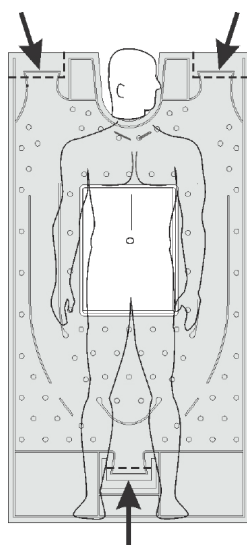
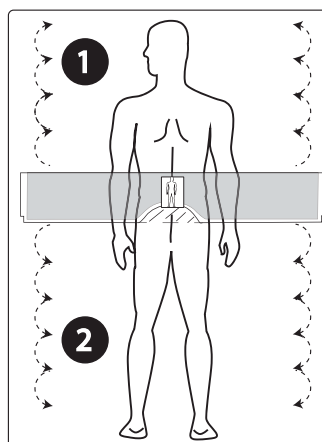
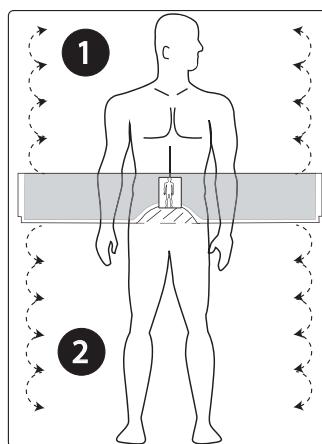
WarmTouch™


Surgical Access Blanket

en	Instructions For Use
fr	Mode d'emploi
de	Gebrauchsanweisung
nl	Gebruiksaanwijzing
it	Istruzioni per l'uso
es	Instrucciones de uso
sv	Bruksanvisning
da	Brugsanvisning
no	Bruksanvisning
fi	Käyttöohjeet
pt	Instruções de uso
ru	Инструкция по применению
zh	使用说明
pl	Instrukcja użytkowania
cs	Návod k použití
sk	Návod na použitie
sl	Navodila za uporabo
hu	Használati útmutató
el	Οδηγίες χρήσης
tr	Kullanma Talimatı
ar	إرشادات الاستعمال
bg	Указания за употреба
ro	Instrucțiuni de folosire
ja	使用説明書
et	Kasutusjuhend
lt	Naudojimo instrukcija
lv	Lietošanas norādījumi
hr	Upute za uporabu
sr	Uputstva za upotrebu

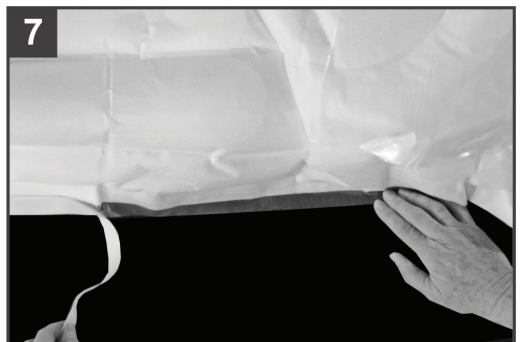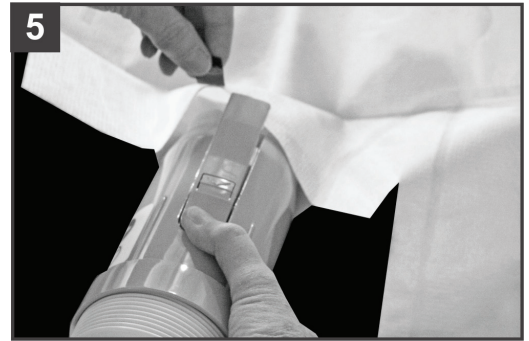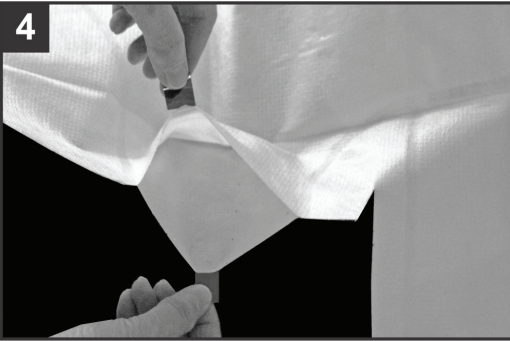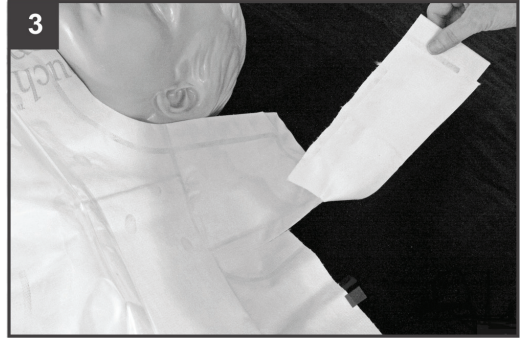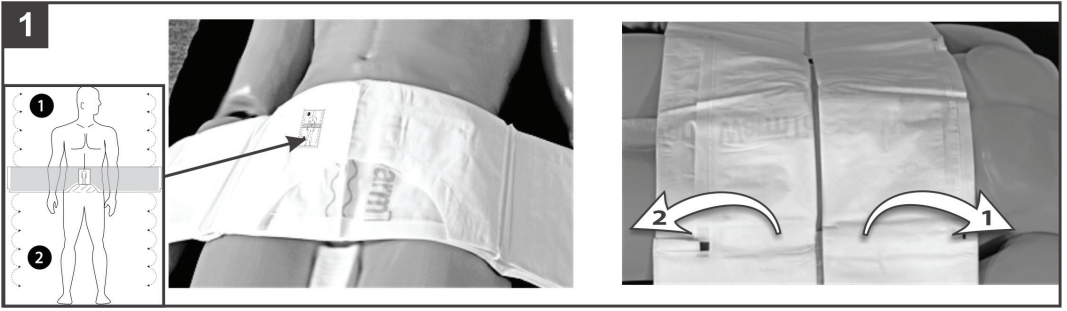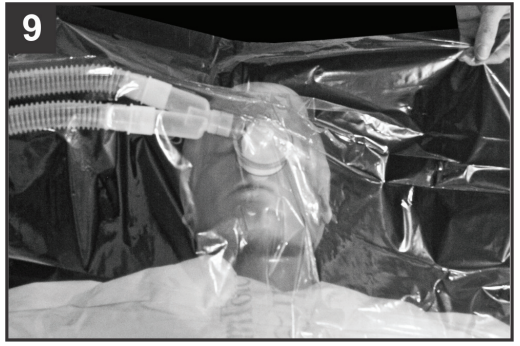WarmTouch™

Surgical Access Blanket

Identification of a substance that is not contained or present within the product or packaging.

No Free Hosing

Must consult instructions for use

Do not use for greater than 12 hours

This product cannot be adequately cleaned and/or sterilized by the user in order to facilitate safe reuse, and is therefore intended for single use. Attempts to clean or sterilize these devices may result in a bio-incompatibility, infection or product failure risks to the patient.

Directions for Use

Intended Use

The WarmTouch™ Convective Warming System is intended for prevention and treatment of hypothermia and for the management of appropriate normothermia.

The WarmTouch™ Surgical Access Blanket is intended for prevention and treatment of hypothermia and for the management of appropriate normothermia for adult and adolescent surgical patients in the peri-operative environment, when used as part of the WarmTouch™ Convective Warming System (warming unit and blanket).

The WarmTouch™ Surgical Access Blanket is to be used only with WarmTouch™ convective warming units with model numbers 5015300A, 5015800, 5015900 and 5016000.

Indications

The WarmTouch™ Surgical Access Blanket is intended for prevention and treatment of hypothermia, and for the management of appropriate normothermia. The warming system is for use only in a hospital or clinical environment, and only by trained and licensed caregivers.

Contraindications

None known.

WARNINGS

- **Possible explosion hazard. Do not use the device in the presence of flammable anesthetics or in an oxygen-rich environment.**
- **No free-hosing. Keep hose nozzle connected to the WarmTouch blanket at all times or thermal injury may occur.**
- **Possible fire hazard. Prevent the blanket material from coming into contact with a laser or an electrosurgical active electrode, or combustion may result.**
- **Possible burn or infection hazard. Do not allow blanket to come into contact with open wounds or mucous membranes. All patients' wounds should be covered while using the warming system.**
- **Possible patient burns. Use caution and consider discontinuing use on patients during vascular surgery when an artery to an extremity is clamped. Do not apply the warming system to ischemic limbs. Use caution and monitor closely if used on patients with severe peripheral vascular disease.**
- **Do not cover the patient's face with the head cover unless the patient is intubated and ventilated.**
- **The WarmTouch Surgical Access Blanket should not be used on patients who exhibit allergic reactions to adhesive tape.**
- **Use only WarmTouch blankets with the WarmTouch Convective Warming System. The performance of the WarmTouch Convective Warming System has only been evaluated and validated using WarmTouch blankets.**
- **Using the warming system on patients with transdermal medication patches may increase the rate of drug delivery, potentially causing harm to the patient.**
- **If a fault or sudden change in performance occurs in the warming system, discontinue use. Notify your sales/service center. The unit must be serviced according to the Service Manual by qualified personnel.**

CAUTIONS

- Federal (U.S.A.) law restricts the use of the warming system to sale by or on the order of a physician.
- Ensure the patient is dry or the warming system may be ineffective.
- Use WarmTouch blankets only as directed. Carefully follow these instructions regarding handling and positioning of the blanket.
- WarmTouch blankets are for use by trained personnel only.
- The WarmTouch Surgical Access Blanket is a single-patient sterile item.
- When handling blanket in the vicinity of the sterile field, observe proper sterile technique.
- Do not allow objects that are heavy enough to constrict airflow (such as cables) to drape over the blanket at the blower inlet.
- Do not allow sharp objects to come into contact with the blanket, as they may puncture or tear it.
- As the blanket initially inflates, check that it fills evenly over the entire blanket surface. Do not use if cuts or tears are present that prevent even inflation.

- Ensure that other blanket inlets besides the one selected for use remain unopened, to prevent warmed air from escaping the blanket.
- Continuously monitor the patient's temperature and vital signs.
- Reduce air temperature or discontinue therapy when normothermia is reached. Use good clinical judgment in selecting and adjusting temperature settings based on the patient's warming needs and response to treatment.
- If hypotension occurs, consider reducing air temperature or turning off the warming system.
- The WarmTouch Convective Warming System uses an air filter; however, the risk of airborne contamination should not be overlooked when the WarmTouch Convective Warming System is used. Refer to the WarmTouch Convective Warming System Service Manual for filter replacement.

Instructions for Use

1. Make sure the patient is dry, to ensure maximum effectiveness of the warming system.
2. Clean the patient's skin to remove oils or debris, to maximize adhesion of tape to the patient.
3. Position the warming unit as close as possible to the desired blanket inlet. Plug the warming unit into an approved AC power outlet.
4. Remove the blanket from its pouch, then remove the sterile wrap from the blanket. Do NOT use the blanket if sterile packaging has already been opened, or is damaged.
5. Unfold the blanket so that the positioning label is visible and facing upward, away from the patient.
6. Place the blanket on top of the patient, as shown on the positioning label. The head of the patient icon on the label should be pointing toward the head of the patient [1].
7. Unfold the blanket first toward the patient's head [1— step 1], then toward the patient's feet [1— step 2]. The blanket should now be completely unfolded, with the printed WarmTouch logos visible on the side of the blanket facing away from the patient.
8. Peel the backing from the tape at one edge of the surgical access window [2].
9. Press the tape against the patient's skin to secure the edge of the surgical access window to the patient.
10. Repeat steps 8 and 9 with the tape on the remaining three sides of the surgical access window.
11. Grip the perforated flap on the desired blanket inlet with one hand and the inlet itself with the other hand, then carefully tear off the flap [3].
12. Using the tabs on the blanket next to the inlet, pull the inlet open and hold it open [4].
13. Grip the warming unit nozzle. While holding the nozzle clip open [4], insert the nozzle into the blanket inlet [5].
14. Ensure the nozzle clip overlaps the outside of the blanket inlet, with the inlet tab to one side of the clip [6], then let go of the clip to secure the nozzle to the blanket.
15. Turn the warming unit on to inflate the blanket. Refer to the WarmTouch Convective Warming Unit Operator's Manual.
16. If securing the blanket to the operating table, peel the backing from the tape on one of the outside edges of the blanket, then press the tape against the table to secure it in place [7].
17. If securing the blanket to the table, repeat step 16 to secure the other edge of the blanket.
18. Tuck the flaps at the top of the blanket around and underneath the patient's shoulders [8].
19. If desired, and only if the patient is intubated and ventilated, apply the head cover over the patient's head [9].
20. Set appropriate temperature on the warming unit. Continually monitor patient's temperature while the warming system is in use , and reduce air temperature or discontinue therapy when normothermia is reached.

NOTE: Do not use the WarmTouch™ Surgical Access Blanket for longer than 12 hours.

Disposal

Dispose of the blanket after a single use. Perform disposal in accordance with applicable national regulations for biologically hazardous waste.

Storage and Shipping Conditions

	Storage	Shipping
Temperature	0°C to 32°C	-40°C to 70°C
Relative Humidity	30% to 75%	15% to 85%
Atmospheric Pressure	700hPa to 1060 hPa	500 hPa to 1060 hPa

Additional Copies of Instructions

Additional copies of these instructions can be obtained through your authorized distributor. In addition, permission is hereby granted under Covidien copyrights to purchasers of products obtained from your authorized distributor to make copies of these instructions for use by such purchasers.

Warranty

To obtain information about a warranty, if any, for this product, contact Covidien's Technical Services Department, or your local Covidien representative.

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registered trademarks of Covidien AG.
Other brands are trademarks of a
Covidien LLC, 15 Hampshire Street,
Covidien Ireland Limited,