

WarmTouch™

Torso Blanket


en	Instructions For Use
fr	Mode d'emploi
de	Gebrauchsanweisung
nl	Gebruiksaanwijzing
it	Istruzioni per l'uso
es	Instrucciones de uso
sv	Bruksanvisning
da	Betjeningsvejledning
no	Bruksanvisning
fi	Käyttöohjeet
pt	Instruções de utilização
ru	Инструкции по применению
zh	使用说明
pl	Instrukcja użytkowania
cs	Návod k použití
sk	Návod na použitie
sl	Navodilo za uporabo
hu	Használati útmutató
el	Οδηγίες χρήσης
tr	Kullanma Talimatları
ar	إرشادات الاستعمال
bg	Инструкции за употреба
ro	Instrucțiuni de utilizare
et	Kasutusjuhend
lt	Naudojimo instrukcija
lv	Lietošanas instrukcija
hr	Upute za uporabu
sr	Uputstva za upotrebu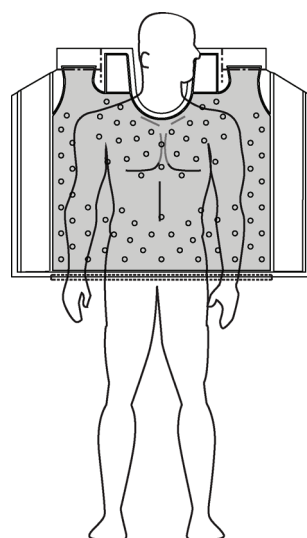


1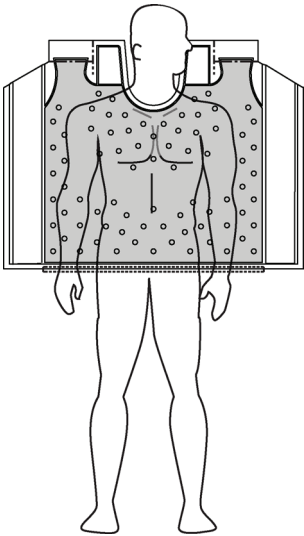


2

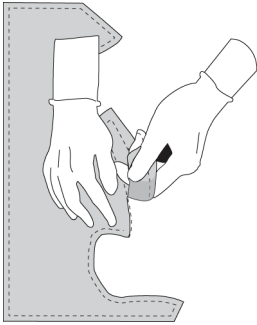
3

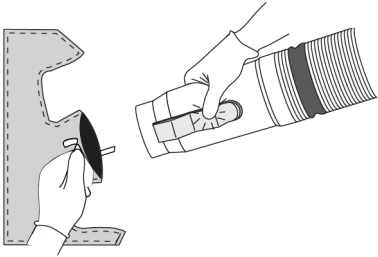
4

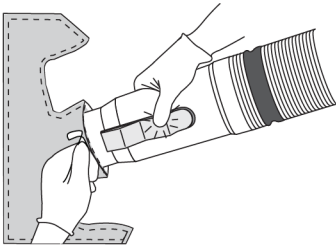5

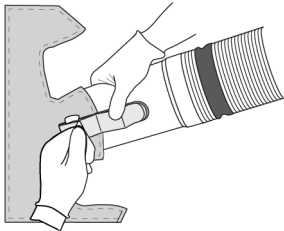
6

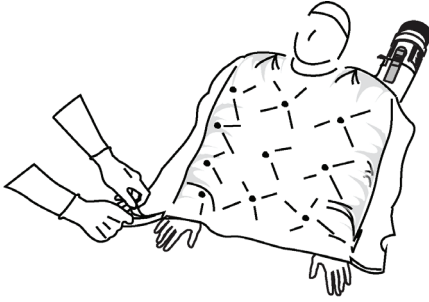7

8

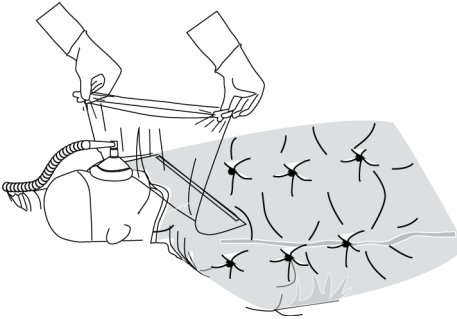Page left intentionally blank

WarmTouch™

Torso Blanket

Identification of a substance that is not contained or present within the product or packaging

No Free Hosing

Must consult instructions for use

This product cannot be adequately cleaned and/or sterilized by the user in order to facilitate safe reuse, and is therefore intended for single use. Attempts to clean or sterilize these devices may result in a bio-incompatibility, infection or product failure risks to the patient.

Directions for Use

Intended Use

The WarmTouch™ Convective Warming System is intended for prevention and treatment of hypothermia and for the management of appropriate normothermia.

The WarmTouch™ Torso Blanket is intended for prevention and treatment of hypothermia and for the management of appropriate normothermia for adult and adolescent surgical patients in the peri-operative environment, when used as part of the WarmTouch™ Convective Warming System (warming unit and blanket).

The WarmTouch™ Torso Blanket is to be used only with WarmTouch™ convective warming units with model numbers 5015300A, 5015800, 5015900 and 5016000.

Indications

The WarmTouch™ Torso Blanket is intended for prevention and treatment of hypothermia, and for the management of appropriate normothermia. The warming system is for use only in a hospital or clinical environment, and only by trained and licensed caregivers.

Contraindications

None known.

WARNINGS

- **Possible explosion hazard. Do not use the device in the presence of flammable anesthetics or in an oxygen-rich environment.**
- **No free-hosing. Keep hose nozzle connected to the WarmTouch blanket at all times or thermal injury may occur.**
- **Possible fire hazard. Prevent the blanket material from coming into contact with a laser or an electrosurgical active electrode, or combustion may result.**
- **Possible burn or infection hazard. Do not allow blanket to come into contact with open wounds or mucous membranes. All patients' wounds should be covered while using the warming system.**
- **Possible patient burns. Use caution and consider discontinuing use on patients during vascular surgery when an artery to an extremity is clamped. Do not apply the warming system to ischemic limbs. Use caution and monitor closely if used on patients with severe peripheral vascular disease.**
- **Do not cover the patient's face with the head cover unless the patient is intubated and ventilated.**
- **The WarmTouch Torso Blanket should not be used on patients who exhibit allergic reactions to adhesive tape.**
- **Use only WarmTouch blankets with the WarmTouch Convective Warming System. The performance of the WarmTouch Convective Warming System has only been evaluated and validated using WarmTouch blankets.**
- **Using the warming system on patients with transdermal medication patches may increase the rate of drug delivery, potentially causing harm to the patient.**
- **If a fault or sudden change in performance occurs in the warming system, discontinue use. Notify your sales/service center. The unit must be serviced according to the Service Manual by qualified personnel.**

CAUTIONS

- Federal (U.S.A.) law restricts the use of the warming system to sale by or on the order of a physician.
- Ensure the patient is dry or the warming system may be ineffective.
- Use WarmTouch blankets only as directed. Carefully follow these instructions regarding handling and positioning of the blanket.
- WarmTouch Torso Blanket is a single-patient non-sterile item.
- When handling blanket in the vicinity of the sterile field, observe proper sterile technique.
- Do not allow objects that are heavy enough to constrict airflow (such as cables) to drape over the blanket at the blower inlet.
- Do not allow sharp objects to come into contact with the blanket, as they may puncture or tear it.

- As blanket initially inflates and throughout duration of application, check that the blanket fills evenly over the entire blanket surface and replace if cuts or tears are present.
- Ensure that other blanket inlets besides the one selected for use remain unopened, to prevent warmed air from escaping the blanket.
- Continuously monitor the patient's temperature and vital signs. Reduce air temperature or discontinue therapy when normothermia is reached. Use good clinical judgment in selecting and adjusting temperature settings based on the patient's warming needs and response to treatment.
- If hypotension occurs, consider reducing air temperature or turning off the warming system.
- The WarmTouch Convective Warming System uses an air filter; however, the risk of airborne contamination should not be overlooked when the WarmTouch Convective Warming System is used. Refer to the WarmTouch Convective Warming System Service Manual for filter replacement.
- WarmTouch blankets are for use by trained personnel only.

Instructions for Use

1. Make sure the patient is dry, to ensure maximum effectiveness of the warming system.
2. Clean the patient's skin to remove oils or debris, to maximize adhesion of tape to the patient.
3. Position the warming unit at the head of the patient (under the head of the operating table, outside any sterile area), as close as possible to the desired blanket inlet. Plug the warming unit into an approved AC power outlet.
NOTE: The Torso Blanket is not sterile. Use caution to avoid any sterile area when opening, unfolding and positioning the blanket.
4. Remove the blanket and head cover from the pouch. Do NOT use the blanket if packaging is damaged.
5. Unfold the blanket and place it over the patient, with the U-shaped cutout at the patient's neck [1], and the perforated bottom layer facing downward, touching the patient's skin.
6. Select the blanket inlet to be used. Grip the perforated flap on the desired blanket inlet with one hand and the inlet itself with the other hand, then carefully tear off the flap [2].
7. Using the tabs on the blanket next to the inlet, pull the inlet open and hold it open [3].
NOTE: The warming unit and nozzle are not sterile. Use caution to avoid any sterile area when connecting the warming unit to the blanket.
8. Grip the warming unit nozzle. While holding the nozzle clip open [3], insert the nozzle into the blanket inlet [4].
9. Ensure the nozzle clip overlaps the outside of the blanket inlet, with the inlet tab to one side of the clip [5], then let go of the clip to secure the nozzle to the blanket.
10. Peel the backing from one of the lengths of tape on the bottom edge of the blanket [6].
11. Press the tape against the patient's skin to secure that part of the bottom edge to the patient, so that warmed air does not escape below the tape.
12. Repeat steps 10 and 11 with the other length of tape on the bottom edge of the blanket.
13. Turn the warming unit on to inflate the blanket. Refer to the WarmTouch Convective Warming Unit Operator's Manual.
14. Tuck the flaps at the top of the blanket around and underneath the patient's shoulders.
15. Tuck the flaps on each side of the blanket underneath the patient [7].
16. If desired, and only if the patient is intubated and ventilated, apply the head cover over the patient's head [8].
17. Set appropriate temperature on the warming unit. Continually monitor patient's temperature while the warming system is in use, and reduce air temperature or discontinue therapy when normothermia is reached.
NOTE: The WarmTouch blanket may be used as long as normothermia or recovery from hypothermia can be functionally achieved.

Disposal

Dispose of the blanket after a single use. Perform disposal in accordance with applicable national regulations for biologically hazardous waste.

Storage and Shipping Conditions

Temperature	-40°C to 70°C
Relative Humidity	10% to 95%
Atmospheric Pressure	120hPa to 1060hPa

Additional Copies of Instructions

Additional copies of these instructions can be obtained through your authorized distributor. In addition, permission is hereby granted under Covidien copyrights to purchasers of products obtained from your authorized distributor to make copies of these instructions for use by such purchasers.

Warranty

To obtain information about a warranty, if any, for this product, contact Covidien's Technical Services Department, or your local Covidien representative.

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registered trademarks of Covidien AG.
Other brands are trademarks of a
Covidien Inc, 15 Hampshire Street,
Covidien Ireland Limited,