
# CALIMA™

## Pediatric Blanket

|    |                           |
|----|---------------------------|
| en | Instructions For Use      |
| es | Instrucciones de uso      |
| fr | Mode d'emploi             |
| de | Gebrauchsanweisung        |
| nl | Indicaties voor gebruik   |
| it | Istruzioni per l'uso      |
| sv | Bruksanvisning            |
| da | Betjeningsvejledning      |
| no | Bruksanvisning            |
| fi | Käyttöohjeet              |
| pt | Instruções de utilização  |
| pl | Instrukcja użytkowania    |
| cs | Návod k použití           |
| sk | Návod na použitie         |
| sl | Navodilo za uporabo       |
| hu | Használati útmutató       |
| el | Οδηγίες χρήσης            |
| tr | Kullanma Talimatları      |
| bg | Инструкции за употреба    |
| ro | Instrucțiuni de utilizare |
| et | Kasutusjuhend             |
| lt | Naudojimo instrukcija     |
| lv | Lietošanas instrukcija    |
| hr | Upute za uporabu          |
| sr | Uputstva za upotrebu      |
| uk | Показання до застосування |
| mk | Индикации за употреба     |
| ru | Инструкции по применению  |
| kk | Пайдалану айғақтары       |
| vi | Chỉ định sử dụng          |
| id | Indikasi untuk Digunakan  |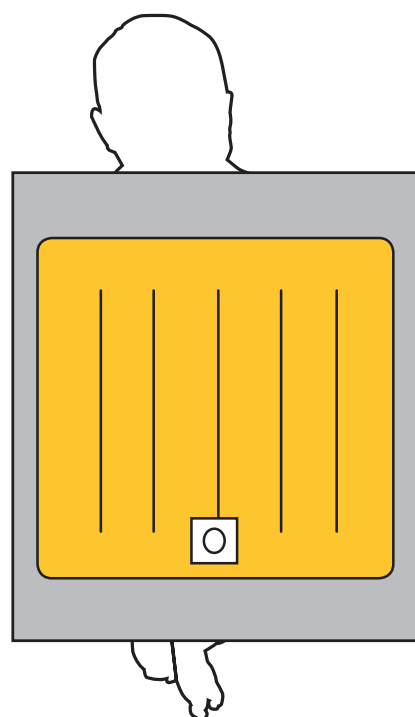

# CALIMA™

## Pediatric Blanket

|                                                                                                                          |                                                                                                                                         |                                                                                                                                      |                                                                                                                             |
|--------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------|
|  Manufacturer.                           |  Contains unique device identification information.    |  Intended for one single use only.                  |  Not subject to a sterilization process. |
|  Manufacturer's catalogue number.        |  Need for the user to consult the instruction for use. |  Not use if the package has been damaged or opened. |                                                                                                                             |
|  Indicates the item is a medical device. |  CE marking of conformity.                             |  Latex free.                                        |                                                                                                                             |
|  Manufacturer's batch code.              |  Prescription Only.                                    |  Not be used directly on the patient.               |                                                                                                                             |

This product intended for one single use only, therefore shall not be cleaned and/or sterilized by the user in order to facilitate safe reuse. Attempts to cleans or sterilize these devices may result in bio-incompatibility, infection, or product failure risks to the patient.

### INDICATIONS FOR USE

The CALIMA™ Thermal Regulating Warming System (warming unit and blanket) is intended for prevention and treatment of hypothermia, and for the management of appropriate normothermia

CALIMA™ blankets can be used for both adult and pediatric patients.

CALIMA™ devices should only be used by qualified healthcare personnel.

CALIMA™ blankets are designed for a single use per patient; they should never be reused.

Use the CALIMA™ blanket model best suited to the clinical need, requirements, and surgical protocols.

### CONTRAINDICATIONS

Do not use on the lower limbs during transverse clamping of the aorta.

Do not use on ischemic extremities. Thermal injuries may occur.

### WARNINGS

- Do not apply hot air directly (without an approved blanket) from the CALIMA™ heater hose on the patient.
- Only use CALIMA™ blankets or compatible approved blankets.
- Avoid direct contact between the outside of the hose and the patient.
- Never leave the patient unattended during treatment.
- Do not use the CALIMA™ blanket in patients with any type of allergy to the device or its components.
- The use of the CALIMA™ convection warming system in patients with transdermal drug patches could alter the administration of the drugs and cause harm to the patient.
- In the event of any failure, malfunction, or abnormal performance of the CALIMA™ heating system, switch it off, disconnect the heating unit from the power supply, and notify Technical Services.
- Always place the microperforated side of the CALIMA™ blanket towards the patient for optimal performance.
- Do not use CALIMA™ blankets to move or transfer the patient.
- Never cover the patient's head or airway when the patient is not receiving mechanical ventilation.
- Use securing elements, such as fixing clamps, or the ones specific to each blanket, such as pre-cut tapes, adhesive tapes, etc., to secure and stabilize the CALIMA™ blanket to adapt it to the patient's position.
- According to U.S. federal law, this type of device can only be used by order of a licensed physician.
- Any serious incident that has occurred in relation to the CALIMA™ devices or blankets disposables, should be reported to the manufacturer and the competent authority of the Member State in which the user is established.

### CAUTIONS

- Never apply heat to diseased or damaged skin, open wounds, or mucosa.
- Wounds should be covered to avoid direct contact with the blanket, and possible infection and thermal injury.

- Avoid contact with laser elements, surgical plates, or electrodes that can cause combustion.
- Never collapse the hose in a position that prevents normal supply of air to the blanket.
- If low blood pressure occurs during the procedure, consider lowering the working temperature or discontinuing treatment.
- The CALIMA™ blanket is designed to be placed directly under, on, or around the patient. Placing sheets between the blanket and the patient does not prevent biological cross-contamination.
- Do not use if the packaging is open or damaged.
- Never place the nozzle and/or air intake port next to the patient's groin or other sensitive areas.
- Ensure that the patient is dry for the treatment to be effective, except if the objective is to lower the patient's temperature.
- Avoid folding the blanket for efficient heat distribution.
- Use the appropriate blanket model for each need, trying to cover as much skin surface as possible, ensuring comfortable access to the patient and leaving the head and airway of the patient uncovered at all times.
- Inspect the blanket at the beginning of treatment for proper placement and performance.
- Try to monitor the patient's temperature according to the official protocols established during treatment.

## INSTRUCTIONS FOR USE

1. Determine the most appropriate blanket for the type of intervention and patient position in order to get the right balance between device performance and clinical care.
2. Clean the patient's skin to remove oils or debris, to maximize adhesion of tape to the patient.
3. Place the CALIMA™ heating unit close to the patient, allowing easy and comfortable access to the patient, and connect the power cord to the electrical mains.
4. Remove the packaging of the corresponding blanket, unfold the blanket, and place it in position, checking that the microperforated side is facing the patient and that the patient's head is not covered. Dispose of the blanket packaging according to the corresponding waste disposal protocol.
5. Secure the adhesive tapes to the patient or operating table according to the blanket model, use the pre-cuts of the corresponding model to position the patient, tear the pre-cut tapes, and secure them to the patient if necessary.
6. Secure the plastic protectors in their case according to use and model.
7. Choose the port to connect the nozzle to the blanket, remove the seal, separate the two layers of the blanket so that the nozzle can be inserted as far as possible, insert the hose nozzle by operating the clamp and clamp it to the top layer of the blanket.
8. Use the hose clamp to stabilize the system connection.
9. Once the blanket is connected and secured, turn on the switch of the CALIMA warming unit.
10. Select the appropriate temperature level and the desired power deemed necessary for the specific case, according to the CALIMA™ user manual, and the blanket should start to inflate, if does not inflate check the correct working of the CALIMA™ or the correct connection of the CALIMA™ to the blanket, or that the hose is not block.
11. Start the treatment and, if necessary, modify the temperature and power settings as needed.

## WASTE

Dispose of the blanket after a single use. Perform disposal in accordance with applicable national regulations for biologically hazardous waste.

## STORAGE AND SHIPPING CONDITIONS

|                      |                   |
|----------------------|-------------------|
| Temperature          | -40°C to 70°C     |
| Relative Humidity    | 10% to 95%        |
| Atmospheric Pressure | 120hPa to 1060hPa |

## ADDITIONAL COPIES OF INSTRUCTIONS

Additional copies of these instructions can be obtained through your authorized distributor. In addition, permission is hereby granted under Celsius Medical copyrights to purchasers of products obtained from your authorized distributor to make copies of these instructions for use by such purchasers.

## WARRANTY

To obtain information about warranty, if any, for this product, contact Celsius Medical Technical Services Department.

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CALIMA™ and Celsius Medical logo are U.S. and internationally registered trademarks of Celsius Medical, S.L