
FR - Lames de laryngoscope réutilisables
 EN - Reusable laryngoscope blades
 DE - Wiederverwendbare laryngoskopspatel
 ES - Hojas de laringoscopia reutilizables
 IT - Lame di laringoscopia riutilizzabili
 NL - Herbruikbare laryngoskopspatel
 PT - Lâmi,as de laringoscópio reutilizável

LRMAC0 - LRMAC1 - LRMAC2
LRMAC3 - LRMAC4 - LRMAC5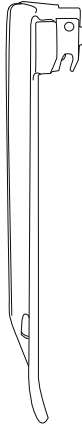

LRMIL00 - LRMIL0 - LRMIL1
LRMIL2 - LRMIL3 - LRMIL4

comepa
 INDUSTRIES

35, rue du Sausset - 93290 Tremblay-en-France - France
 Tel. : +33 (0)1 42 43 50 50 - Fax : +33 (0)1 42 43 64 44
 info@comepa.com - www.comepa.com

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REUSABLE LARYNGOSCOPE BLADE

1. Indications

The laryngoscope blade is used to visualise the patient's upper airway and is designed to facilitate access to the trachea in the setting of endotracheal intubation.

2. Precautions and warnings

The reusable laryngoscope blade should be used by persons with training corresponding to this medical method of intubation. The reusable laryngoscope blade should be attached to a green ring compatible handle according to specification ISO7376. The reusable laryngoscope blade should be used only after disinfection and/or sterilisation. Use of a non-sterile blade can result in risk of cross-infection. It is important to comply with the care institution's internal procedures on use of reusable instruments. During the cleaning procedure, avoid placing the blade in ordinary water for a too long duration or in contact with a saline and moist atmosphere. This can cause corrosion. During the cleaning/disinfection procedure, use only products recommended by their manufacturer for use with stainless steel and fibre optic equipment. In particular, follow the manufacturer's instructions. Use only demineralised water or solutions with pH 7.0. During the cleaning/disinfection/sterilisation procedure, make certain to not mix laryngoscope blades with other reusable metallic surgical instruments.

3. Instructions for use: Cleaning/Disinfection/Sterilisation

Cleaning :

- Immediately after use, remove any residue and traces of material, for example, using a pliable nylon brush.
- Soak reusable blades in solutions with a concentration and for a duration recommended by the manufacturer.
- After soaking, dry blade thoroughly and verify that no residue remains on the blade.

Disinfection :

- The blade can be disinfected by soaking it in appropriate solutions (neutral pH) or by processing in an appropriate apparatus at a maximum temperature of 65°C.
- The last rinse should be performed with demineralised water.
- Blade can be disinfected by thermal process with max. temperature of 95°C.
- After disinfection, dry blade thoroughly and verify that no residue remains on the blade.

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- The blade can be sterilised in an autoclave at a maximum temperature of 134°C during 5 minutes (the recommended cycle).
- An 18 minute cycle at 134°C can also be used on the product. However, the fibre optic component will deteriorate more quickly (the lifespan will be reduced by a factor of 3). A premature decrease in transmission of light can be observed.

4. Contraindications

Do not use blades if corroded. Do not clean with ultrasound. Any processing with ultrasound will damage the blade irreversibly. Never immerse the blade in a saline solution or in any detergent solution containing hydrogen peroxide (example: Cidex PA, Sporox, Virox3, Javel water) or halide ions (example: chloride and iodine) or any other caustic ingredient (example: sodium hydroxide). Do not immerse in Betadine or in potassium hydroxide. Do not place the reusable laryngoscope blade in the same bath or in the same basket as other metallic surgical instruments during cleaning – and disinfection – and sterilisation. This practice can result in chemical reactions which cause damage to all the instruments. Do not use emergency sterilisation (Flash). This type of sterilisation can deteriorate the fibre optic component.

5. Warranty

These operations should be performed by trained and qualified staff.

The reusable blade is guaranteed for a 1 year period under normal conditions of use starting from date of purchase. All complaints should include return of the defective parts and proof of proper application of the cleaning – disinfection – sterilisation procedure.

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