# LDH 21 FS\*

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Référence	Composition du kit		
1 4251 99 10 021	R1 5 x 20 mL	+ R2	1 x 25 mL
1 4251 99 10 704	R1 8 x 50 mL	+ R2	8 x 12,5 mL
1 4251 99 10 930	R1 4 x 20 mL	+ R2	2 x 10 mL

## Emploi Prévu

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1. Thomas L. Clinical laboratory diagnostics. 1st ed. Frankfurt: TH-Books Verlagsgesellschaft; 1998. p. 89-94.
2. Moss DW, Henderson AR. Clinical enzymology In: Burtis CA, Ashwood ER, editors. Tietz Textbook of Clinical Chemistry. 3rd ed. Philadelphia: W.B Saunders Company; 1999. 617-721.
3. Bakker AJ, Mücke M. Gammopathy interference in clinical chemistry assays: Mechanisms, detection and prevention. Clin Chem Lab med 2007; 45(9): 1240-1243.
4. Guder WG, da Fonseca-Wollheim F, Heil W, et al. The Quality of Diagnostic Samples. 3rd ed. Darmstadt: GIT Verlag; 2010. p. 52-3.
5. Young DS. Effects of Drugs on Clinical Laboratory Tests. 5th ed. Volume 1 and 2. Washington, DC: The American Association for Clinical Chemistry Press 2000.
6. Young DS. Effects on Clinical Laboratory Tests - Drugs Disease, Herbs & Natural Products, <https://clinf.wiley.com/aaccweb/aacc/>, accessed in March 2021. Published by AACC Press and John Wiley and Sons, Inc.
7. Schumann G, Bonora R, Ceriotti F, Féraud G et al. IFCC primary reference procedure for the measurement of catalytic activity concentrations of enzymes at 37°C. Part 3: Reference procedure for the measurement of catalytic concentration of lactate dehydrogenase. Clin Chem Lab Med 2002;40:643-48.
8. Soldin SJ, Brugnara C, Wong EC. Pediatric reference ranges. 6th Edition. Washington: AACC Press; 2007:140.

DiaSys Diagnostic Systems GmbH  
Alte Strasse 9 65558 Holzheim Allemagne  
[www.diasys-diagnostics.com](http://www.diasys-diagnostics.com)

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1. Thomas L. Clinical laboratory diagnostics. 1st ed. Frankfurt: TH-Books Verlagsgesellschaft; 1998. p. 89-94.
2. Moss DW, Henderson AR. Clinical enzymology In: Burtis CA, Ashwood ER, editors. Tietz Textbook of Clinical Chemistry. 3rd ed. Philadelphia: W.B Saunders Company;1999. 617-721.
3. Bakker AJ, Mücke M. Gammopathy interference in clinical chemistry assays: Mechanisms, detection and prevention. Clin Chem Lab med 2007; 45(9): 1240-1243.
4. Guder WG, da Fonseca-Wollheim F, Heil W, et al. The Quality of Diagnostic Samples. 3rd ed. Darmstadt: GIT Verlag; 2010. p. 52-3.
5. Young DS. Effects of Drugs on Clinical Laboratory Tests. 5th ed. Volume 1 and 2. Washington, DC: The American Association for Clinical Chemistry Press 2000.
6. Young DS. Effects on Clinical Laboratory Tests - Drugs Disease, Herbs & Natural Products, <https://clinfo.wiley.com/aaccweb/aacc/>, accessed in March 2021. Published by AACC Press and John Wiley and Sons, Inc.
7. Schumann G, Bonora R, Ceriotti F, Férard G et al. IFCC primary reference procedure for the measurement of catalytic activity concentrations of enzymes at 37°C. Part 3: Reference procedure for the measurement of catalytic concentration of lactate dehydrogenase. Clin Chem Lab Med 2002;40:643-48.
8. Soldin SJ, Brugnara C, Wong EC. Pediatric reference ranges. 6th Edition. Washington: AACC Press;2007:140.

DiaSys Diagnostic Systems GmbH  
Alte Strasse 9 65558 Holzheim Allemagne  
[www.diasys-diagnostics.com](http://www.diasys-diagnostics.com)

\* Fluid Stable = Liquide & Stable

## LDH 21 FS

### Application for serum and plasma samples

This application was set up and evaluated by DiaSys. It is based on the standard equipment at that time and does not apply to any equipment modifications undertaken by unqualified personnel.

Identification	
This method is usable for analysis:	Yes
Twin reaction:	No
Name:	LDH21
Shortcut:	
Reagent barcode reference:	074
Host reference:	

Technic	
Type:	Linear Kinetic
First reagent:[μL]	160
Blank reagent	Yes
Sensitive to light	
Second reagent:[μL]	40
Blank reagent	No
Sensitive to light	
Main wavelength:[nm]	340
Secondary wavelength:[nm]	405
Polychromatic factor:	1.000
1 st reading time [min:sec]	06:24
Last reading time [min:sec]	09:48
Reaction way:	Increasing
Linear Kinetics	
Substrate depletion: Absorbance li	0.6000
Linearity: Maximum deviation [%]	100
Fixed Time Kinetics	
Substrate depletion: Absorbance limit	
Endpoint	
Stability: Largest remaining slope	
Prozone Limit [%]	

Reagents	
Decimals	
Units	

Sample	
Diluent	DIL A (NaCl)
Hemolysis:	
Agent [μL]	0 (no hemolysis)
Cleaner	
Sample [μL]	0
Technical limits	
Concentration technical limits-Lower	35
Concentration technical limits-Upper	1500
SERUM	
Normal volume [μL]	3
Normal dilution (factor)	1
Below normal volume [μL]	
Below normal dilution (factor)	1
Above normal volume [μL]	3
Above normal dilution (factor)	6
URIN	
Normal volume [μL]	3
Normal dilution (factor)	1
Below normal volume [μL]	
Below normal dilution (factor)	1
Above normal volume [μL]	3
Above normal dilution (factor)	6
PLASMA	
Normal volume [μL]	3
Normal dilution (factor)	1
Below normal volume [μL]	
Below normal dilution (factor)	1
Above normal volume [μL]	3
Above normal dilution (factor)	6
CSF	
Normal volume [μL]	3
Normal dilution (factor)	1
Below normal volume [μL]	
Below normal dilution (factor)	1
Above normal volume [μL]	3
Above normal dilution (factor)	6
Whole blood	
Normal volume [μL]	3
Normal dilution (factor)	1
Below normal volume [μL]	
Below normal dilution (factor)	1
Above normal volume [μL]	3
Above normal dilution (factor)	6

Results	
Decimals	1
Units	U/L
Correlation factor-Offset	0.000
Correlation factor-Slope	1.000

Range	
Gender	Male
Age	
SERUM	>= <=248
URINE	
PLASMA	>= <=248
CSF	
Whole blood	
Gender	Female
Age	
SERUM	>= <=247
URINE	
PLASMA	>= <=247
CSF	
Whole blood	

Contaminants	
Please refer to r910 Carryover Pair Table	

Calibrators details	
Calibrator list	Concentration
Cal. 1/Blank	0
Cal. 2	*
Cal. 3	
Cal. 4	
Cal. 5	
Cal. 6	
	Max delta abs.
Cal. 1	0.002
Cal. 2	0.004
Cal. 3	
Cal. 4	
Cal. 5	
Cal. 6	
Drift limit [%]	0.8

Calculations	
Model	X
Degree	1

\* Enter calibrator value